



MAGLUMI[®] System Tubing Cleaning Solution

INTENDED USE

The System Tubing Cleaning Solution is used for the maintenance of the MAGLUMI series Fully-auto chemiluminescence immunoassay analyzer and Biolumi series Integrated System.

SUMMARY


The System Tubing Cleaning Solution is required when performing maintenance of the MAGLUMI series Fully-auto chemiluminescence immunoassay analyzer and Biolumi series Integrated System to clean and maintain the tubes by dissolving protein precipitate.

MATERIALS PROVIDED

REF	Description	Specification
130299007M	Sodium hypochlorite (2.9% Cl active)	1x500 mL
The System Tubing Cleaning Solution is ready-for-use.		

WARNINGS AND PRECAUTIONS

- For *in vitro* diagnostic use.
- For professional use only.
- Do not use the product beyond the expiration date indicated on the label.
- Exercise the normal precautions required for handling all laboratory reagents.
- Avoid ingestion and contact with skin and eyes, if you unfortunately experience it, please drink or rinse with plenty of water, and seek emergency medical treatment.
- All waste should be considered potentially infectious and should be disposed of in accordance with local guidelines.
- Safety data sheets are available on request.
- Note: If any serious incident has occurred in relation to the device, please report to Shenzhen New Industries Biomedical Engineering Co., Ltd. (Snibe) or our authorized representative and the competent authority of the Member State in which you are established.
- Pursuant to EC Regulation 1272/2008 (CLP) hazardous reagents are classified and labeled as follows:

REAGENTS:	System tubing cleaning solution
CLASSIFICATION:	Skin Irritant, Category 2
SIGNAL WORD:	Warning
SYMBOLS / PICTOGRAMS:	 GHS07 Exclamation mark
HAZARD STATEMENTS:	H315 Causes skin irritation.
PRECAUTIONARY STATEMENTS:	P264: Wash hands thoroughly after handling. P280: Wear protective gloves/protective clothing/eye protection/face protection. P302+P352: IF ON SKIN: Wash with plenty of soap and water. P362: Take off contaminated clothing.
CONTAINS: (only substances prescribed pursuant to Article 18 of EC Regulation 1272/2008).	Sodium hypochlorite, solution 2.9% Cl active.

PROCEDURE

Materials Provided

System Tubing Cleaning Solution.

Materials Required (But Not Provided)

Fully-auto chemiluminescence immunoassay analyzer Maglumi 600, Maglumi 800, Maglumi 1000, Maglumi 1000 Plus, Maglumi 2000, Maglumi 2000 Plus, Maglumi 4000, Maglumi 4000 Plus, MAGLUMI X3, MAGLUMI X6 and MAGLUMI X8, or Integrated System Biolumi 8000 and Biolumi CX8.

Handling

- Insert the empty reagent integral filled with MAGLUMI System Tubing Cleaning Solution integrally in















the reagent area track 1.

- Run the tubing cleaning program in the software.
- For one washing cycle: Transfer the solution from MAGLUMI System Tubing Cleaning Solution bottle into the 3 blank reagent integral vials near the kit's handle. The reagent integral is added with 13 mL liquid each time.
- For two washing cycles: Transfer the solution from MAGLUMI System Tubing Cleaning Solution bottle into the 3 blank reagent integral vials near the kit's handle. The reagent integral is added with 25 mL liquid each time.
- The whole procedure needs about 40 minutes.
- For further information, please refer to the appropriate Operators Manual.

STORAGE AND STABILITY

- Unopened at 2-30°C: until the expiry date as stated on the labels.
- Opened at 2-30°C: 6 months.
- Keep container tightly closed when not in use.
- Protect from direct sunlight.

SYMBOLS EXPLANATIONS

	Consult instructions for use		Manufacturer
	Temperature limit (Store at 2-30°C)		Keep away from sunlight
	Catalogue number		Use-by date
	<i>In vitro</i> diagnostic medical device		Batch code
	Authorized representative in the European Community		Kit component
	CE marking		This way up
	Unique device identifier		Importer

MAGLUMI® and Biolumi® are trademarks of Snibe. All other product names and trademarks are the property of their respective owners.




Shenzhen New Industries Biomedical Engineering Co., Ltd.

No.23, Jinxiu East Road, Pingshan District, 518122 Shenzhen, P.R. China

Tel: +86-755-21536601

Fax: +86-755-28292740

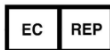

Shanghai International Holding Corp. GmbH (Europe)

Eiffestrasse 80, 20537 Hamburg, Germany

Tel: +49-40-2513175

Fax: +49-40-255726