

INSTRUCTION FOR USE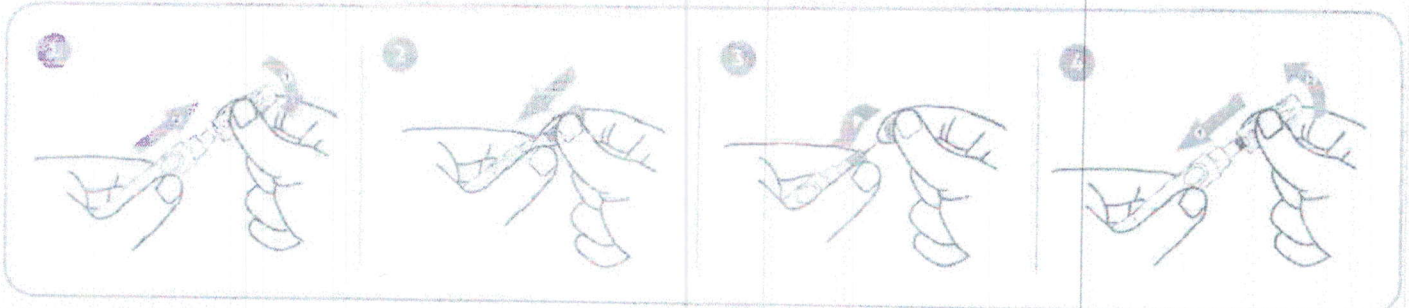


1 Twist and pull off the cap of the lancing device.

2 Insert a lancet into the lancet holder and push down firmly until it is fully secured.

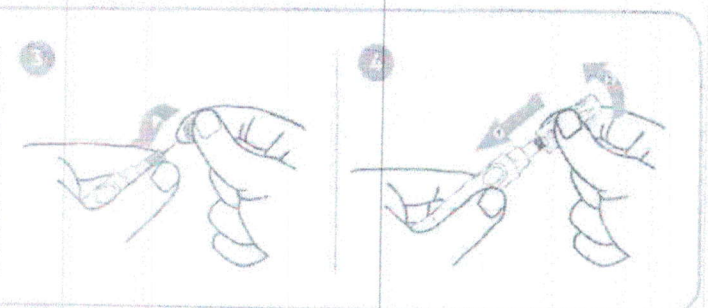3 Twist the protective disk off the lancet.

4 Replace and twist the cap as shown in the picture.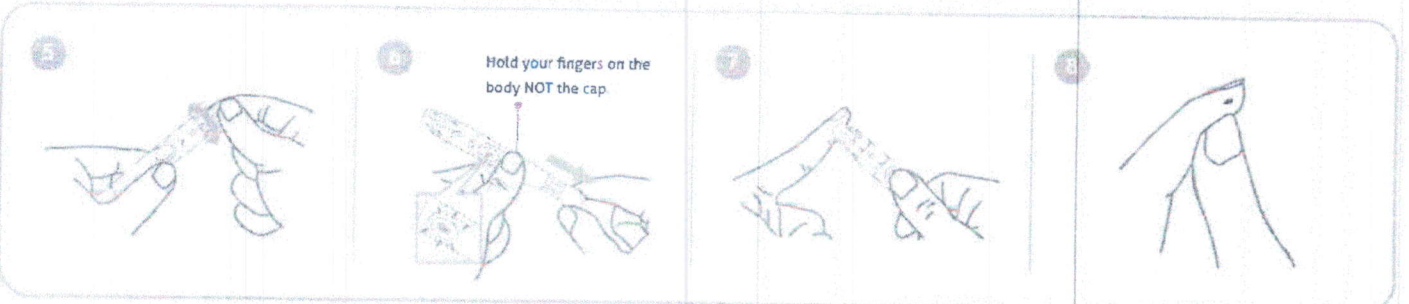


5 Select the depth of penetration by turning the adjustable tip in either direction so that the arrow on the cap points to the desired depth.

6 Pull the cocking control back until it clicks. You will see a color change inside the release button when it is ready.

7 Lancing the finger
Press the lancing device's tip firmly against the lower side of your fingertip. Press the release button to prick your finger, then a click indicates that the puncture is complete.

8 Remove the device from the puncture site. After penetration, remove the first drop of blood with a clean cotton swab. Gently squeeze the punctured area to obtain another drop of blood. Be careful NOT to smear the blood sample. Apply the blood to the test strip.

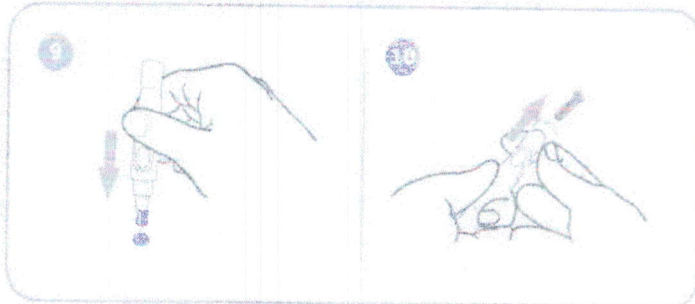9 Remove the lancet by pulling the cap off first. Safely dispose of the used lancet by placing the protective disk on a hard surface, and pushing the exposed tip into the disk.

10 Slide the ejector forward to remove the used lancet. Dispose of the lancet according to your local regulations.

TaiDoc Technology Corp.


B1-7F, No.127, Wugong 2nd Rd., Wugu Dist.,
New Taipei City 24888, Taiwan
Tel: +886-2-6625-8188 Fax: +886-2-6625-0288

www.taidoc.com

LANCING DEVICE and TWIST LANCET

TD-5010

TaiDoc's Lancing Device is designed with more comfortable grip, different depth selections, and twist-release cap. It provides more convenient, more comfortable, and easier operation for the users.

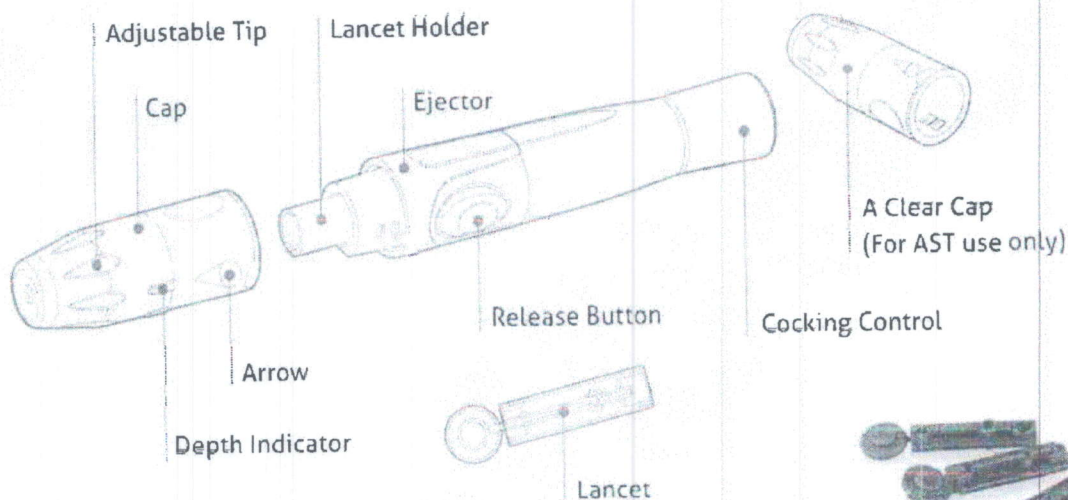FEATURES

- Twist-Release provides safer and easier operation for users.
- 6-level depth selection. More noticeable clicks between each level.
- The lancet can be ejected anytime.
- Large contact area – provides steady and comfortable puncture.
- Better adjustable tip grip
- Large holding area on the cap
- Status display on the release button is more visible
- Less vibration less pain

SPECIFICATIONS

LANCING DEVICE

Model Number: TD-5010
 Penetration Depth Setting: 6-level comfort dial
 Penetration Range: 0.8 mm to 1.8 mm
 Dimension: 118.7 (L) x 17.5 (D) mm
 Weight: 16.5 g
 Priming of Device:
 Push button to prime with status display window
 Lancet Ejection: Yes
 Alternative Site Testing: Yes

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