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INSTRUCTIONS FOR USE: break the inside water bubble, by squeezing the bag with both hands. Then shake it in order to mix the contents. Apply on the interested part.

INDICATIONS: haematomas, sprains, injuries, hyperthermia and whenever cold application is required.

CAUTION: for external use only; disposable product. Do not use if the bag is damaged. Do not break the bag and by all means prevent any content leakage. The content of the bag causes stains and irritation (in case of contact, rinse with water for 15 minutes), and is harmful (if swallowed seek immediate medical assistance). Store at room temperature out of the reach of children and away from heat sources and flammable materials. Do not use on injured skin. Contact your doctor in case of cardiovascular diseases, circulatory diseases, diabetes and/or cutaneous insensitivity (Raynaud's syndrome), if the product is used on infants, children and the elderly, or in case of doubt on how to perform the treatment and if the symptoms persist.

CONTENT: urea, ammonium chloride and water.

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