READ THESE INSTRUCTIONS

- 1 This is a ZC3B-53L Category 2 protective coverall.
- 2 It is limited life chemical protective clothing.
- 3 Is a limited use clothing offering limited protection against exposure to liquid aerosol sprays and light splashing.
- 4 The manufacturer is Portavita SRL see ref.2
- 5 Indicates the garment style.
- 6 The product number (ZC3B-53L).
- 7 Read this instruction sheet before use.
- 8 Size Code - see garment bag for size information.

GARMENT USE

Check you have the right size garment. Use only where the risk of chemical exposure is low and the potential is defined as low risk. Check the garment carefully before wearing. When used with additional equipment, such as gloves and/or face mask, enhanced protection can be achieved by taping the garment to the additional equipment.

LIMITATIONS OF USE

This garment may not be suitable for prolonged or intensive exposure to chemical spray and splash. Avoid exposure to very small particulate matter. Do not use with highly toxic or hazardous chemicals. Avoid naked flames, sparks or hot surfaces.

STEPS TO PUT ON THE GARMENT

1. Remove all personal items (jewelry, watches, cell phones, pens, etc.)
2. Move to the clean area at the entrance of the isolation unit.
3. By visual inspection, ensure that all sizes of the garment set are correct and the quality is appropriate.
4. Undertake the procedure of putting on the garment under the guidance and supervision of a trained observer

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- (colleague).
5. Put on gloves (examination, nitrile gloves).
 6. Step into the coveralls and pull up to the neck.
 7. Put on face mask.
 8. Put on face shield or goggles.
 9. Put on head and neck covering surgical bonnet covering neck and sides of the head (preferable with face shield) or hood.
 10. Put on second pair of (preferably long cuff) gloves over the cuff.

SUITABILITY

Product performance data - See Ref. 1 the employer must carry out a safety assessment and provide personal protective equipment that is fit for purpose. Portavita SRL accepts no responsibility for the use of the garment in an improper way for advice on the garment or its usage applications, you can contact www.portavita.md

PRODUCT DISPOSAL

Means of disposal will depend on any contamination and local regulations. The preferred method is incineration with energy recovery.

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