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MICROSCOPE SLIDES

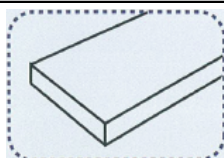
Slides produced in accordance with standard ISO 8037-1 and ISO 8037-2 (Optics and optical instruments — Microscopes — Slides), with clear glass, pre-washed and degreased, ready for use. Packed in handy packs of 72 slides, without interleaved tissue paper. Each box of 72 pieces is individually packed with plastic film to ensure maximum integrity and cleaning of the device. On every single case is reported the production batch and shelf life for full traceability of the device. The device is latex-free and/or natural rubber (latex-free).

Prodotto con marchio CE - conforme al Regolamento (UE) 2017/746 (IVDR)
CE Marked product - manufactured in compliance with Regulations (EU) 2017/746 (IVDR)

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

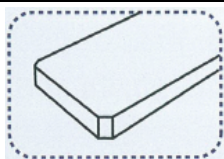
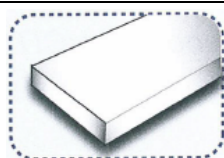
Ground edges type, it reduces the danger of cuts and infections to laboratory personnel.

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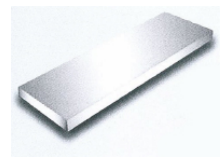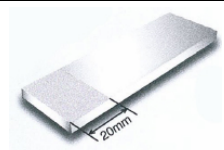Frosted at one end on one side with 20 mm depth, which allows an easy and permanent writing.

Codice articolo Item code	Bordi Edges	Angoli Corners	Area di scrittura Marking area	UDI-DI	R.D.M.*
13501	Tagliato Cut edges	 90°	No	8059575192230	1898293/R
13502	Molato Ground edges	 45°	No	8059575192247	1898298/R
13503	Tagliato Cut edges	 90°	Banda sabbata Frosted end	8059575192254	1898299/R
13504	Molato Ground edges	 45°	Banda sabbata Frosted end	8059575192261	1898300/R

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Basic UDI-DI (Global Model Number – GMN): 805577609F112SLIDES01FU

In Vitro Diagnostic Medical Device suitable to be used together with human biological samples intended to be examined under the microscope. For professional use only.

National classification of medical devices (CND - For Italian law) > W0503900201 (Microscope Slides)

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- For professional use only -

Use only for laboratory diagnostic analysis of human biological samples in healthcare settings.

Device without measuring function.

Any presence of a graduated scale is intended for indicative purposes and without a metrological function.

Do not vary the intended purpose of the device. Please contact the Manufacturer in the event of doubts concerning the intended purpose.

Before use check the perfect closure of the device if applicable.

Handle using personal protective equipment: risk of contamination.

After collection, store the sample as prescribed by the analytical method and deliver to the Laboratory within the maximum time specified by the same.

Do not use the Device in case of evident signs of breakage, injuries, cracks that could compromise its correct use.

Keep out of flame or heat sources which might damage the device.

Do not use after expiry date or if packing is opened.

Do not re-use: Disposable Device. Any reuse could cause sample contamination.

If sterile (ionizing radiations) or aseptic Device: Microbiological status in undamaged pack.

Do not re-sterilize.

Use the Device only with compatible and / or supplied accessories (caps, supports, etc.)

Do not put under direct sun rays; store in a dry, cool place (U.R. max 75% at 26°C)

Store in dry place, temperature range: min. -10°C (14°F) max +50°C (122°F)

Disposal according to applicable regulations, risk of infection.

Before use with particular substances check the resistance/compatibility chart in our catalogue.

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