

INSTRUCTION FOR USE

INTENDED USE

ONECARE İncontinence Pad&Wipe and Gloves are designed to be used in incontinence care.

These devices protect the skin of different patient populations including bedridden, patients admitted to intensive care unit (ICU) or patients with limited mobility by preventing the contact with urine and feces

WARNINGS AND PRECAUTIONS

Do not use products if the package is damaged.
Do not use it as a tampon.
Do not reuse the used product.
Do not use any other cleansing solution before using the product.
Avoid eye contact. Rinse with water in case eye contact occurs.
External use only.
Keep places children cannot reach.
Suitable for all skin types.
Do not wet the product.
Tightly seal the package after use.

WASH GLOVES

CONTRAINDICATIONS

None -Know

ADVERSE EFFECTS

None- Know

INSTRUCTION FOR USE

1. Open the package
2. Take the underwear off.
3. Clean the dirty area with pad/wipe/glove
4. Repeat the cleaning step until the intended are is cleaned.

MICROWAVE USING INSTRUCTIONS

1. Place the package in the microwave.
2. Heat it for 20 s in 700 W.
3. Check the temperature. Do not use the product in temperature exceeding 35-40 C

REF: 04/18

CE

Made in TURKEY

FACTORY
KAF GRUP SAĞ. HİZM. İNS. SAN. VE TIC. LTD. ŞTİ.

ADDRESS
ATAKENT MAH. 221. SK. NO: 3A ROTA OFFICE K. ÇEKMECE / İSTANBUL TURKEY