

DETROCID ACTIV- USER MANUAL

DETROCID ACTIV

- **PERACETIC ACID BASED MEDICAL INSTRUMENT AND ENDOSCOPE PRECLEANING, DISINFECTION, STERILIZATION PRODUCT**
- **CONCENTRATE – POWDER**

Usage Area

It is used for cleaning and disinfection of general and dental surgical instrument and equipment, healthcare equipment and anesthesia equipments, rigid and flexible endoscopes, laparoscopic instruments, incubators. Detroid Activ has multi property; enzymatic cleaning, disinfection, sterilization.

Usage form

- For sterilization, prepare %2 solution (add 20 gr product into 1 lt water and mix during 3-5 minutes).
- Immerse instruments into solution completely.
- After effect time (10 min), take out instruments and rinse with water.
- Prepared solution life is 24 hours. In the case of intensive contamination, it is recommended to renew solution earlier.

Usage Concentration

| Water Amount | % 2 – Solution |
|--------------|----------------|
| 1 litre | 20 gr |
| 5 litre | 100 gr |
| 10 litre | 200 gr |
| 15 litre | 300 gr |
| 20 litre | 400 gr |

Ingredient

Peracetic acid occuring from reaction of disodium carbonate and TAED with water

First aid Informations

- Eyes; In the case of eye contact, wash plenty of water around 5 min.
- Skin: Wash your skin immediately.
- Ingestion: Drink water and apply your doctor.

DETRO HEALTHCARE KİMYA SANAYİ A.Ş.

☎ 212 659 77 62 📞 212 659 77 63

✉ info@detrox.com.tr - www.detrox.com.tr

