

Autopsy Table YS-Z2

(Simple Autopsy Table)



It is specially designed according to the construction requirements of the anatomy room, and is suitable for public security forensic anatomy room, animal anatomy room, hospital pathology room and other units. It adopts 304 stainless steel high-quality plate for compression molding, digital laser cutting, precision production, acid and alkali resistance, the overall shape is luxurious, simple and durable.

1 Instruction Manual

1. Please open plywood case , **Please be careful**

Place of Installation

Place the autopsy table directly on the floor and check the horizontal position with a water level. If the floor is not completely even, place small metal strips under the table. Please pay attention that the autopsy table is aligned evenly lengthwise and in cross direction and that the base seats solidly.

2. Install the legs of the table




Fix the four legs to the table with screws

3. Connect the hot and cold water connections with the tap using the provided flexible hoses.
Connect the hot and cold water pipes, the interface size of the hot and cold water pipes is 1/2"



4. Please connect the waste pipe to the drain pipe

Connect the sewage pipe, the sewage pipe is suitable for pipe diameter 40~50 mm , (Sewage pipe diameter 38*38mm)

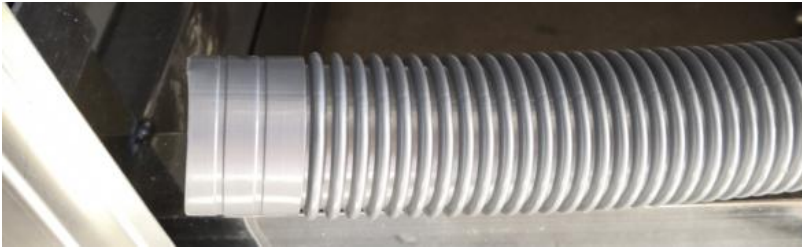

Gray tube is waste pipe



5. Install faucets and pool hair isolator.

Connect the power supply, Afterwards check the tap for successful operation. It is opened from the right to the left Side



Note :

Test again all waste water connections for leaks. Subsequently the table is ready for use.

When finished all installation work test all connections for leaks as well as for successful operation.

2.1 Product specifications

Material: high quality 304 stainless steel cold rolled steel Surface hardness: HV200 Bending: 180

Stretching: Tensile Test T_s/nmm^2 min 520 Thickness: 1.2mm of the countertop steel plate

Load: ≥ 180 kg

Specification: length 2400*width 700*height 800mm fixed type

The main function:

- ◆ The arc-shaped connection in the table top is not easy to scale and is easy to clean and disinfect
- ◆ There is a hair isolator in the pool to prevent hair from entering the drainage pipe and blocking
- ◆ Equipped with heat and cooling regulator, high manual faucet (need to connect water heater)
- ◆ The water pressure of the mobile spray and flushing gun can be adjusted to flush the whole dirt of the dissection table

- ◆ Equipped with infrared sensor faucet, aseptic contact, effectively prevent cross infection of bacteria
- ◆ The pulverizer is equipped with induction motor, cut-off flow, acid and alkali resistance
- ◆ The crusher under the pool can crush the dissected tissues and discharge them through the drain

2.2 General Information

The autopsy table is absolutely maintenance-free. Maintenance is limited to following points

- Regular cleaning of complete table
- Regular cleaning of drain pipes
- Regular testing of shower hose and hand shower for leaks
- Regular checking of tap for operation
- Regular testing of all connections for leaks