Quatrodes Forte

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(*clean conditions)

EN: Liquid concentrate for the simultaneous cleaning and disinfection of surfaces in medical field (units, treatment beds and chairs, operating tables, medical equipment) - as a medical device. Liquid concentrate for cleaning and disinfection of floor and above floor surfaces (walls, doors, worktops, cupboards, tables) in the medical field, in catering, in nursing homes, sanatoriums, gyms, fitness and wellness centres, hairdressing salons, beauty salons, solariums (for disinfection of sunbeds), massage and tattoo studios, kindergartens, schools and for disinfection of surfaces in contact with food - as a biocidal product.

Instructions for use: Prepare a working solution by diluting the concentrate with water in appropriate proportions. The concentration chosen will depend on the spectrum and duration required. To obtain a 1% concentration, dilute 10 ml of the concentrate in 990 ml of water. The surface does not need to be rinsed with water after disinfection. Do not combine with other detergents or disinfectants. The product is intended for professional use.

Ingredients: 100 g of the formulation contains: 3.76 g N-3-aminopropyl)-N-dodecylpropane-1.3-diamine (CAS: 2372-82-9), 3.39 g Poly(oxy-1.2-ethanediyl),.alpha.-[2-(dide-cylmethylammonio)ethyl]-.omega.hydroxy-,propanoate (salt) (CAS: 94667-33-1), 0.75 g 2-aminoethanol (CAS: 141-43-5).

Hazard statements: H302 - Harmful if swallowed. H410 - Very toxic to aquatic life with long lasting effects. H314 - Causes severe skin burns and eye damage.

Precautionary statements: P273: Avoid release to the environment. P280: Wear protective gloves/protective clothing/eye protection. P301+P330+P331: IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. P303+P361+P353: IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water or shower. P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P405: Store locked up. In the event of a medical incident, the manufacturer and URPL [Office for Registration of Medicinal Products, Medical Devices and Biocides] should be notified.

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