

STERILISATION REELS USER GUIDE

*Reels are 5, 7.5, 10, 15, 20, 25, 30, 35 and 40 cm(+/- 1 cm) width and 200m length (+/- 1mt)

*The reels are cut with 5-7cm part from top and bottom.

*After cutting process , the product is placed into the reel.

*Top and bottom part will be opened. It is closed with suitable heat and sealing machine.

*If necessary the indicator can be put into the reel. After that it will be ready for sterilization process.

*There are 2 or 3 indicator on the side of reels. Indicators are for steam, ethylene oxide and formaldehyde.

*After sterilization process color change as below;

(STEAM) Blue to Green (EO) Pink to Orange (FO) Purple to Green

*The sterilization process is made at 134°C for 3,5min or 7 min 121°C for 15min

ATTENTION

*Please pay attention to the deformation and tear on the packs.

*Do not overload the packs.

*Please pay attention to the seal strength of both ends.

*Do not use damaged products

*Keep products at room temperature 25°C