
**Station Name, Station AE Title**

The station name, station AE Title and station port are used as the DICOM information.

System	
Station Name	
Station AE Title	VXVUE
Station Port	9104

4.9.2 Queue Menu

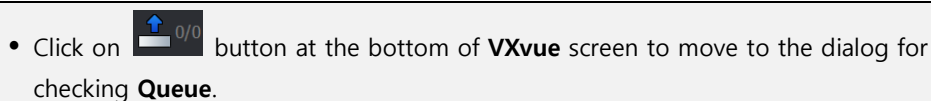

- When you start to send or print a DIOCM image, it is registered in **Queue** automatically.
- You can check the sending or printing status of DICOM image.
 - **Storage Queue** – Indicates the sending status of DICOM image to the DICOM storage server.
 - **Print Queue** - Indicates the printing status of DICOM image to the DICOM print server.

Storage Queue				
State	Server	AE Title	Patient ID	Patient Name

RetryDelete

Print Queue				
State	Server	AE Title	Patient ID	Patient Name

RetryDelete