Main catalogue
Catalogue général
Catálogo general
Catálogo geral

EN MOBILITY PRODUCTS
F PRODUITS D'AIDE À LA MOBILITÉ
ES AYUDAS PARA LA MOVILIDADE
P AUXÍLIO À MOBILIDADE

Our line of self-propelling, foldable, transit, children, travel, reclining and multifunctional wheelchairs.

- EN** Folding wheelchair in painted steel
- F** Fauteuil roulant pliant en acier vernis
- ES** Silla de ruedas plegable en acero pintado
- P** Cadeira de rodas dobrável em aço pintado

Start

Single cross frame

Series **CP100**

Optional accessories

Cod. **CPA150**
Armrest adjustable height 74÷82cm from floor - PAIR

Cod. **CPA120V**
Cod. **CPA122V** (only for 38 size)
Painted elevating footrests -PAIR

Cod. **CPA120C**
Cod. **CPA122C** (only for 38 size)
Chrome-plated elevating footrests -PAIR

Cod. **CPA501**
Linear table for wheelchair - 55x32 cm

Cod. **CPA506**
Table for wheelchair 58x38 cm

Spare parts

Cod. **CPR100** PAIR
Complete armrest

Cod. **CPR120** PAIR PU
Rear wheel Ø 60 cm

Cod. **CPR111** PAIR
Swing away lateral footrest

Cod. **CPR140** SINGLE
PVC front wheel Ø 20x2,5 cm

Cod. **CPR135** PAIR
PAIR Swing away lateral footrest (size 38 only)

Cod. **CPR241** SINGLE
Plastic footrest

Cod. **CPR105**
Only armrest padding

Technical features

SPECIFICATIONS																
Front wheels	PVC, black, steel fork															
Rear wheels	Aluminium with radius, PU, black															
Upholstery	Padded nylon - Fire reaction tested, according to EN1021-1/2															
WEIGHT AND DIMENSION																
Measures in cm																
CP100-38								38		56						
CP100-40								40		58						
CP100-43								43		61						
CP100-45	113	83	43	90	74	49	42÷56	45	43	63	27	17,5	60	20	-	120
CP100-46								46		63,5						
CP100-48								48		65						

- EN** Folding wheelchair in painted steel
- F** Fauteuil roulant pliant en acier vernis
- ES** Silla de ruedas plegable en acero pintado
- P** Cadeira de rodas dobrável em aço pintado

Start

Single cross frame

Series **CP100R**

Optional accessories

Cod. **CPA121**
Painted elevating footrests -PAIR

Cod. **CPA501**
Linear table for wheelchair - 55x32 cm

Cod. **CPA506**
Table for wheelchair 58x38 cm

Spare parts

Cod. **CPR400** PAIR
Complete armrest

Cod. **CPR413** PAIR
Rear wheel Ø 60 cm

Cod. **CPR410** SINGLE
PVC front wheel Ø 20x2,5 cm

Cod. **CPR415** PAIR
Swing away lateral footrest

Cod. **CPR402**
Only armrest padding

Technical features

SPECIFICATIONS																
Front wheels	PVC, black, steel fork															
Rear wheels	Aluminium with radius, PU, black															
Upholstery	Padded nylon - Fire reaction tested, according to EN1021-1/2															
WEIGHT AND DIMENSION																
Measures in cm																
CP100R-40																
CP100R-43																
CP100R-45	103	85	40	91	75	50	20	45	43	63	26,5	17,5	60	20	-	120
CP100R-48								48		65						

- EN** Folding wheelchair in painted steel
- F** Fauteuil roulant pliant en acier vernis
- ES** Silla de ruedas plegable en acero pintado
- P** Cadeira de rodas dobrável em aço pintado

Start1

Single cross frame - Quick release

Series **CP102**

Optional accessories

Cod. **CPA150**
Armrest adjustable height 74÷82cm from floor - PAIR

Cod. **CPA120V**
Cod. **CPA122V** (only for 38 size)
Painted elevating footrests -PAIR

Cod. **CPA120C**
Cod. **CPA122C** (only for 38 size)
Chrome-plated elevating footrests -PAIR

Cod. **CPA501**
Linear table for wheelchair - 55x32 cm

Cod. **CPA506**
Table for wheelchair 58x38 cm

Spare parts

Cod. **CPR100** PAIR
Complete armrest

Cod. **CPA242** PAIR PU
Rear wheel Ø 60 cm

Cod. **CPR111** PAIR
Swing away lateral footrest

Cod. **CPR140** SINGLE
PVC front wheel Ø 20x2,5 cm

Cod. **CPR135** PAIR
PAIR Swing away lateral footrest (size 38 only)

Cod. **CPR241** SINGLE
Plastic footrest

Cod. **CPR105**
Only armrest padding

Technical features

SPECIFICATIONS																
Front wheels	PVC, black, steel fork															
Rear wheels	Aluminium with radius, PU, black															
Upholstery	Padded nylon- Fire reaction tested, according to EN1021-1/2															
WEIGHT AND DIMENSION																
Measures in cm															Max loading weight	
CP102-38								38		56						
CP102-40								40		58						
CP102-43								43		61						
CP102-45	113	83	43	90	74	49	42÷56	45	43	63	27	17,5	60	20	-	120
CP102-46								46		63,5						
CP102-48								48		65						

- EN** Folding wheelchair in painted steel
- F** Fauteuil roulant pliant en acier vernis
- ES** Silla de ruedas plegable en acero pintado
- P** Cadeira de rodas dobrável em aço pintado

Start²

Double cross frame - Quick release

Series **CP103R**

Optional accessories

Cod. **CPA121**
Painted elevating footrests - PAIR

Cod. **CPA501**
Linear table for wheelchair - 55x32 cm

Cod. **CPA506**
Table for wheelchair 58x38 cm

Spare parts

Cod. **CPR401** PAIR
Complete armrest

Cod. **CPR413** PAIR
Rear wheel Ø 60 cm

Cod. **CPR411** SINGLE
PVC front wheel Ø 20x5 cm

Cod. **CPR415** PAIR
Swing away lateral footrest

Cod. **CPR403**
Only armrest padding

Technical features

SPECIFICATIONS																
Front wheels	PVC, black, steel fork															
Rear wheels	Aluminium with radius, PU, black															
Upholstery	Padded nylon- Fire reaction tested, according to EN1021-1/2															
WEIGHT AND DIMENSION																
Measures in cm																
CP103R-40																
CP103R-43																
CP103R-45	103	85	40	91	75	50	20	45	43	63	26,5	17,5	60	20	-	130
CP103R-48								48		65						
CP103R-50																

- EN** Folding wheelchair in painted steel
- F** Fauteuil roulant pliant en acier vernis
- ES** Silla de ruedas plegable en acero pintado
- P** Cadeira de rodas dobrável em aço pintado

Next

Double cross frame - Convertible to Transit

Series **CP110**

- Back pocket
- Flip up desk type armrests, nylon padded
- Swing away adjustable footrests
- Heel stopper
- Black plastic footrest
- Spare parts
- Cod. **CPR107** PAIR Complete armrest
- Cod. **CPR111** PAIR Swing away lateral footrest
- Cod. **CPR241** SINGLE Plastic footrest
- Cod. **CPA242** PAIR PU rear wheel Ø 60 cm
- Cod. **CPR140** SINGLE PVC front wheel Ø 20x2,5 cm
- Cod. **CPR105** Only armrest padding
- Painted steel self-propelling ring
- Quick release wheels
- Hole for Ø 30 cm wheel
- Safety wheel

Optional accessories

Cod. **CPA150**
Armrest adjustable height 74÷82cm from floor - PAIR

Cod. **CPA120V**
Painted elevating footrests - PAIR

Cod. **CPA120C**
Chrome-plated elevating footrests - PAIR

Cod. **CPA501**
Linear table for wheelchair 55x32 cm (except size 50)

Cod. **CPA506**
Table for wheelchair 58x38 cm (except size 50)

Cod. **CPA517**
Table for wheelchair large 68x57 cm (only for size 50)

Technical features

SPECIFICATIONS															
Front wheels	PVC, black, steel fork														
Rear wheels	Aluminium with radius, PU, black														
Upholstery	Padded nylon - Fire reaction tested, according to EN1021-1/2														
WEIGHT AND DIMENSION															
Measures in cm															Max loading weight
CP110-40								40		64					
CP110-43								43		67					
CP110-45								45		69					
CP110-46	105	81	43	90	73	49	41+56	46	42	69,5	26	18,5	60	20	- 150
CP110-48								48		71					
CP110-50								50		74					

- EN** Folding transit wheelchair - Painted steel
- F** Fauteuil roulant pliant de transit - Acier verni
- ES** Silla de ruedas plegable de tránsito - Acero pintado
- P** Cadeira de rodas dobrável de trânsito - Aço pintado

Next go!

Double cross frame - Convertible to self-propelling

Series **CP115**

Back pocket

Flip up desk type armrests, nylon padded

Hole for Ø 60 cm wheel

Swing away adjustable footrests

Quick release wheel

Heel stopper

Safety wheel

Black plastic footrest

Spare parts

Cod. **CPR107** PAIR Complete armrest

Cod. **CPR111** PAIR Swing away lateral footrest

Cod. **CPR241** SINGLE Plastic footrest

Cod. **CPA240** SINGLE PVC front wheel Ø 20x2,5 cm

Cod. **CPR140** SINGLE PVC front wheel Ø 20x2,5 cm

Cod. **CPR105** Only armrest padding

Optional accessories

Cod. **CPA150**
Armrest adjustable height 77÷85cm from floor - PAIR

Cod. **CPA120V**
Painted elevating footrests - PAIR

Cod. **CPA120C**
Chrome-plated elevating footrests - PAIR

Cod. **CPA501**
Linear table for wheelchair 55x32 cm (except size 50)

Cod. **CPA506**
Table for wheelchair 58x38 cm (except size 50)

Cod. **CPA517**
Table for wheelchair large 68x57 cm (only for size 50)

Technical features

SPECIFICATIONS																
Front wheels	PVC, black, steel fork															
Rear wheels	With plastic radius, PU, black															
Upholstery	Padded nylon - Fire reaction tested, according to EN1021-1/2															
WEIGHT AND DIMENSION																
Measures in cm																
CP115-40								40		53						
CP115-43								43		56						
CP115-45								45		58						
CP115-46	98	75	41	92	76	50	41÷56	46	43	58,5	25	18,5	30	20	-	150
CP115-48								48		61						
CP115-50								50		63						

- EN** Folding wheelchair in painted steel
- F** Fauteuil roulant pliants en acier vernis
- ES** Silla de ruedas plegable en acero pintado
- P** Cadeira de rodas dobrável em aço pintado

Double cross frame - Convertible to Transit

Series **CP200**

Optional accessories

Cod. **CPA120V**
Cod. **CPA122V** (only for 38 size)
Painted elevating footrests -PAIR

Cod. **CPA120C**
Cod. **CPA122C** (only for 38 size)
Chrome-plated elevating footrests -PAIR

Cod. **CPA501**
Linear table for wheelchair
55x32 cm

Cod. **CPA506**
Table for wheelchair
58x38 cm (except size 50)

Cod. **CPA517**
Table for wheelchair large
68x57 cm (except size 50)

Spare parts

Cod. **CPR108** PAIR
Height adjustable armrest

Cod. **CPR143** SINGLE PU
front wheel Ø 20x3 cm

Cod. **CPR111** PAIR Swing away lateral footrest

Cod. **CPR241** SINGLE
Plastic footrest

Cod. **CPR135** PAIR
Lateral detachable footrest (size 38 only)

Cod. **CPR150** PAIR Anti-tipper wheel Ø 4 cm

Cod. **CPA242** PAIR PU
rear wheel Ø 60 cm

Cod. **CPR106**
Only PU armrest padding

Technical features

SPECIFICATIONS

Front wheels PU, black, aluminium fork

Rear wheels Aluminium, with radius, PU, black

Upholstery Padded nylon- Fire reaction tested, according to EN1021-1/2

WEIGHT AND DIMENSION

Measures in cm																	Max loading weight
CP200-38																	
CP200-40																	
CP200-43																	
CP200-45	113	81	42	90	72÷80	49	41÷56	45	41	65	29	17,5	60	20	4	150	
CP200-46								46		65,5							
CP200-48								48		68							
CP200-50								50		70							

- EN** Folding transit wheelchair - Painted steel
- F** Fauteuil roulant pliant de transit - Acier verni
- ES** Silla de ruedas plegable de tránsito - Acero pintado
- P** Cadeira de rodas dobrável de trânsito - Aço pintado

Double cross frame - Convertible to self-propelling

Series **CP205**

Optional accessories

Cod. **CPA120V**
Cod. **CPA122V** (only for 38 size)
Painted elevating footrests -PAIR

Cod. **CPA120C**
Cod. **CPA122C** (only for 38 size)
Chrome-plated elevating footrests - PAIR

Cod. **CPA501**
Linear table for wheelchair
55x32 cm

Cod. **CPA506**
Table for wheelchair
58x38 cm
(except size 50)

Cod. **CPA517**
Table for wheelchair large
68x57 cm
(except size 50)

Spare parts

Cod. **CPR108** PAIR
Height adjustable armrest

Cod. **CPR111** PAIR
Swing away lateral footrest

Cod. **CPR135** PAIR
Lateral detachable footrest (size 38 only)

Cod. **CPA240** PAIR
PU rear wheel Ø 30 cm

Cod. **CPR143** SINGLE
PU front wheel Ø 20x3 cm

Cod. **CPR241** SINGLE
Plastic footrest

Cod. **CPR150** PAIR
Anti-tipper wheel Ø 4 cm

Cod. **CPR106**
Only PU armrest padding

Technical features

SPECIFICATIONS																												
Front wheels	PU, black, aluminium fork																											
Rear wheels	Plastic radius, PU, black																											
Upholstery	Padded nylon - Fire reaction tested, according to EN1021-1/2																											
WEIGHT AND DIMENSION																												
Measures in cm																												
CP205-38																												
CP205-40																												
CP205-43																												
CP205-45	104	76	42	90	71÷80	49	41÷55	45	41	59	25	17,5	30	20	4	150												
CP205-46								46		59,5																		
CP205-48								48		62																		
CP205-50								50		64																		

- EN** Folding wheelchair - Painted steel
- F** Fauteuil roulant pliant - Acier verni
- ES** Silla de ruedas plegable - Acero pintado
- P** Cadeira de rodas dobrável - Aço pintado

Plus

Double cross - Bariatric

Series **CP300**

Optional accessories

Cod. **CPA154**
Armrest adjustable height 69÷79cm from floor - PAIR

Cod. **CPA120V**
Painted elevating footrests - PAIR

Cod. **CPA120C**
Chrome-plated elevating footrests - PAIR

Cod. **CPA517**
Table for wheelchair large 68x57 cm

Spare parts

Cod. **CPR103** PAIR
Complete armrest

Cod. **CPR111** PAIR Swing away lateral footrest

Cod. **CPR241** SINGLE
Plastic footrest

Cod. **CPR122** PAIR Solid rear wheel Ø 60 cm

Cod. **CPR141** SINGLE PU
front wheel Ø 20x5 cm

Technical features

SPECIFICATIONS																
Front wheels	PU, black, steel fork															
Rear wheels	Plastic radius, PU, solid, black															
Upholstery	Padded nylon- Fire reaction tested, according to EN1021-1/2															
WEIGHT AND DIMENSION																
Measures in cm																
CP300-50																
CP300-55	118	90	48	92	76	52	43+57	55	41	78	32	26	60	20	-	200
CP300-60								60		83						

- EN** Folding transit wheelchair - Painted steel
- F** Fauteuil roulant pliant de transit - Acier verni
- ES** Silla de ruedas plegable de tránsito - Acero pintado
- P** Cadeira de rodas dobrável de trânsito - Aço pintado

*Plus!
go!*

Double cross - Bariatric

Series **CP310**

Optional accessories

Cod. CPA154
Armrest adjustable height 69÷79cm from floor - PAIR

Cod. CPA120V
Painted elevating footrests -PAIR

Cod. CPA120C
Chrome-plated elevating footrests - PAIR

Cod. CPA517
Table for wheelchair large 68x57 cm

Spare parts

Cod. CPR103 Complete armrest - PAIR

Cod. CPR111 Swing away lateral footrests - PAIR

Cod. CPR241 SINGLE Plastic footrest

Cod. CPR132 PAIO PU rear wheel Ø 32 cm - PAIR

Cod. CPR141 SINGOLA PU front wheel Ø 20x5 cm - SINGLE

Technical features

SPECIFICATIONS																
Front wheels	PU, black, steel fork															
Rear wheels	Plastic radius, PU, solid, black															
Upholstery	Padded nylon- Fire reaction tested, according to EN1021-1/2															
WEIGHT AND DIMENSION																
Measures in cm																
CP310-50	104	77	48	92	70	52	43÷57	50	39	63	30	26	32	20	-	200
CP310-55								55		68						

- EN** Folding transit wheelchair - Painted steel
- F** Fauteuil roulant pliant de transit - Acier verni
- ES** Silla de ruedas plegable de tránsito - Acero pintado
- P** Cadeira de rodas dobrável de trânsito - Aço pintado

Single cross

Series CP500

Optional accessories

Cod. **CPA150**
Armrest adjustable height 73÷81 cm from floor - PAIR

Cod. **CPA120V**
Painted elevating footrests - PAIR

Cod. **CPA120C**
Chrome-plated elevating footrests - PAIR

Cod. **CPA501**
Linear table for wheelchair 55x32 cm

Cod. **CPA506**
Table for wheelchair 58x38 cm

Spare parts

Cod. **CPR101** PAIR
Complete armrest

Cod. **CPR131** PAIR PU
Rear wheel Ø 30 cm

Cod. **CPR111** PAIR
Swing away lateral footrest

Cod. **CPR241** SINGLE
Plastic footrest

Cod. **CPR143** SINGLE
PU front wheel Ø 20x3 cm

Technical features

SPECIFICATIONS															
Front wheels	PU, black, steel fork														
Rear wheels	Plastic radius, PU, black														
Upholstery	Padded nylon- Fire reaction tested, according to EN1021-1/2														
WEIGHT AND DIMENSION															
Measures in cm															
CP500-40								40		55					
CP500-43								43		58					
CP500-45	94	70	42	92	72	48	42÷54	45	43	60	26	17,5	30	20	- 120
CP500-46								46		60,5					
CP500-48								48		63					

- EN** Folding transit wheelchair - Painted steel
- F** Fauteuil roulant pliant de transit - Acier verni
- ES** Silla de ruedas plegable de tránsito - Acero pintado
- P** Cadeira de rodas dobrável de trânsito - Aço pintado

Go!

Single cross

Series **CP500R**

Optional accessories

Cod. **CPA121**
Painted elevating footrests - PAIR

Cod. **CPA501**
Linear table for wheelchair
55x32 cm

Cod. **CPA506**
Table for wheelchair
58x38 cm

Spare parts

Cod. **CPR401** PAIR
Complete armrest

Cod. **CPR412** PAIR
Rear wheel Ø 30 cm

Cod. **CPR411** SINGLE
PU front wheel Ø 20x5 cm

Cod. **CPR415** PAIR
Swing away lateral footrest

Technical features

SPECIFICATIONS																
Front wheels	PU, black, steel fork															
Rear wheels	Plastic radius, PU, black															
Upholstery	Padded nylon- Fire reaction tested, according to EN1021-1/2															
WEIGHT AND DIMENSION																
Measures in cm																
CP500R-40								40		58						
CP500R-43	103	75	40	91	73	50	20	43	43	60,5	26,5	16,5	30	20	-	120
CP500R-45								45		63						

- EN** Folding transit wheelchair - Painted steel
- F** Fauteuil roulant pliant de transit - Acier verni
- ES** Silla de ruedas plegable de tránsito - Acero pintado
- P** Cadeira de rodas dobrável de trânsito - Aço pintado

Double cross

Series **CP520**

Optional accessories

Cod. **CPR108**
Armrest adjustable height 73÷81 cm from floor - PAIR

Cod. **CPA120V**
Painted elevating footrests -PAIR

Cod. **CPA120C**
Chrome-plated elevating footrests - PAIR

Cod. **CPA501**
Linear table for wheelchair 55x32 cm

Cod. **CPA506**
Table for wheelchair 58x38 cm

Spare parts

Cod. **CPR102** PAIR
Complete armrest

Cod. **CPR131** PAIR PU
rear wheel Ø 30 cm

Cod. **CPR111** PAIR
Swing away lateral footrest

Cod. **CPR241** SINGLE
Plastic footrest

Cod. **CPR143** SINGLE
PU front wheel Ø 20x3 cm

Cod. **CPR215S** Left
Cod. **CPR215D** Right
Nurse brake

Technical features

SPECIFICATIONS																
Front wheels	PU, black, steel fork															
Rear wheels	Plastic radius, PU, black															
Upholstery	Padded nylon- Fire reaction tested, according to EN1021-1/2															
WEIGHT AND DIMENSION																
Measures in cm																
CP520-40								40		55						
CP520-43								43		58						
CP520-45	99	73	42	93	72	48	42÷56	45	42	60	26	17,5	30	20	-	150
CP520-46								46		60,5						
CP520-48								48		63						

- EN** Folding wheelchair in painted steel
- F** Fauteuil roulant pliants en acier vernis
- ES** Silla de ruedas plegable en acero pintado
- P** Cadeira de rodas dobrável em aço pintado

Skinny

Single cross frame - Convertible to Transit

Series **CP620**

Compact

Optional accessories

			
Cod. CPA120V Painted elevating footrests - PAIR	Cod. CPA120C Chrome-plated elevating footrests - PAIR	Cod. CPA501 Linear table for wheelchair 55x32 cm	Cod. CPA506 Table for wheelchair 58x38 cm

Spare parts

- Cod. CPR610 PAIR**
Complete armrest
- Cod. CPR621 PAIR PU**
rear wheel Ø 50 cm
- Cod. CPR625 PAIR**
Detachable footrest
- Cod. CPR241 SINGLE**
Plastic footrest
- Cod. CPR620 SINGLE**
PU Front wheel Ø 18 cm

Technical features

SPECIFICATIONS																	
Front wheels	PU, black, plastic fork																
Rear wheels	Aluminium radius, PU, black																
Upholstery	Padded nylon- Fire reaction tested, according to EN1021-1/2																
WEIGHT AND DIMENSION																	
Measures in cm																	 Max loading weight
CP620-40	100	73	41	92	80	53	45÷57	40	37	51	26	16,5	50	18	-	110	
CP620-43								43		54							

- EN** Folding transit wheelchair - Painted steel
- F** Fauteuil roulant pliant de transit - Acier verni
- ES** Silla de ruedas plegable de tránsito - Acero pintado
- P** Cadeira de rodas dobrável de trânsito - Aço pintado

Single cross frame - Convertible to a self-propelling

Series **CP625**

Compact

Optional accessories

- Cod. CPA120V**
Painted elevating footrests - PAIR
- Cod. CPA120C**
Chrome-plated elevating footrests - PAIR
- Cod. CPA501**
Linear table for wheelchair
55x32 cm
- Cod. CPA506**
Table for wheelchair
58x38 cm

Spare parts

- Cod. CPR610 PAIR**
Complete armrest
- Cod. CPR622 PAIR**
PU rear wheel Ø 30cm
- Cod. CPR625 PAIR**
Detachable footrest
- Cod. CPR241 SINGLE**
Plastic footrest
- Cod. CPR620 SINGLE**
PU Front wheel Ø 18 cm

Technical features

SPECIFICATIONS																
Front wheels	PU, black, plastic fork															
Rear wheels	Plastic radius, PU, black															
Upholstery	Padded nylon- Fire reaction tested, according to EN1021-1/2															
WEIGHT AND DIMENSION																
Measures in cm																
CP625-40	88	61	41	92	80	53	45÷57	40	37	51	26	16,5	30	18	-	110
CP625-43								43		54						

Accessories for standard wheelchairs

Cod. **CPA110-45**
WC seat 45 cm
(size 45 only)

Cod. **CPA100-40**,
CPA100-43, **CPA100-45**,
CPA100-50
Foam cushion 5 cm h

Cod. **CAR900**
Headrest

Cod. **CPA501**
Linear table
55x32 cm (except sizes
50-55-60)

Cod. **CPA506**
Table for wheelchair
58x38 cm (except sizes
50-55-60)

Cod. **CPA517**
Table for wheelchair
68x57 cm
(sizes 50-55-60 only)

Cod. **CPA120V**
Cod. **CPA122V** (only for
38 size)
Painted elevating
footrests -PAIR

Cod. **CPA120C**
Cod. **CPA122C** (only for
38 size)
Chrome-plated
elevating footrests
-PAIR

Cod. **CPA121**
Pedana elevabile
verniciata - PAIO
DM332/99:
12.24.03.109
DPCM 12.01.2017
12.22.91.718

Cod. **CPA130**
Footrest
(sizes 45, 46, 48, 50, 55
only - No red models)

Cod. **CPA605**
Oxygen bottle holder in
chromed steel Ø bottle
max 12 cm

Cod. **CPA606** Cod. **CPA600**
Polyester Adjustable IV pole
Oxygen
bottle holder
Ø bottle max
12 cm

Cod. **CPA234**
One hand drive kit

Cod. **CPA240**
PU rear wheel - PAIR
(only for CP110,
CP200)

Cod. **CPR622**
PU solid rear wheel Ø
30 cm - PAIR (only for
CP620)

Cod. **CPA242**
PU rear wheel Ø 60 cm
- PAIR (sizes CP115,
CP205 only)

Cod. **CPA244**
Quick release rear wheel,
PNEUMATIC Ø 60 cm
- PAIR
(only for CP102,
CP110, CP200)

Cod. **CPR150**
Anti-tipper wheels
4 cm - PAIR
(only for CP100, CP102,
CP110, CP115, CP205,
CP500, CP520)

Cod. **CAR605**
Anti-heft bar
(except CP620,
CP625)

Cod. **CPA150**
Armrest adjustable
height 74÷82 cm from
floor - PAIR (only for
CP100, CP102, CP110,
CP115, CP500)

Cod. **CPA154**
Armrest adjustable
height 69÷79cm from
floor - PAIR (only for
CP300, CP310)

Cod. **CPR108**
Armrest adjustable
height 73÷81 cm from
floor - PAIR (only for
CP520)

Cod. **CPA160**
Armrest PAIR
(only for CP100,
CP102, CP110, CP115,
CP500)

Cod. **CPA140-xx**
Universal backrest
extension - specify the size
38/40/43/45/48/50/55/60
(No red models)

KOALA (MOPEDIA)
Restraining system line

Cod. **RP245**
Safety belt

Cod. **RP247**
Safety belt with spring
buckle

Kit to convert a transit to a self-propelling:

From Next (CP110) to Next Go! (CP115)
CPR216D, right brake;
CPR216S, left brake;
CPA240, small wheel - pair;
nr. 2 **CPR342**, quick release pivot for back wheel;

From Next Go! (CP115) to Next (CP110)
CPR210D, right brake dx;
CPR210S, left brake;
CPA242, big wheel - pair;
nr. 2 **CPR343**, quick release pivot for back wheel;

From Dual (CP200) to Dual Go! (CP205)
CPA240, small wheel - pair;
nr. 2 **CPR342**, quick release pivot for back wheel;

From Dual Go! (CP205) to Dual (CP200):
CPA242, big wheel - pair;
CPR343, quick release pivot for back wheel;

From Skinny (CP620) to Skinny Go! (CP625)
CPR622, small wheel - pair;
CPR211D, right brake;
CPR211S, left brake;

From Skinny Go! (CP625) to Skinny (CP620)
CPR621, big wheel - pair;
CPR630D, right brake;
CPR630S, left brake;

Spare parts for standard wheelchairs

Cod. CPR100
Complete armrest PAIR
(only for CP100,
CP102)

Cod. CPR107
Complete armrest PAIR
(only for CP110,
CP115)

Cod. CPR108
Height adjustable
armrest PAIR (only for
CP200, CP205)

Cod. CPR103
Complete armrest PAIR
(only for CP300-
CP310)

Cod. CPR101
Complete armrest PAIR
(only for CP500)

Cod. CPR102
Complete armrest PAIR
(only for CP520)

Cod. CPR610
Complete armrest PAIR
(only for CP620-
CP625)

Cod. CPR400
Complete armrest PAIR
(only for CP100R)

Cod. CPR401
Complete armrest PAIR
(only for CP103R-
CP500R)

Cod. CPR215S Left
Cod. CPR215D Right
Brake (only for CP520)

Cod. CPR111
Swing away lateral
footrests – PAIR

Cod. CPR135
Swing away lateral
footrest PAIR
(size 38 only)

Cod. CPR625
Swing away lateral
footrests – PAIR
(only for CP620-CP625)

Cod. CPR415
Swing away lateral
footrests – PAIR (series
CP100R-CP103R-CP500R)

Cod. CPR241
Plastic footrest SINGLE (only
blue models)

Cod. CPR120
PU rear wheel Ø 60 cm
– PAIR (only for
CP100)

Cod. CPA242
U rear wheel Ø 60 cm
– PAIR
(only for CP102-
CP110-CP200-CP205)

Cod. CPR122
Solid rear wheel Ø 60
cm – PAIR
(only for CP300)

Cod. CPR621
PU rear wheels
Ø 50cm – PAIR
(only for CP620)

Cod. CPR413
PU rear wheels
Ø 60cm – PAIR
(only for CP100R-
CP103R)

Cod. CPR150
Anti-tipper wheels 4 cm -
PAIR (only for CP200-
CP205)

Cod. CPR132
PU rear wheel Ø 32 cm
- PAIR
(only for CP310)

Cod. CPA240
PU rear wheels – PAIR
(only for CP115-
CP205)

Cod. CPR131
PU rear wheel Ø 30 cm
- PAIR (only for
CP500-CP520)

Cod. CPR622
PU solid rear wheel
Ø 30cm – PAIR (only
for CP625)

Cod. CPR412
PU solid rear wheel
Ø 30cm – PAIR
(only for CP500R)

Cod. CPR140
PVC front wheel
Ø 20 x 2,5 cm - SINGLE
(only for CP100-
CP102-CP110-CP115)

Cod. CPR141
PU front wheel Ø 20x5
cm - SINGLE
(only for CP300-
CP305)

Cod. CPR143
PU front wheel Ø 20x3
cm - SINGLE
(only for CP200-
CP205-CP500-
CP520)

Cod. CPR620
PU front wheel
Ø 18cm – SINGLE
(only for CP620-
CP625)

Cod. CPR410
PVC front wheel
Ø 20 x 2,5 cm - SINGLE
(only for CP100R)

Cod. CPR411
PU front wheel
Ø 20 x 5 cm - SINGLE
(only for CP103R-
CP500R)

- EN** Folding wheelchair in painted steel
- F** Fauteuil roulant pliants en acier vernis
- ES** Silla de ruedas plegable en acero pintado
- P** Cadeira de rodas dobrável em aço pintado

Kiddy

Single cross frame - Children

Cod. CP880-35

Height adjustable handle
86÷99 cm

Supplied with

Elevating footrests

Optional accessories

Cod. CPA600
Adjustable IV Pole

Cod. CPA605
Chromed steel oxygen bottle holder Ø Max oxygen bottle 12 cm

Cod. CPA606
Polyester Oxygen bottle holder - Ø oxygen bottle Max 12 cm

Cod. CPA507
Table for wheelchair 58x38 cm

Cod. CPA501
Linear table 55x32 cm

Cod. RP247
Safety belt with spring buckle

Spare parts

Cod. CPR810
Flip up armrest -PAIR

Cod. CPR821
PU rear wheel Ø 56 cm - SINGLE

Cod. CPR820
PVC front wheel Ø 15 cm - SINGLE

Cod. CPR827
Armrest padding only

Cod. CPR815
Lateral footrests

Cod. CPR817
Elevating footrests

Technical features

SPECIFICATIONS

Front wheels	PVC, black, painted steel fork
Rear wheels	Plastic radius, grey PU wheels
Upholstery	Padded nylon- Fire reaction tested, according to EN1021-1/2

WEIGHT AND DIMENSION

Measures in cm																			
CP880-35	94	74	36	86÷99	62	42	37÷42	35	37	56	31	14,5	56	15	-	-	-	100	

- EN** Folding travel wheelchair in anodized aluminium
- F** Fauteuil roulant pliant de voyage en aluminium anodisé
- ES** Silla de transporte de viaje plegable en aluminio anodizado
- P** Cadeira de rodas de viagem dobrável em alumínio anodizado

Travel

Double cross frame

Cod. **CP850-37**

7 Kg

Wheelchair weight

Optional accessories

Cod. **CBR850**
Bag for transport

Spare parts

Cod. **CBR294**
Back wheels Ø 20 cm
- SINGLE

Cod. **CBR301**
Front wheel Ø 13 cm
- SINGLE

Cod. **RP247**
Safety belt with
spring buckle

Technical features

SPECIFICATIONS																	
Front wheels	Grey, solid with plastic fork																
Rear wheels	Plastic radius, grey solid wheels																
Upholstery	Padded nylon- Fire reaction tested, according to EN1021-1/2																
WEIGHT AND DIMENSION																	
Measures in cm																	
CP850-37	86	58	38	89	65	43	33	37	41	44	29	7	20	13	-	75	

- EN** Folding wheelchair in painted steel
- F** Fauteuil roulant pliant en acier vernis
- ES** Silla de ruedas plegable en acero pintado
- P** Cadeira de rodas dobrável em aço pintado

Single cross frame - Reclining backrest

Series **CP800**

Optional accessories

Spare parts

Cod. **CPA501**
Linear table
for wheelchair

Cod. **CPA506**
Table for wheelchair
58x38 cm

Cod. **CAR831**
Armrest desk type
- PAIR

Cod. **CAR171**
Elevating footrest
- PAIR

Cod. **CAR174**
PU rear wheel Ø 60
cm - PAIR

Cod. **CAR172**
PU front wheel Ø 20
cm - PAIR

Cod. **CAR173**
Anti-tipper wheel Ø
5 cm - PAIR

Technical features

SPECIFICATIONS																	
Front wheels	Grey, PU, plastic fork																
Rear wheels	Aluminium radius, grey PU wheels																
Upholstery	Padded nylon																
WEIGHT AND DIMENSION																	
Measures in cm																	
CP800-40																	
CP800-43	123	90	44	100÷126	75	50	50÷64	43	83	65	30	26	60	20	5	100	
CP800-46								46		68							

- EN** Folding transit wheelchair - Painted steel
- F** Fauteuil roulant pliant de transit - Acier verni
- ES** Silla de ruedas plegable de tránsito - Acero pintado
- P** Cadeira de rodas dobrável de trânsito - Aço pintado

Single cross frame - Reclining backrest

Series **CP805**

Optional accessories

Spare parts

Cod. **CPA501**
Linear table
for wheelchair

Cod. **CPA506**
Table for wheelchair
58x38 cm

Cod. **CAR831**
Armrest desk type
- PAIR

Cod. **CAR171**
Elevating footrest
- PAIR

Cod. **CAR175**
PU rear wheel
(only CP805)

Cod. **CAR172**
PU front wheel Ø 20
cm - PAIR

Cod. **CAR173**
Anti-tipper wheel Ø
5 cm - PAIR

Technical features

SPECIFICATIONS																	
Front wheels	Grey, PU, plastic fork																
Rear wheels	Plastic radius, grey PU wheels																
Upholstery	Padded nylon																
WEIGHT AND DIMENSION																	
Measures in cm																	Max loading weight
CP805-40																	
CP805-43	123	90	44	100÷126	75	50	50÷64	43	83	65	30	25	30	20	5	100	
CP805-46								46		68							

- EN** Folding wheelchair in painted aluminium
- F** Fauteuil roulant pliant en aluminium verni
- ES** Silla de transporte plegable en aluminio pintado
- P** Cadeira de rodas dobrável em alumínio pintado

Double cross frame - Reclining Backrest

Series CP810

Optional accessories

Cod. **CPA506**
Table for wheelchair
58x38 cm

Spare parts

Cod. **CAR830**
Elevating Armrest
- PAIR

Cod. **CAR168**
Elevating footrest
- PAIR

Cod. **CAR217**
PU rear wheel - PAIR

Cod. **CAR226**
PU front wheel - PAIR

Cod. **CAR314**
Anti-tipper wheel - PAIR

Technical features

SPECIFICATIONS																
Front wheels	Grey, PU, aluminium fork															
Rear wheels	Aluminium radius, grey PU wheels															
Upholstery	Padded nylon															
WEIGHT AND DIMENSION																
Measures in cm																
CP810-40								40			62					
CP810-43	116	92	46	138	70	50	50÷64	43	89	65	33	20	60	20	5	100
CP810-46								46			68					

Accessories for Comfy-S, Comfy-S Go!, Comfy

Cod. **CPA100-40, CPA100-43, CPA100-45**
Foam cushion h 5 cm

Cod. **CAR605**
Antitheft bar

Cod. **CPA130**
Footrest
(for sizes 46 only)

KOALA Restraining system line

Cod. **RP247**
Safety belt with spring buckle

Cod. **RP245**
Safety belt
(CP810 only)

Cod. **RCA120**
IV pole (image for illustrative purpose only)

Cod. **CPA506**
Table for wheelchair
58x38 cm

Cod. **CPA501**
Linear table
55x32 cm

Spare parts for Comfy-S, Comfy-S Go!

Cod. **CAR831**
Desk type armrest
- PAIR (for CP800-CP805)

Cod. **CAR171**
Elevating footrest
- PAIR (for CP800-CP805 only)

Cod. **CAR173**
Anti-tipper wheel Ø 5 cm - PAIR (for CP800-CP805 only)

Cod. **CAR174**
PU rear wheel Ø 60 cm - PAIR (only for CP800)

Cod. **CAR172**
PU front wheel Ø 20 cm - PAIR (only for CP800)

Cod. **CAR175**
PU rear wheel - PAIR (only for CP805)

Spare parts for Comfy

Cod. **CAR830**
Elevating armrest - PAIR

Cod. **CAR168**
Elevating footrest - PAIR

Cod. **CAR217**
PU rear wheel - PAIR

Cod. **CAR226**
PU front wheel - PAIR

Cod. **CAR314**
Anti-tipper wheel - PAIR

- EN** Multifunction wheelchair - Painted steel
- F** Fauteuil roulant multifonction - Acier verni
- ES** Silla de rueda polifuncional - Acero pintado
- P** Cadeira de rodas multifuncional - Aço pintado

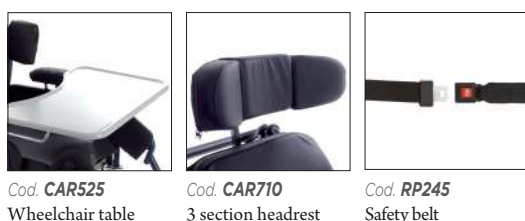
Rear wheels Ø 30 cm


Series CP900

Optional accessories

Spare parts

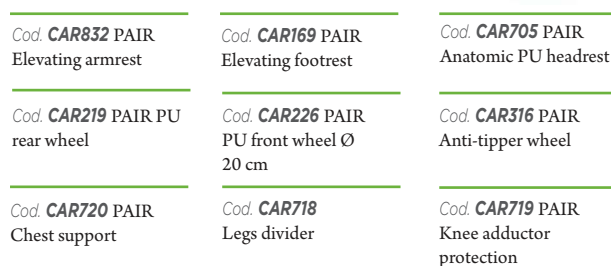Technical features

SPECIFICATIONS																	
Front wheels	Grey, PU, aluminium fork adjustable																
Rear wheels	Plastic radius, grey PU wheels																
Upholstery	Black washable nylon, removable																
Padding	Anatomic, in printed polyurethane																
Optional included	Adjustable divider - Knee protection - Chest support in height, width and angle adjustable - Anti-tipper wheels																
WEIGHT AND DIMENSION																	
Measures in cm																	Max loading weight
CP900-40	113	81	45	108	64÷77	57	40÷60	40	49	57	-	41	30	20	5	120	
CP900-45								45		62		43					

- EN** Multifunction wheelchair - Painted steel
- F** Fauteuil roulant multifonction - Acier verni
- ES** Silla de rueda polifuncional - Acero pintado
- P** Cadeira de rodas multifuncional - Aço pintado

Relief

Rear wheels Ø 60 cm

Series **CP910**

Optional accessories

Cod. **CAR525**
Wheelchair table

Cod. **CAR710**
3 section headrest

Cod. **RP245**
Safety belt

Spare parts

Cod. **CAR832** PAIR
Elevating armrest

Cod. **CAR218** PAIR PU
rear wheel Ø 60 cm

Cod. **CAR720** PAIR
Chest support

Cod. **CAR169** PAIR
Elevating footrest

Cod. **CAR226** PAIR
PU front wheel Ø 20 cm

Cod. **CAR718**
Legs divider

Cod. **CAR705**
Anatomic PU headrest

Cod. **CAR316** PAIR
Anti-tipper wheel

Cod. **CAR719** PAIR
Knee adductor protection

Technical features

SPECIFICATIONS																
Front wheels	Grey, PU, aluminium fork adjustable															
Rear wheels	Aluminium radius, grey PU wheels															
Upholstery	Black washable nylon, removable															
Padding	Anatomic, in printed polyurethane															
Optional included	Adjustable divider - Knee protection - Chest support in height, width and angle adjustable - Anti-tipper wheels															
WEIGHT AND DIMENSION																
Measures in cm																
CP910-40	113	81	45	108	64÷77	57	40÷60	40	49	60	-	45	60	20	5	120
CP910-45								45		65		46				

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**ARDEA ONE WHEELCHAIRS
INSTRUCTION MANUAL**

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CE I Class Medical Device
according to Directive 93/42/EEC
and further modifications

1. Codes

CP100-xx	Series START 60 cm wheels - Self-Propelled
CP102-xx	Series START1 60 cm wheels - Self-Propelled
CP110-xx	Series NEXT 60 cm wheels - Self-Propelled
CP115-xx	Series NEXT GO 30 cm wheels - Transit
CP200-xx	Series DUAL 60 cm wheels - Self-Propelled
CP205-xx	Series DUAL GO 30 cm wheels - Transit
CP300-xx	Series PLUS 60 cm wheels - Self-Propelled - Heavy duty
CP310-xx	Series PLUS GO 30 cm wheels - Transit - Heavy duty
CP500-xx	Series GO! 30 cm wheels - Transit
CP520-xx	Series GO UP! 30 cm wheels - Transit
CP620-xx	Series SKINNY 50 cm wheels - Small spaces
CP625-xx	Series SKINNY GO! 30 cm wheels - Transit - Small spaces
CP880-35	Series KIDDY 56 cm wheels - Self-Propelled

(XX =Refers to size available for each wheelchair model ex. 38-40-43-45-46-48-50-55-60 cm)

2. Introduction

Thank you for purchasing a **ARDEA ONE** wheelchair by Moretti S.p.A. product range. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

Note: check all parts for shipping damage and test before using. In case of damage, do not use the device. Contact your dealer for further instruction.

3. Intended use

This wheelchair is designed for the mobility of people with mobility problems and can be used for self-propelling (if equipped with big rear wheels) or with the aid of an operator.

WARNING!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without notice

4. EC declaration of conformity

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the wheelchair category are compliant with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications. To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of **directive 93/42/EEC** as prescribed by Annex VII of the said directive
2. The devices in question **ARE NOT MEASUREMENT DEVICES.**
3. The devices in question **ARE NOT INTENDED FOR CLINICAL INVESTIGATION**
4. The devices in question are sold in **NON STERILE** packing
5. The devices in question should be considered as belonging to class I
6. **DO NOT** use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
7. Moretti S.p.A. maintains and submits the documentation certifying conformity with **Directive 93/42/EEC** and further modifications to the proper authorities

5. General warning

- **DO NOT** use this product or any available optional equipment without first completely reading and understanding this instruction manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before using this equipment otherwise injury or damage may occur.
- Advise with physician or therapist to determinate the correct adjustment and the correct use of the wheelchair
- Keep the packed wheelchair away from heat sources
- **SERVICE LIFE**- the wheelchair use limit is defined by the wear of the parts
- **ALWAYS** keep hands and fingers clear of moving parts to avoid injury
- **DO NOT** allow children to play on or operate the wheelchair
- Close the wheelchair before lifting it.
- **DO NOT** attempt to lift the wheelchair by any removable parts

6. General description

6.1 Parts view

The components can change depending on the model you choose:

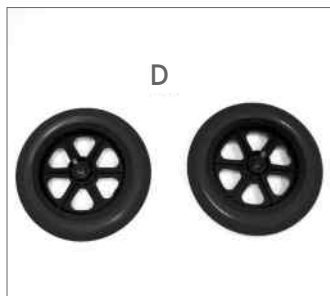
A - Frame


B - Wrenches kit for assembling


C - Swing away legrest

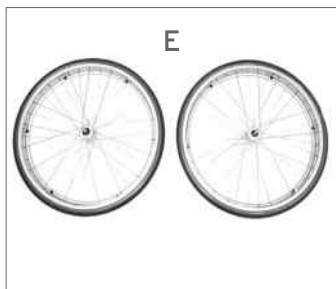
D - 30 cm casters
(diameter 30 cm) Transit model

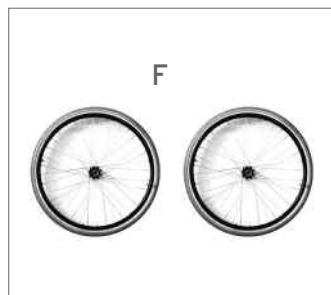E - 60 cm Wheels
(diameter 60 cm) Self-propelled model

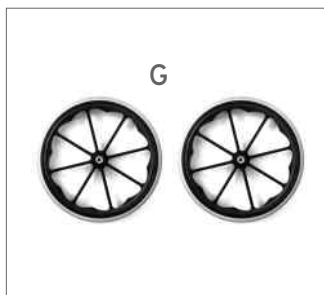
F - 50 cm Wheels for small spaces
(diameter 50 cm) Self-propelled model

G - 56 cm Wheels - Kids model
(diameter 56 cm) Self-propelled model

7. Assembling/disassembling

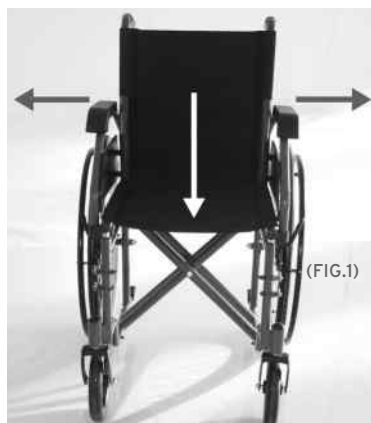7.1 Opening

Note: make reference to safety cautions in this manual

- Get the handles and pull outward, then press the seat until the wheelchair is completely open (FIG.1)

7.2 Closing

- Lift powerfully the seat and close the wheelchair. Push the handles close (FIG.2)

7.3 Install/remove the armrest


CP100-xx / CP102-xx / CP110-xx / CP115-xx / CP500-xx / CP520-xx / CP880-35

CP300-xx / CP310-xx / CP200-xx / CP205-xx

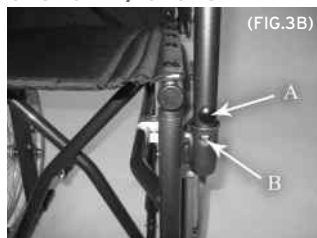CP620-xx / CP625-xx

- A - Push button
- B - Pin for stop

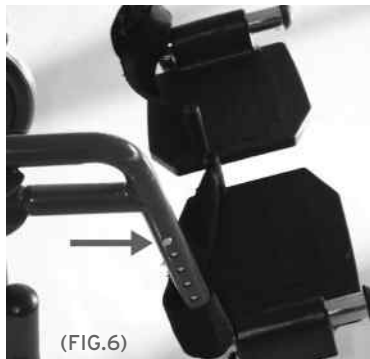
- Place the armrest rear side into the rear, then hook them into the front by overturning them (FIG.3, FIG.3A)
- Insert the tubes in their supports triggering properly locking clip (FIG. 3B)
- To remove, unlock the armrest by pushing the front release lever, then overturn them and extract by pulling the rear release lever (FIG.4, FIG.4A)
- For removal press the release button (A) and then lift to up (FIG. 3B)

7.4 Install/remove the swing away or elevating legrest

- Assembly the swing away or elevating legrest by installing the hinge plates on the footrest onto the hinge pins on the wheelchair frame. Push the legrest toward inside of the wheelchair until it locks into place (FIG.5)
- Remove the legrest by pushing the legrest release lever in ward, rotate outward, lift the legrest off the hinge pins

7.5 Adjust the swing away legrest

- To adjust the length of the footrest, unlock the safety lever at the bottom on the left side, then press the push button, slide the tube until the desired hole, then relock the safety lever. (FIG.6)

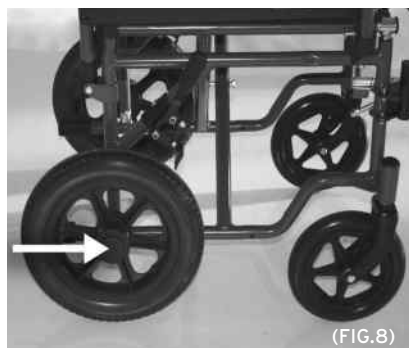
7.6 Adjust the elevating legrest (OPTIONAL)

- To adjust the length of the footrest, unlock the safety lever at the bottom on the left side, then press the push button, slide the tube until the desired hole, then relock the safety lever. (FIG.7) (Rif.P)
- Adjust the legrest height by pushing the lever (FIG.7) (Rif.L), and release to lock at the desired height

7.7 Install/remove the 30cm wheels (only for NEXT, DUAL and SKINNY)

Place the caster into the hole on the wheelchair frame

(FIG.8)

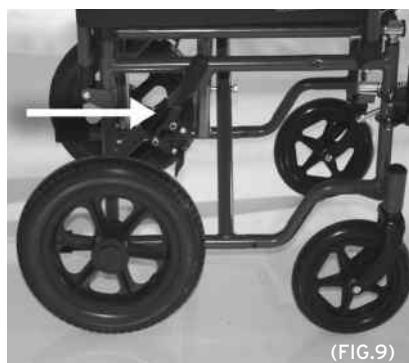
Only for quick release wheels

- Remove the plastic cap and push the wheel until it locks into the place (FIG.8)
- Remove the wheels: push the button on the centre of the wheel, and extract the wheel (FIG.8)

Only for wheels with fixed axle

- Place the small wheel and block it with the nut
- To remove the wheel, unscrew the nut and remove the small wheel

7.8 Adjust the brakes with 30cm wheels (only for NEXT, DUAL and SKINNY)

(FIG.9)

- When the 30 cm wheels are installed, place the brake on the milling. Lock the device by tightening the Allen wrenches (FIG.9)

Note adjust the brake device so that it can make the right pressure to securely lock the wheel

7.9 Install/remove the 60cm wheels (only for NEXT GO, DUAL GO and SKINNY GO)

Place the wheel into the central hole on the wheelchair frame

(FIG.10)

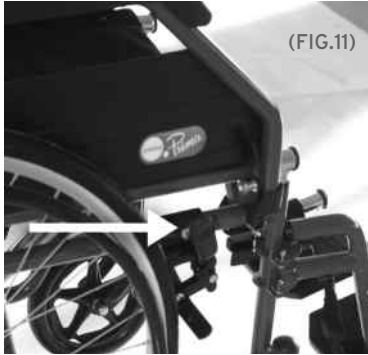
Only for quick release wheels

- Remove the plastic cap and push the wheel until it locks into the place (FIG.10)
- Remove the wheels: push the button on the centre of the wheel, and extract the wheel (FIG.10)

Only for wheels with fixed axle (SKINNY GO)

- Place the big wheel and block it with the nut
- Unscrew the nut and remove the big wheel

7.10 Adjust the brakes with 60cm wheels (just for DUAL GO)

- When the 60 cm wheels are installed, place the brake on the milling. Lock the device by tightening the Allen wrenches (FIG.11)

Note adjust the brake device so that it can make the right pressure to securely lock the caster

8. Before use

- Inspect the frame for any damage. Check the tubing for cracks and rust, in order to guarantee the safe for people ant thing
- Check the correct assembly, in order to guarantee a safe use
- Check the seat is completely open. If not push the seat down from the lateral frame.
- Check the brake correctly works

9. Using warnings

- Check the automotive parts usage, in order to guarantee a safe use of the device for people and things
- Assure that the seat is completely open and locked
- Assure that all parts are correctly assembled. Checking carefully the correct assembling of the wheels
- Assure the brakes correctly works

10. How to use

Note: make reference to "GENERAL WARNINGS" chapter

WARNING!

- Getting in and out of the wheelchair may be performed independently, provided you have a adequate mobility and upper body strength
- Do not stand on removable parts

DO NOT STAND ON THE FOOTRESTS WHEN GETTING IN OR OUT OF THE WHEELCHAIR, OVERTURNING RISK!!!

10.1 Get in the wheelchair

1. Position the wheelchair as close as possible to you
2. Engage wheel locks . We suggest to remove the armrest to ease the patient getting in the wheelchair
3. Lean on the frame with arms and slowly go down towards the seat

10.2 Get out of the wheelchair

1. Position the wheelchair as close as possible along side the seat to which you are transferring
2. Engage wheel locks. We suggest to remove the armrest to ease the patient getting out of the wheelchair
3. Shift body weight into seat with transfer

10.3 Stairs and height difference

**WARNING!**

The following methods must be used only from experienced patients. If you are not experienced, let you help from an assistant!

GET ON

Position the wheelchair at the curb to be crossed. Balance the wheelchair on the rear wheels, in order to lift the front wheels, enough to lean them on the obstacle. Bend your trunk forward and vigorously push the handrail. Continue pushing until the rear wheels are climbing over the obstacle.

GET OFF

To get off the stair follow the reverse procedure, positioning the rear side of the wheelchair at the curb, keep the handrails vigorously and get off the obstacle slowly; then balance the wheelchair on the rear wheels in order to complete the descent

**WARNING!**

We recommend to use this method only in presence of obstacle lower than 10cm!

10.4 Use the armrest

The armrest can be flip back, detachable or flip back and detachable

Flip back - this feature is useful when you have to get in or out of the wheelchair.

Push the release lever then turnover the armrest

10.5 Use the 60cm wheels

The rear 60cm wheels are used when self-propelled.

Grasp the handrail and wheel through the desired direction

10.6 Use the 30cm wheels

The rear 30cm wheels are used when wheel chair is driven by an assistant through the handles

10.7 Use the narrow casters (only on models with quick release wheels)

The safety wheels are used to cross narrow spaces (elevator, etc.)

To use the safety wheels remove the rear 60cm wheels

10.8 Carrying

The wheelchair has been designed to be easily carried.

Once the armrest and the footrest are removed, can be closed and easily positioned in the car boot, for example. If this can't be possible, position the wheelchair in front of the car rear seat so that it is stable and cannot slip or slide. Lock the wheelchair with the seatbelt

11. Maintenance

The **ARDEA ONE** devices by Moretti are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years. In case of repair use only original spare parts. Before using follow the "BEFORE USE" chapter.

Moretti **ARDEA ONE** wheelchairs require few elementary and periodic maintenance, below we list the checks to be carried out periodically

braking capacity	monthly
seat cleaning	as needed

12. Cleaning and disinfection

12.1 Cover and padding

The cover and padding are in Nylon. Clean with warm water and mild soap.

12.2 Frame and metallic parts

Clean with warm water and mild soap. Dry with a soft cloth in order to not scratch the painting.

12.3 Brakes and wheels

Clean the wheels with warm water and mild soap. Pay attention in drying the wheels, it's dangerous use the wheelchair with wet wheels.

12.4 Disinfection

if you need to disinfect the device use a common disinfectant cleaner

Note: do not use solvent and abrasive product

13. Disposal

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the wheelchair in the appropriate disposal areas for recycle.

14. Spare parts/accessories

For spare parts only refer to MORETTI master catalogue

15. Technical features

15.1 Dimensions

DIMENSION AND WEIGHT

CP100-38								38		56							
CP100-40								40		58							
CP100-43	113	83	43	90	74	49	58	43	43	61	27	17,5	60	20	-		120
CP100-45								45		63							
CP100-46								46		63,5							
CP100-48								48		65							
CP102-38								38		56							
CP102-40								40		58							
CP102-43	113	83	43	90	74	49	58	43	43	61	27	17,5	60	20	-		120
CP102-45								45		63							
CP102-46								46		63,5							
CP102-48								48		65							
CP110-40								40		64							
CP110-43								43		67							
CP110-45	105	81	43	90	73	49	56	45	42	69	26	18,5	60	20	-		150
CP110-46								46		69,5							
CP110-48								48		71							
CP110-50								50		74							
CP200-38								38		58							
CP200-40								40		60							
CP200-43								43		63							
CP200-45	113	81	42	90	71	49	59	45	41	65	29	17,5	60	20	4		150
CP200-46								46		65,5							
CP200-48								48		68							
CP200-50								50		70							
CP300-50								50		73							
CP300-55	118	90	48	92	76	52	59	55	41	78	32	26	60	20	-		200
CP300-60								60		83							
CP620-40								40		51							
CP620-43	100	73	41	92	80	53	47	43	37	54	26	16,5	50	18	-		110
CP880-35	94	74	36	86	62	42	38	35	37	56	31	14,5	56	15	-		100

DIMENSION AND WEIGHT

CP115-40								40		53							
CP115-43								43		56							
CP115-45								45		58							
CP115-46	98	75	41	92	76	50	56	46	43	58,5	25	18,5	30	20	-		150
CP115-48								48		61							
CP115-50								50		63							
CP205-38								38		52							
CP205-40								40		54							
CP205-43								43		57							
CP205-45	104	76	42	90	71	49	59	45	41	59	25	17,5	30	20	4		150
CP205-46								46		59,5							
CP205-48								48		62							
CP205-50								50		64							
CP310-50								50		63							
CP310-55	104	77	48	92	70	52	59	55	39	68	30	26	32	20	-		200
CP500-40								40		55							
CP500-43								43		58							
CP500-45	94	70	42	92	72	48	54	45	43	60	26	17,5	30	20	-		120
CP500-46								46		60,5							
CP500-48								48		63							
CP520-40								40		55							
CP520-43								43		58							
CP520-45	99	73	42	93	72	48	52	45	42	60	26	17,5	30	20	-		150
CP520-46								46		60,5							
CP520-48								48		63							
CP625-40								40		51							
CP625-43	88	61	41	92	80	53	47	43	37	54	26	16,5	30	18	-		110

*The dimensions may change $\pm 0,5$ cm

15.2 Technical specifications

- **Steel frame**
- **Max weight for CP100, CP102, CP500: 120 kg - for CP110, CP115, CP200, CP205, CP520: 150 kg - for CP300, CP310: 200 kg - for CP620, CP625: 110 kg - for CP880-35: 100 kg**
- **Front wheels: 20cm PU**
- **Front wheels (kids model): 15cm PU**
- **Rear small wheels: 30cm PU**
- **Rear big wheels: 60cm PU**
- **Rear big wheels (kids model): 56cm**
- **Fabric: Nylon**
- **Resistance to ignition:** the materials used for cover and padding have been treated with methods specified in Directive EN1021-1/2
- **Reference standards: EN12183**
- **Environmental temperature: -20°C/+ 40°C**
- **Storage:** store in a cool place, away from heat sources, such as sunlight (suggested temperature between +10°C and +40°C)

16. Warranty

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and for the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual. Moretti is not responsible for consequent damages, personal injuries or whatever caused by or in relation to mistaken installation or improper use. Moretti warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, alteration not authorized, shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

16.1 Repairing

Warranty repair: If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be cover from warranty. Moretti, at its unquestionable discretion, can repair or replace the item, by a Moretti dealer or to Moretti headquarters. Labor cost can be charged to Moretti if the repair is covered by warranty. A repair or a replacement doesn't extend the warranty.

Repair not cover by warranty: A product out of warranty can be sent after Moretti authorization. The labor and shipping costs for good out of warranty are to be paid by the customer or by the dealer. The repairs are guaranteed for 6 months from the good received.

Non defective device: The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be paid by the customer.

16.2 Spare parts

The original Moretti spare parts are guaranteed for 6 months from the receiving date.

16.3 Exempt clauses

Moretti does not offer any other declaration, explicit or implicit warranty or conditions, including possible declarations, warranties or conditions of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti does not guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be imposed by the law is limited by the warranty period, in the limits of law. Some states or countries don't allow limiting the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.

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DECLARATION OF CONFORMITY

Directive 93/42/EEC

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of

MANUAL WHEELCHAIRS - ARDEA ONE

complies with the

**European Directive on Medical devices 93/42/EEC
as modified by directive 2007/47/EEC**

and the following international standards

EN ISO 14971:2012 – EN ISO 15223-1:2017 – EN ISO 12183:2014

For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the essential requisites requested by the Annex I directive 93/42/EEC as laid down by the Annex VII of the above mentioned Directive.
2. The complete list of this range of medical devices is indicated in Annex I
3. The devices *ARE NOT MEASURING INSTRUMENTS*.
4. The devices *ARE NOT MADE FOR CLINICAL TESTS*.
5. The devices are packed in *NON-STERILE BOX*.
6. The devices belong to class I.
7. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 93/42/EEC directive, for at least 10 years from the last lot production.

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GIMA

HYRIDIA 7 LEDS LIGHT with flexible arm - trolley

Code: 49040
Category: Diagnostic, examination lights
Unit of sale: 1 pc.
Minimum order: 1
Type: Medical device
Class: I
NSIS: 1860820
CND: Z12010702
EAN13: 8023279490404

Description: **HYRIDIA 7 LEDS LIGHT - flexible arm - trolley**
7 leds 10W medical procedure lights on trolley with flexible soft arm or metal spring arm.
Digital brightness control.
Manual: GB, FR, IT, ES, PT, DE, PL, GR.
Handle on the lamphead.
Metal steering head makes direction more flexible.
360° Free position flexible metal arm, 76 cm length.
Detachable large base with brake.

Technical Specifications:

- Consumption: 10 W
- Colour temperature: 4,800-5,500 °K
- External Ø of reflector: 89 mm
- Colour rendering index (CRI): 75 ra
- Operating voltage: 110-265 V, 50/60 Hz
- Light intensity and light field Ø:
illumination 78,000 lux / Ø 9 cm / distance 30 cm
illumination 65,000 lux / Ø 12 cm / distance 50 cm
illumination 22,000 lux / Ø 18 cm / distance 100 cm

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HIGH VACUUM, HIGH FLOW HOSPITAL SUCTION ASPIRATOR

Standard Clinic Plus and Hospi Plus

Clinic Plus and Hospi Plus with foot switch and electronic flow direction regulator

New innovative range of high vacuum, high flow aspirator ideal for hospitals. Pump piston type, lubrication and maintenance free. Supplied with hydrophobic 99% bacterial filter, vacuum gauge for control of vacuum and 2 autoclavable jars (2 or 4 litres) made in makrolon with 200/400 ml graduation and double security valve. Available on request 2 or 3 litre disposable liners. Range includes version with foot

TECHNICAL SPECIFICATIONS

	Hospi plus	Clinic plus
Operating voltage:	110 V or 230 V - 50/60 Hz	230 V - 50/60 Hz - 110 V on request
Maximum suction:	- 0.90 bar (675 mmHg)	- 0.90 bar (675 mmHg)
Operating cycle:	continuous	continuous
Flow:	90 l/min	60 l/min
Power:	300 VA	230 VA
Norms:	CEI 62-5 (IEC 601-1) 93/42 EEC. Made in Italy	CEI 62-5 (IEC 601-1) 93/42 EEC. Made in Italy

Size: 460x850x420 mm
Weight: 20 kg

switch and flow direction regulator, 110 and 230 V. Four antistatic castors, two of which with brake, allow a perfect mobility. User manual in GB, FR, IT, ES, DE. See spare accessories at page 159, 166. Made in Italy.

GIMA code	CLINIC PLUS / HOSPI PLUS	Jar	Power	Foot switch	Flow direction diverter*
28194	CLINIC PLUS aspirator	2x2 l	220-230 V	-	-
28196	CLINIC PLUS aspirator	2x4 l	220-230 V	-	-
28198	CLINIC PLUS aspirator	2x4 l	220-230 V	yes	yes
28200	HOSPI PLUS aspirator	2x2 l	220-230 V	-	-
28199	HOSPI PLUS aspirator	2x2 l	110 V	-	-
28201	HOSPI PLUS aspirator	2x4 l	220-230 V	-	-
28203	HOSPI PLUS aspirator	2x2 l	220-230 V	yes	yes
28204	HOSPI PLUS aspirator	2x4 l	220-230 V	yes	yes

*Allow to direct suctioned liquids to any of the 2 jars

MAXI ASPEED HIGH FLOW ASPIRATOR FOR HOSPITAL USE

TECHNICAL SPECIFICATIONS

Operating voltage:	230 V - 50/60 Hz. Available on request 110 V - 60 Hz
Air flow:	60 or 90 l/min
Operating cycle:	continuous
Adjustable vacuum level:	0-0.90 bar (0-90 kPa)
Size:	470x580x560 mm
Weight:	15 kg (60 l), 17 kg (90 l)
MDD class:	Ila

MAXI ASPEED SURGICAL ASPIRATORS

Surgical aspirators for aspiration of body liquids in operating theatres and hospitals.

Easy to carry

Equipped with 4 antistatic wheels (2 with brake) and carrying telescopic handle.

Developed to run continuously

Made in highly heat resistant, electrically insulated plastic material in conformity with latest European safety standards. Vacuum regulator with BAR/PSI vacuometer and automatic flow direction regulator to easily switch from a jar to the other. Supplied with 2 autoclavable polycarbonate jars with safety valve (overflow protection), set of atoxic sterilizable silicone tubes, 99% antibacterial hydrophobic filters (2) and multilingual manual (GB, FR, IT, DE, ES). Made in Italy.

GIMA code	MAXI ASPEED	Jar	Power	Foot switch	Flow direction regulator
28285	Maxi Aspeed 60 l	2x2 l	230 V	-	yes
28286	Maxi Aspeed 60 l	2x2 l	230 V	yes	yes
28288	Maxi Aspeed 60 l	2x4 l	230 V	yes	yes
28289	Maxi Aspeed 90 l	2x2 l	230 V	-	yes
28291	Maxi Aspeed 90 l	2x4 l	230 V	-	yes
28292	Maxi Aspeed 90 l	2x4 l	230 V	yes	yes

ACCESSORIES

28294	Autoclavable jar 2 l with safety valve	
28295	Autoclavable jar 4 l with safety valve	
28297	Hydrophobic 99% antibacterial filter	
25482	Silicone tube - 8x14 mm - roll of 30m	

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▸ Brands

PRODUCT CATEGORY

- Wireless products
- Diagnostic tests - laboratory
- Health care - pharmacy
- Surgical instruments
- Medical bags
- Scales and measures
- Patient aids
- Furniture
- Electromedical devices
- Gynaecology
- Electrosurgery - cautery
- Loupes & mirrors
- Endoscopy
- Medical lights
- ENT devices
- Stethoscopes & sphygmometers
- Audiometry & spirometry
- Ecg, monitors & ultrasound
- First aid & emergency
- Sterilization
- Veterinary
- Human anatomy

List of Products with CH-UK REP

- Products with CH-REP
- Products with UK-REP

PRIVACY & COOKIE POLICY

- Privacy & cookie policy
- Legal note

HOSPI PLUS SUCTION 2x4 l jar 230V with footswitch, flow regulator

Category: Hospital suction pumps - aspirators

browse: << previous next >>

Stock availability: **51**

Product Code: **28204**

Unit of sale: **1 pc.**

Minimum order: **1**

Type: **Medical device** Class: **II A**

[Print data sheet](#)

[Download image](#)

NSIS: **590729**

CND: **Z120105**

EAN/UPC: **8023279282047**

GMDN: **36777**

HOSPI PLUS Full

- Autoclavable Jars: 2x4 l
- Footswitch: YES
- Flow direction diverter (Allows to direct suctioned liquids to any of the 2 jars): YES

Designed for professional aspiration of bodily fluids, tissues or bones of patients during or after surgery.

Oilless and maintenance free piston type pumps provide high performances. Excellent suction capacities and max vacuum built up within a few seconds.

Available with vacuum gauge and autoclavable jars made in makrolon with 200 ml (2 l jar) or 500 ml (4 or 5 l jars) graduation and 2-3 litres Flovac jars with disposable liners.

Wide range of versions with different features

- 60 l/m (Clinic Plus) or 90 l/m (Hospi Plus) flow rate
- 2, 4 or 5 l jars
- MPR system for versatility
- footswitch and flow direction diverter

High standards of safety in overflow protection system

Double security valve integrated in the jar and hydrophobic filter (all models), safety trap bottle (only MPR models).

MPR (Multi Purpose Rail) models

This system enhances the versatility for easy and quick exchange of different accessories, with no need for tools.

Equipped with five connections for rings of various diameters to fit jars of different sizes and types (2l, 3l, 5l), cannula holders and safety trap bottle.

Main applications: EMERGENCY DEPT. / GYNAECOLOGY / OPERATING THEATRE / GENERAL SURGERY / NEUROSURGERY / DENTAL PRACTICE / OBSTETRICS

Size: 460 x 850 x 420 mm

Weight: 20 kg

Made in Italy.

Technical Specifications

Operating voltage: 230 V - 50/60 Hz
 Maximum suction: - 0,90 bar (675 mmHg)
 Operating cycle: continuous
 Flow: 90 l/min
 Power: 385 VA
 Noise level: 46.4 dBA
 Norms: CEI 62-5 (IEC 601-1); 93/42 EEC

Standard accessories

- 121°C autoclavable jar with overflow valve system: 2x4 l
- Rings to accommodate 5 l jars: NO
- Safety Trap Bottle (220 ml): NO
- Antibacterial & Hydrophobic Filter (single-patient): 1
- Autoclavable silicone tubes ø 8x14 mm length 150 cm: 1
- Conical Connector ø 10-11-12mm: 1
- Air suction inlet: 1
- Footswitch with intermittent or continuous operation: 1
- Change-Over System from jar to jar by soft-touch keys: 1
- Power Cord with Schuko plug: 1
- Multilingual manual: GB, FR, IT, ES, DE, GR, PL, RO

In the same family are available

- "SUPER VEGA BATTERY ON TROLLEY" SUCTION ASPIRATOR
- "SUPER VEGA BATTERY ON TROLLEY" SUCTION ASPIRATOR with footswitch
- "SUPER VEGA ON TROLLEY" SUCTION ASPIRATOR
- "SUPER VEGA ON TROLLEY" SUCTION with footswitch

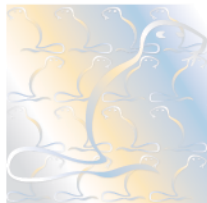
- ✕ CLINIC PLUS MPR SUCTION 2x5 l jar 230V
- ✕ CLINIC PLUS MPR SUCTION 2x5 l jar 230V with footswitch, flow regulator
- ✕ CLINIC PLUS SUCTION 2x2 l jar 110V
- ✕ CLINIC PLUS SUCTION 2x2 l jar 230V
- ✕ CLINIC PLUS SUCTION 2x2 l jar 230V with footswitch, flow regulator
- ✕ CLINIC PLUS SUCTION 2x4 l jar 110V with footswitch, flow regulator
- ✕ CLINIC PLUS SUCTION 2x4 l jar 230V
- ✕ CLINIC PLUS SUCTION 2x4 l jar 230V with footswitch, flow regulator
- ✕ HOSPI PLUS MPR SUCTION 2x5 l jar 230V
- ✕ HOSPI PLUS MPR SUCTION 2x5 l jar 230V with footswitch, flow regulator
- ✕ HOSPI PLUS SUCTION 2x2 l jar 230V
- ✕ HOSPI PLUS SUCTION 2x2 l jar 230V with footswitch, flow regulator
- ✕ HOSPI PLUS SUCTION 2x4 l jar 110V with footswitch, flow regulator
- ✕ HOSPI PLUS SUCTION 2x4 l jar 230V
- ✕ HOSPI PLUS SUCTION 2x4 l jar 230V with footswitch, flow regulator
- ✕ MAXI ASPEED SUCTION 60 l 2x2 l jar - 230V
- ✕ MAXI ASPEED SUCTION 60 l 2x2 l jar with footswitch - 230V
- ✕ MAXI ASPEED SUCTION 60 l 2x4 l jar with footswitch - 230V
- ✕ MAXI ASPEED SUCTION 90 l 2x2 l jar - 230V
- ✕ MAXI ASPEED SUCTION 90 l 2x2 l jar with footswitch - 230V
- ✕ MAXI ASPEED SUCTION 90 l 2x4 l jar - 230V
- ✕ MAXI ASPEED SUCTION 90 l 2x4 l jar with footswitch - 230V
- ✕ MINI ASPEED EVO BATTERY SUCTION 2 l
- ✕ TOBI CLINIC SUCTION ASPIRATOR with 2x2 l jars
- ✕ TOBI CLINIC SUCTION ASPIRATOR with 2x4 l jars
- ✕ TOBI CLINIC SUCTION with 2x2 l jar + footswitch
- ✕ TOBI CLINIC SUCTION with 2x4 l jars + footswitch
- ✕ TOBI HOSPITAL SUCTION with 2 x 4l jars - 230V
- ✕ TOBI HOSPITAL SUCTION with 2x4l jars+footswitch - 230V
- ✕ TOBI HOSPITAL" SUCTION ASPIRATOR with 2x2l jars - 230V
- ✕ TOBI HOSPITAL" SUCTION with 2x2l jars+footswitch - 230V

[Back](#)

Gima S.p.A. Via Marconi, 1 - 20060 Gessate (MI) Tel. ++39 02 953854209/221/225 - Fax ++39 02 95380056
 Email: export@gimaitaly.com - P.IVA:00734640154

All rights on the contents of this web site are reserved. Any reproduction by third parties is forbidden if not expressly authorized by a written authorization by GIMA. Every infringement will be pursued under applicable laws. Images and technical specifications are not binding. Products are subject to modifications without prior notice. We declare that some products carrying GIMA brand are manufactured by external OEM according to our specifications.

This website uses cookies. By continuing to browse the site, you accept the use of cookies in accordance with the terms of use.
 To learn more see the [Cookie Policy](#)

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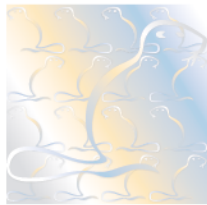
- Via Grossi, 2 20121 Milano - Italia

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- Via Marconi, 1 20060 Gessate (MI) - Italia

Reg. Number	10164 - A	Valid From	2021-10-14
First issue date	2012-10-15	Last change date	2021-10-14
Valid Until	2024-10-14	IAF Sector	29 a

Quality Management System Certificate **ISO 9001:2015**

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GIMA S.p.A.

Is in compliance with the standard UNI EN ISO 9001:2015 for the following products/services:

Trade, packaging and service of medical devices (MD), in vitro diagnostic products (IVD), personal protective equipments (PPE), biocides, veterinary items, medical accessories furniture and aids.

Chief Operating Officer
Giampiero Belcredi

The maintaining of the certification is subject to annual surveillance and dependent on the observance of Kiwa Cermet Italia contractual requirements.

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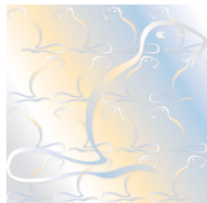
E-mail: info@kiwacermet.it

www.kiwacermet.it

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Reg. Number	10164 - M	Valid From	2021-10-14
First issue date	2012-10-15	Last change date	2021-10-14
Valid until	2024-10-14		

Quality Management System Certificate **ISO 13485:2016**

We certify that the Quality Management System of the Organization:

GIMA S.p.A.

Is in compliance with the standard UNI CEI EN ISO 13485:2016 for the following products/services:

Management of design and production, packaging and service of:

General non-active, non-implantable medical devices (except: injection, infusion, transfusion and dialysis; disinfecting, cleaning, rinsing; IVF, ART; ingestion), Devices for wound care, Non-active dental devices and accessories (except dental implants), General active medical devices (except: extra-corporal circulation, infusion and haemopheresis; stimulation or inhibition, rehabilitation devices and active prostheses; IVF, ART; software; medical gas supply systems and parts thereof), Monitoring devices, Devices for imaging and thermo therapy (except: ionizing radiation, lithotripsy), In Vitro Diagnostic Medical Devices (IVD).

Trade and service of: General non-active, non-implantable medical devices (except: IVF, ART; ingestion), Devices for wound care, Non-active dental devices and accessories (except dental implants), General active medical devices (except IVF, ART), Devices for imaging (except ionizing radiation), Monitoring devices, Devices for radiation therapy and thermo therapy (except: ionizing radiation, lithotripsy), In Vitro Diagnostic Medical Devices(IVD)

Chief Operating Officer
Giambiero Belcredi

The maintaining of certification is subject to annual surveillance and dependent upon the observance of Kiwa Cermet Italia contractual requirements.

This certificate is composed of 1 page.

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- Via Marconi, 1 20060 Gessate (MI) - Italia

EU DECLARATION OF CONFORMITY

According to Art. 19 of Regulation (EU) 2017/745 on Medical Devices

Manufacturer: Shantou Easywell Electronic Technologies Co.,Ltd
NO.1 West side of 6th Floor H5 Industrial Building, No. 16
Lianjiang Road,Longhu district,Shantou,China(515041)

Trademark: Easywell

European Representative: Kingsmead Service B.V.
Zonnehof 36, 2632 BE, Nootdorp, Netherland

SRN NL-AR-000002066

Trade name: Medical lights

Product name: Medical lights

Product code / Catalogue number: KS Series, MST Series, SL Series, LED Series

Basic UDI 697447153000L4 / 697447153001L6/
697447153004LC

Classification acc. to MDR Ax. VIII: Class I, rule 13

Applied Standard & Common Specification: EN ISO14971:2012 , EN1041:2008, EN60601-1:2006+A1:2013, EN60601-1-2:2015, EN60601-2-41:2009

Conformity assessment procedure: Annex II + Annex III of MDR

We, the manufacturer, herewith declare under our sole responsibility that the above-mentioned products meet the provisions of the Regulation (EU) 2017/745 on Medical Devices(MDR). All supporting documentations are retained under the premises of the manufacturer.

Justin Liu
General Manager

Shantou City, Guangdong
Province, China 10. 03. 2021

MANAGEMENT SYSTEM CERTIFICATE

Certificate no.:
CERT-16906-2006-AQ-IND-SINCERT

Initial certification date:
04 January 2006

Valid:
16 December 2020 – 15 December 2023

This is to certify that the management system of

Moretti S.p.A.

Via Bruxelles, 3 - Loc. Meleto - 52022 CAVRIGLIA (AR) - Italy

has been found to conform to the Quality Management System standard:

ISO 13485:2016

This certificate is valid for the following scope:

Design, production and management of the production, distribution and placing on the market of durable medical equipment in the rehabilitation and home healthcare and devices for patient positioning and transport. Marketing and distribution of general non-active, non-implantable medical devices and Devices for wound care (sterile and non-sterile), Monitoring devices, Device for home care, Active rehabilitation devices

Place and date:
Vimercate (MB), 23 December 2020

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For the issuing office:
DNV GL - Business Assurance
Via Energy Park, 14, - 20871 Vimercate
(MB) - Italy

Zeno Beltrami
Management Representative

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