



Realizzazioni grafiche e cartotecniche
Progettazione packaging
Grafica pubblicitaria

LD LEONARDO Design S.r.l.

Via Pontina Vecchia Km 31.800 - 00071 Pomezia (Roma)

Tel.: +39 06.91607112 - Fax: +39 06.91251182

e-mail: info@leonardodesign.it

IMPORTANTE: unitamente alla verifica dei testi, si prega di controllare le dimensioni e la corrispondenza dei colori

Cliente **CANTEL**
Codice articolo 002413/03
Fustella
Formato Ap. 150X120
Corpo
Cod. Base 32
N. Lavorazione

Prodotto **NEO PROTEOZIM PLUS 500 - 1000 ml -UK**
Codice Laetus
Disegno N°
Formato Ch.
Supporto
Spessore carta
Codice Datamatrix

293

032

NERO

Numero Bozza

1

2

3

4

5

6

7

25/03/2020



NEO PROTEOZIM Plus

500

MULTI-ENZYME DETERGENT FOR DECONTAMINATION AND DISINFECTION

Description: effective multi-enzyme detergent, decontaminant, and disinfectant for invasive and non-invasive medical devices that require thorough cleaning in channels and indentations. It is formulated with high-purity enzymes that ensure the immediate dissolution of fats, starches, proteins and other materials from difficult to access areas of the instruments. The disinfectant activity due to the synergy between the various components has been verified for its bactericidal, virucidal, and fungicidal activity according to the standard CEN/216 and Italian Ministerial Decree 98-9-90.

Composition: total enzymes 8.0 g - cationic and non-ionic surfactants 51.0 g - DPTA 0.3 g - Coformulants 13.0 g - Purified water as needed to 100 ml.

Method of use: used diluted 1:500 for the decontamination of dirty instruments immediately following use, for cleaning and disinfecting medical devices, in particular endoscope instruments. Immerse the instruments for 10 minutes, then rinse and dry. Used in ultrasonic tanks, the detergent and decontamination time can be reduced to 5 minutes.

Instructions for use: to prepare the solution 1:500, pour 10 ml of product and add water to reach a final volume of 5 litres.

Warnings: Danger. Keep out of the reach of children. Keep at ambient temperature, keep away from heat sources. The expiration date refers to the product in an unopened packaged that has been stored correctly. Do not use after the expiration date.

(H) Hazard statements: (H225) Highly flammable liquid and vapour. (H290) May be corrosive to metals. (H302) Harmful if swallowed. (H314) Causes severe skin burns and eye damage. (H317) May cause an allergic skin reaction. (H412) Harmful to aquatic organisms with long-lasting effects.

(P) Precautionary Statements: (P210) Keep away from heat sources, hot surfaces, sparks, open flames and other ignition sources. Do not smoke. (P273) Avoid release to the environment. (P280) Wear protective gloves / protective clothing and eye / face protection. (P303+P361+P353) IN THE EVENT OF CONTACT WITH SKIN (or with hair): Remove all contaminated clothing immediately. Rinse the skin / take a shower. (P305+P351+P338) IN CASE OF CONTACT WITH EYES: Rinse thoroughly with water for several minutes. Remove contact lenses if present and easy to do. Continue rinsing. (P390): Absorb leaks to avoid material damages.

Contains: Quaternary ammonium compounds, benzyl-C12-16 alkyl dimethyl, chlorides

POLY(OXY-1,2-ETHANEDIYL), ALPHA-TRIDECYL-OMEGA-HYDROXY-, BRANCHED/

Mixture of: 5-chloro-2-methyl-2H-isothiazol-3-one [EC no. 247-500-7], 2-methyl-2H-isothiazol-3-one [EC no. 220-239-6] (3:1)

SUBTILISIN

When diluted for use, the product has no contraindications.

Contents: 1000 ml



CANTEL MEDICAL (Italy) S.r.l.
Via Laurentina, 169
00071 POMEZIA (RM) - Italia
Tel.: +39.06.9145399
Fax: +39.06.9146099
www.cantelmedical.it

Emergency number: +39.069145399



CE
0051

LOT




Col. 002413 Rev.3 - 03/2020

Senso di svolgimento

3



Numerazione:

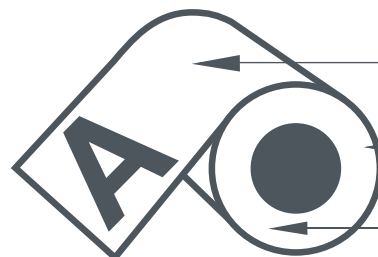

Etichette per bobina

Diametro Esterno

Anima interna

Copia da restituire firmata via fax
o per scansione via email