ALSL

ALSL-0250	R1 8 x 20 mL	+ R2 8 x 5 mL
ALSL-0455	R1 4 x 44 mL	+ R2 4 x 11 mL
ALSL-0410	R1 2 x 50 mL	+ R2 1 x 26 mL
ALSL-0430	R1 4 x 50 mL	+ R2 2 x 26 mL
ALSL-0510	R1 5 x 100mL	+ R2 1 x 127 mL

FTRO-ALSL-v20 (02/2022) (PIT-ALSL-4-v20)

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☞ BIBLIOGRAFIE

1. Panteghini, M., Bais, R., *Enzymes, Tietz Fundamentals of Clinical Chemistry*, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E., (W.B. Saunders eds.), (2008), 317.
2. Wu, A.H.B., *Tietz Clinical guide to laboratory test*, 4th Ed., (W.B. Saunders eds.), (2006), 64.
3. Dufour, R., *The liver: Function and Chemical Pathology, Clinical Chemistry: Theory Analysis, Correlation*, 5th Ed., Kaplan, L.A., Pesce, A.J., (Mosby Inc. eds.), (2010), 586 and appendix.
4. Schumann, G., et al., *Clin Chem Lab Med.*, (2002), **40**, 718.
5. Guder, W.G., et al., *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2.
6. Berth, M. & Delanghe, J., *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, *Acta Clin Belg.*, (2004), **59**, 263.
7. Young, D.S., *Effects of preanalytical variables on clinical laboratory tests*, 2nd Ed., AACC Press, (1997).
8. Young, D.S., *Effects of drugs on clinical laboratory tests*, 4th Ed., AACC Press, (1995).

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☞ BIBLIOGRAFIE

1. Wu, A.H.B., Clinical guide to laboratory tests, 4th Ed., (W.B. Saunders eds.), (2006), 66.
2. Dufour D. R., The Liver: Function and Chemical Pathology. Clinical Chemistry: Theory, Analysis, Correlation, 5th Ed., Kaplan, L.A, Pesce, A.J., (Mosby Inc. eds), (2010), 586 and appendix.
3. Johnson, A. M., Amino Acids and Proteins. Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E. (W.B. Saunders eds.), (2008), 286.

4. Doumas, B. T., et al., Albumin standards and the measurement of serum albumin with bromocresol green, Clin Chim Acta (1971), **31**, 87.

5. Berth, M. & Delanghe, J. Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature, Acta Clin Belg., (2004), **59**, 263.

6. Young, D.S., Effects of preanalytical variables on clinical laboratory tests, 2nd edition, AACC Press (1997).

7. Young D.S., Effects of drugs on clinical laboratory tests, 4th edition, AACC Press (1995).

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Albumin.
101

0
PIT-ALBU

ELITech Clinical Systems SAS
Zone Industrielle-61500 SEES Franța
www.elitechgroup.com

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BIBLIOGRAFIE

1. Panteghini, M. & Biais, R., *Enzymes*, Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E., (W.B. Saunders eds.), (2008), 317.
2. Dufour, D.R., *The Pancreas: Function and Chemical Pathology*, Clinical Chemistry: Theory Analysis, Correlation, 5th Ed., Kaplan, L.A., Pesce, A.J., (Mosby Inc. eds.), (2010), 651 and appendix.
3. Winn-Deen, E.S., *et al*, Clin. Chem., (1988), **34**, 2005
4. Guder W.G., *et al.*, *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma, and serum samples*, (2002). WHO/DIL/LAB/99.1 Rev.2.
5. Schumann, G., *et al.*, Clin. Chem. Lab Med., (2006), **44**, 1146.
6. Berth, M. & Delanghe, J., *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, Acta Clin Belg., (2004), **59**, 263.
7. Young, D.S., Effects of preanalytical variables on clinical laboratory tests, 2nd Ed., AACC Press, (1997).
8. Young, D.S., Effects of drugs on clinical laboratory tests, 4th Ed., AACC Press, (1995).

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BIBLIOGRAPHIE/BIBLIOGRAPHY
BIBLIOGRAFÍA/BIBLIOGRAFIA

1. Panteghini, M., Bais, R., *Enzymes, Tietz Fundamentals of Clinical Chemistry*, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E., (W.B. Saunders eds.), (2008), 317.
2. Wu, A.H.B., *Tietz Clinical guide to laboratory test*, 4th Ed., (W.B. Saunders eds.), (2006), 64.
3. Dufour, R., *The liver: Function and Chemical Pathology, Clinical Chemistry: Theory Analysis Correlation*, 5th Ed., Kaplan, L.A., Pesce, A.J., (Mosby Inc. eds.), (2010), 586 and appendix.
4. Schumann, G., *et al., Clin Chem Lab Med.*, (2002), 40, 718.
5. Guder, W.G., *et al., Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples.* (2002). WHO/DIL/LAB/99.1 Rev.2.
6. Berth, M. & Delanghe, J., *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature, Acta Clin Belg.*, (2004), 59, 263.
7. Young, D.S., *Effects of preanalytical variables on clinical laboratory tests*, 2nd Ed., AACCC Press, (1997).
8. Young, D.S., *Effects of drugs on clinical laboratory tests*, 4th Ed., AACCC Press, (1995).

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- Symbols used on our documentation are defined on ISO-15223-1 standard, except for some presented in the symbols glossary available on the ELITech Website. (Symbols glossary).

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- **Special Programming instructions :**
see § INSTALLATION AND USE

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English - EN

INTENDED USE

ELITech Clinical Systems AST/GOT 4+1 SL is an *in vitro* diagnostic reagent intended for the quantitative determination of aspartate aminotransferase (AST) in human serum and plasma samples on analyzers or semi-automatic analyzers.
This *in vitro* diagnostic device is for professional use only.

CLINICAL SIGNIFICANCE ⁽¹⁻³⁾

Aspartate aminotransferase (AST) is a transaminase also known as glutamate oxaloacetate transaminase (GOT). AST catalyzes the transfer of the aminogroup of L-aspartate to α-ketoglutarate to give L-glutamate. AST is present in both mitochondria and cytoplasm and is widely distributed in the body, with high concentrations in the heart, liver, skeletal muscles and kidneys. Damage to any of these tissues may cause raised levels. In case of acute forms of hepatitis, especially viral hepatitis, the enzyme level is markedly elevated. In case of myocardial infarction, AST activity increases and reaches a peak after 18-24 hours. The activity falls back to normal after 4-5 days, provided no new infarct has occurred.

The following pathological states are examples of disorders also resulting in an increase of enzyme activity: other liver injury (e.g. chronic hepatitis, liver carcinoma, intake of alcohol, administration of drugs), muscular dystrophy, other damage to cardiac or skeletal muscles and acute pancreatitis.

AST measurement is indicated to screen patients at risk of liver diseases and to help diagnose or monitor liver diseases.

LIMITATION OF USE

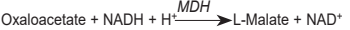
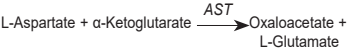
The quantitative assay of aspartate aminotransferase (AST) alone cannot be used to diagnose a disease or a specific pathology.

The results must be interpreted in conjunction with other diagnostic test results, clinical findings and the patient's medical history.

METHOD & PRINCIPLE ⁽⁴⁾

IFCC method without pyridoxal phosphate (P-5'-P).

Kinetic.

MDH = Malate dehydrogenase.

COMPOSITION

Reagent 1 : R1

Tris buffer, pH 7.80 (30°C)

L-Aspartate 330 mmol/L

LDH ≥ 2 000 U/L

MDH ≥ 1 000 U/L

Sodium azide < 0.1% (w/w)

Reagent 2 : R2

α-Ketoglutarate 78 mmol/L

NADH 1,1 mmol/L

Sodium azide < 0.1% (w/w)

MATERIALS REQUIRED BUT NOT PROVIDED

- CALI-0550 ELICAL 2

- CONT-0060 ELITROL I

- CONT-0160 ELITROL II

- Normal saline solution (NaCl 9 g/L).

- Analyzers or semi-automatic analyzers.

- General Laboratory equipment (e.g. pipette).

- Do not use materials that are not required as indicated above.

PRECAUTIONS FOR USE AND WARNINGS

- Consult Safety Data Sheet (SDS) for a proper handling.

- The reagents contain sodium azide which may react with lead or copper plumbing to form potentially explosive metal azides. When disposing of these reagents always flush with copious amounts of water to prevent azide buildup.

- Take normal precautions and adhere to good laboratory practice.

- Use clean or single use laboratory equipment only to avoid contamination.

- Do not interchange reagent vials from different kits.

STABILITY

Store at 2-8 °C and protect from light. Do not freeze.

Do not use after expiration dates indicated on the vial labels.

On board stability :

The on-board stability is specific for each analyzer. (Refer to § PERFORMANCE DATA).

PREPARATION

The reagents are ready to use.

PRODUCT DETERIORATION

- The product should be clear. Cloudiness would indicate deterioration.

- Do not use the product if there is visible evidence of contamination or damage (e.g. particle matter).

- Damage to the product container may impact on product performance. Do not use the product if there is physical evidence of deterioration (e.g. leakages or punctured container).

SAMPLES

Specimen ^(2,8)

- Serum.

- Plasma (lithium heparin).

- Using any other specimen type should be validated by the laboratory.

Warnings and precautions

- Samples must be free from hemolysis. ^(1,3)

- Samples should be collected in accordance with Good Laboratory Practice and appropriate guidelines that may be in place.

Storage and stability ^(2,5)

- 24h at room temperature.

- 7 days at 2-8 °C.

- 3 month at -20 °C.

REFERENCE VALUES ⁽⁴⁾

<i>Serum/plasma</i>	U/L	µkat/L
Men	≤ 35	≤ 0.58
Women	≤ 31	≤ 0.52

Reference values for infants are higher than for adults.

Note : The quoted range should serve as a guide only. It is recommended that each laboratory verifies this range or establishes a reference interval for the intended population.

❖INSTALLATION AND USE

For use on Selectra Pro analyzers :

- Consult operator manual.

- Special Programming instructions: Programming special instructions is mandatory when some combinations of tests are performed together on the analyzer. Refer to Instructions For Use of ACID SOLUTION & SYSTEM CLEANING SOLUTION for adequate programming (See PIT-SOL).

PROCEDURE

Manual Procedure

Wavelength : 340 nm

Optical path : 1 cm

Sample/reagent ratio : 1:20

Temperature : 37 °C

Read against distilled water.

Working reagent (4 volumes of R1 + 1 volume of R2)	1000 µL
Sample	50 µL

Mix and after 1 minute incubation, read absorbance at 1 minute intervals during 3 minutes. Calculate the change of absorbances per minute (ΔA/min)

Automatic Procedure

- These reagents may be used in several automatic analyzers. For ELITech Selectra Analyzers, validated applications are available on request. For Selectra TouchPro software, use the application included in the barcode available at the end of this insert.

- High AST values may induce falsely low results due to the depletion of the substrate (total consumption of NADH before reading of the result). For ELITech Selectra Analysers, the application contains a specific alarm to warn the users.

CALCULATION

Activity (U/L) = ΔA /min x 3333

Conversion factor : U/L x 0.0167 = µkat/L

CALIBRATION

ELICAL 2 is traceable to IFCC reference method.

Calibration frequency : The calibration is specific for each analyzer. (Refer to § PERFORMANCE DATA).

QUALITY CONTROL

It is recommended that quality control sera such as ELITROL I and ELITROL II be used to monitor the performance of the assay.

Controls have to be performed :

- prior to assaying patient samples,

- at least once per day,

- after every calibration,

- and/or in accordance with laboratory and regulatory requirements.

Results should be within the defined ranges. If values fall outside of the defined ranges, each laboratory should take necessary corrective measures.

WASTE MANAGEMENT

Disposal of all waste material should be in accordance with local, state and federal regulatory requirements (please refer to the Safety Data Sheet (SDS)).

PERFORMANCES

Performances were obtained on Selectra ProM, following CLSI technical recommendations, under controlled environmental conditions.

- Measuring range

10.0 to 450.0 U/L (0.17 to 7.50 µkat/L).

Samples having greater concentrations should be diluted 1:10 with NaCl 9 g/L solution and re-assayed. This procedure extends the measuring range up to 4500.0 U/L (75.00 µkat/L).

Do not report results outside this extended range.

For users with Selectra TouchPro software, the «dilute» function performs the sample dilution automatically. Results take the dilution into account.

Limit of Detection (LoD) and Limit of Quantification (LoQ)

LoD = 2.5 U/L (0.04 µkat/L)

LoQ = 5.0 U/L (0.08 µkat/L)

- Precision

Imprecision data has been obtained on 2 Selectra ProM analyzers over 20 days (2 runs per day, tests performed in duplicate).

Representative results are presented below.

		Mean		Within-run	Total
	n	U/L	µkat/L	CV (%)	
Level 1	80	34.1	0.57	1.7	3.4
Level 2	80	67.9	1.13	0.8	1.9
Level 3	80	353.6	5.89	0.4	2.0

- Correlation

A comparative study has been performed between AST/GOT 4+1 SL reagent on a Selectra ProM analyzer and a similar commercially available system on 114 human serum samples. The sample concentrations ranged from 10.0 to 529.8 U/L (0.17 - 8.83 µkat/L). The results are as follows : Correlation coefficient : (r) = 0.999 Linear regression: y = 0.927 x - 0.3 U/L (0.01 µkat/L)

- Limitations/Interferences

- AST can be underestimated in case of severe vitamin B6 deficiency. ⁽³⁾

- Hemolyzed samples should not be used since significant hemolysis may increase AST concentration because of high levels of AST in erythrocytes. ⁽³⁾

- Studies have been performed to determine the level of interference from different compounds. The following AST levels were tested: 35.0 U/L and 350.0 U/L.

No significant interference is defined by a recovery ±10% of the initial value.

Unconjugated Bilirubin : No significant interference up to 30.0 mg/dL (513 µmol/L).

Conjugated Bilirubin : No significant interference up to 29.5 mg/dL (504 µmol/L).

Triglycerides : No significant interference up to 2400 mg/dL (27.1 mmol/L).

Pyruvate : No significant interference up to 3.0 mg/dL (341 µmol/L).

Ascorbic acid : No significant interference up to 20.0 mg/dL.

Acetylsalicylic acid : No significant interference up to 200 mg/dL.

Acetaminophen : No significant interference up to 30.0 mg/dL.

- In very rare cases, monoclonal gammopathies (multiple myeloma), in particular IgM type (Waldenstrom's macroglobulinemia) can cause unreliable results. ⁽⁶⁾

- Many other substances and drugs may interfere. Some of them are listed in reviews published by Young. ^(7,8)

- On board stability/Calibration frequency
On Board Stability: 28 days
Calibration frequency: 28 days
Recalibrate when reagent lots change, when quality control results fall outside the established range and after a maintenance operation.

These performances have been obtained using ELITech Selectra ProM analyzer. Results may vary if a different instrument or a manual procedure is used. The performances of applications not validated by ELITech are not warranted and must be defined by the user.

DECLARATION OF SERIOUS INCIDENT

Please notify the manufacturer (through your distributor) and competent authority of the Member State of the european union in which the user and/or the patient is established, of any serious incident that has occurred in relation to the device. For other jurisdictions, the declaration of serious incident should be in accordance with local, state and federal regulatory requirements. By reporting a serious incident, you provide information that can contribute to the safety of *in vitro* medical devices.

TECHNICAL ASSISTANCE

Contact your local distributor or ELITech Clinical Systems SAS (ccsupport@elitechgroup.com).

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BIBLIOGRAFIE

1. Higgins, T., *et al.*, *Hemoglobin, Iron, and Bilirubin*, Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A., Ashwood, E.R. Bruns, D.E. (W.B. Saunders eds.), (2008), 509.
2. Dufour, D.R. *The liver: Function and Chemical Pathology*, Clinical Chemistry: Theory, Analysis, Correlation, 5th Ed., Kaplan, L.A, Pesce, A.J., (Mosby Inc. eds.), (2010), 586 and appendix.
3. Guder, W.G., *et al.*, *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2
4. Wu, A.H.B., Clinical guide to laboratory tests, 4th Ed., (W.B. Saunders eds.), (2006), 172.
5. Howard, C.R., *Breastmilk & Breastfeeding Jaundice*, Pediatric Clinical Advisor: Instant Diagnosis and Treatment, 2nd Ed., Garfunkel, L.C., Kaczorowski, J.M. & Christy, C., (Mosby Inc. eds.), (2007), 82.
6. Berth, M. & Delanghe, J. *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, Acta Clin Belg., (2004), **59**, 263.
7. Young, D.S., Effects of preanalytical variables on clinical laboratory tests, 2nd edition, AACC Press (1997).
8. Young D.S., Effects of drugs on clinical laboratory tests, 4th edition, AACC Press (1995).

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BIBLIOGRAFIE

1. Higgins, T., et al., *Hemoglobin, Iron, and Bilirubin*, Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A., Ashwood, E.R. Bruns, D.E. (W.B. Saunders eds.), (2008), 509.
2. Dufour, D.R. *The liver: Function and Chemical Pathology*, Clinical Chemistry: Theory, Analysis, Correlation, 5th Ed., Kaplan, L.A, Pesce, A.J., (Mosby Inc. eds.), (2010), 586 and appendix.
3. Guder, W.G., et al., *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2
4. Wu, A.H.B., Clinical guide to laboratory tests, 4th Ed., (W.B. Saunders eds.), (2006), 172.
5. Howard, C.R., *Breastmilk & Breastfeeding Jaundice*, Pediatric Clinical Advisor: Instant Diagnosis and Treatment, 2nd Ed., Garfunkel, L.C., Kaczorowski, J.M. & Christy, C., (Mosby Inc. eds.), (2007), 82.
6. Berth, M. & Delanghe, J. *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, Acta Clin Belg., (2004), **59**, 263.
7. Young, D.S., Effects of preanalytical variables on clinical laboratory tests, 2nd edition, AACC Press (1997).
8. Young D.S., Effects of drugs on clinical laboratory tests, 4th edition, AACC Press (1995).

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7. Berth, M. & Delanghe, J., *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature, Acta Clin Belg.*, (2004), **59**, 263.

8. Young, D.S., *Effects of preanalytical variables on clinical laboratory tests*, 2nd Ed., AACC Press, (1997).

9. Young, D.S., *Effects of drugs on clinical laboratory tests*, 4th Ed., AACC Press, (1995).

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Contactați distribuitorul dumneavoastră local sau ELITech Clinical Systems SAS (ccsupport@elitechgroup.com).