


**Information Display (During Study)**

- Acquisition monitor

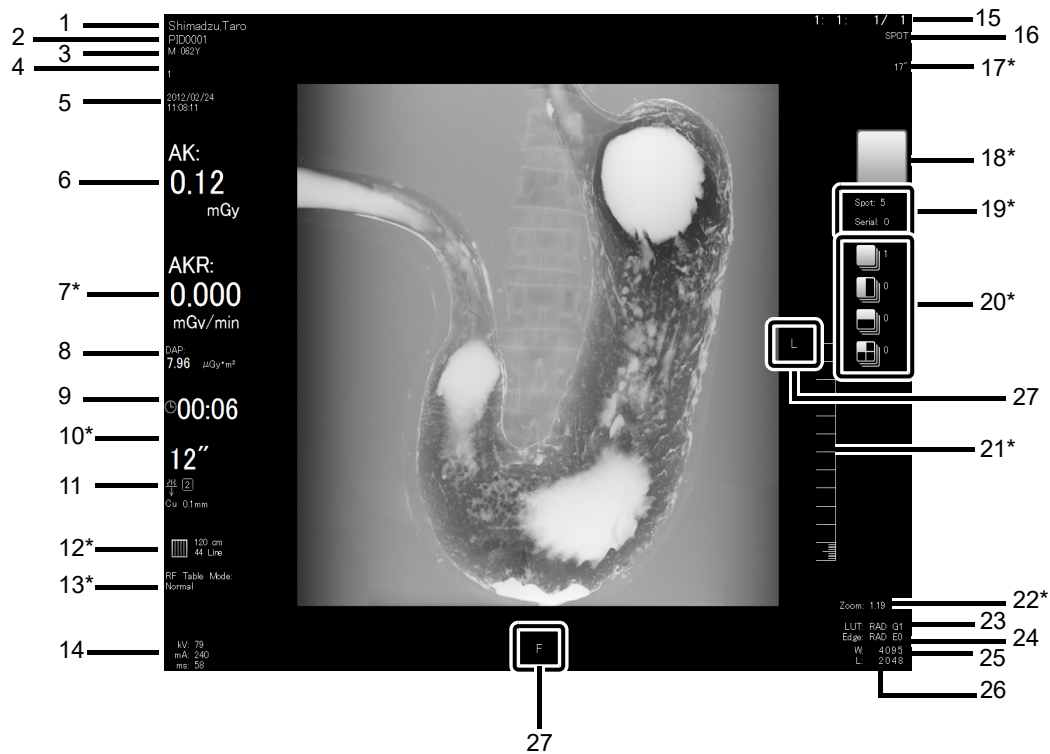

The display specification for items with "\*" changes depending on the detector type.

No.	Item	Description
1	Patient Name	The patient's name is displayed.
2	ID	The patient's ID is displayed.
3	Sex/Age	The patient's sex and age are displayed. M: Male; F: Female; O: Other.
4	Accession Number	The accession number is displayed.
5	Acquisition Date Acquisition Time	The acquisition date is displayed in the "year/month/day" format. The acquisition time is displayed in the "hour:minute:second" format.
6	Cumulative Reference Air Kerma	The cumulated reference air kerma <sup>1</sup> within the current study is displayed.
7*	Reference Air Kerma Rate	The reference air kerma rate <sup>1</sup> is displayed. It is displayed only during fluoroscopy, SERIAL, DSA (option) or RSM-DSA (option) radiography. * It is not displayed for the image acquired with the Wireless FPD (option).
8	DAP	The cumulative dose area product within the current study is displayed. It is not displayed during fluoroscopy, SERIAL, DSA (option) or RSM-DSA (option) radiography.
9	Total Fluoroscopy Time	The total fluoroscopy time is displayed within the current study.