


6.13 LOCAL PRINTER (OPTIONAL)

The equipment can be supplied with a digital, medical black and white printer. The printer is configured and installed by the equipment manufacturer.

Below you will find a list of available commands.

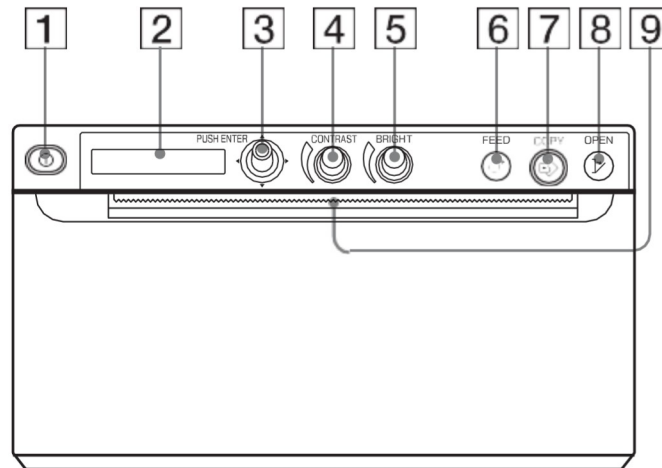

1. ON / OFF button.
2. Interface display.
3. MENU lever: use to navigate within the printer menu.
4. CONTRAST control: adjust the contrast of the printouts.
5. BRIGHT control: adjust the brightness of the printouts.
6. FEED button: use to feed with paper. While a print job is in progress, press to cancel the print job.
7. COPY button: Press to print another copy of the previous printout.
8. OPEN button: Press to open the front door panel of the printer.
9. Automatic paper cutter.

Note: To prevent printouts from fading or changing color, store them in a cool, dry location where the temperature is not higher than 30 °C (86 °F), where they are not exposed to direct sunlight and avoiding any contact with liquids (even volatile ones).