



# ARC 350 Touch-assisted universal system High-tech meets efficiency

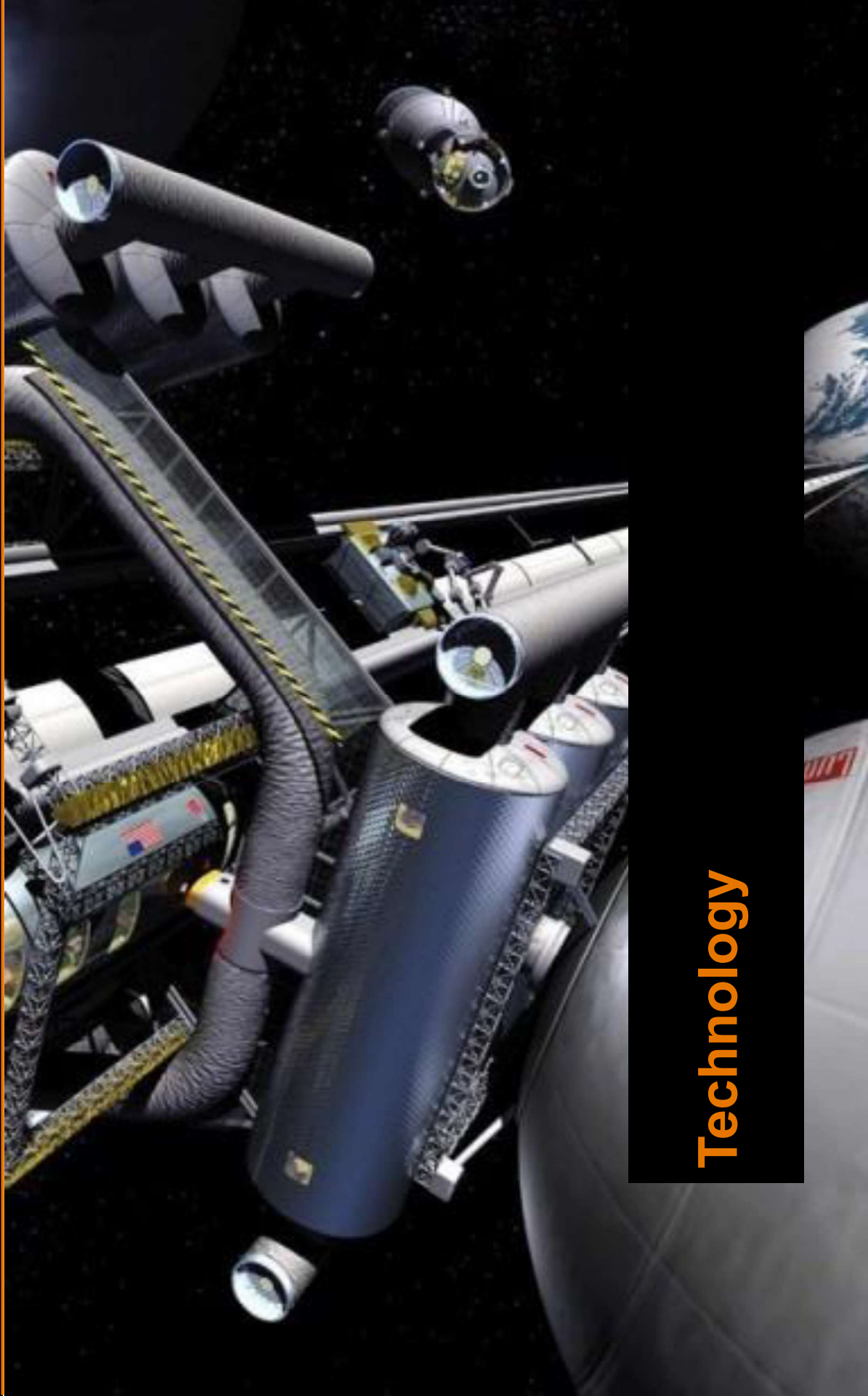

## Technology

# Capacitive glass touchscreen

3,5" TFT-Display  
4 LCD's

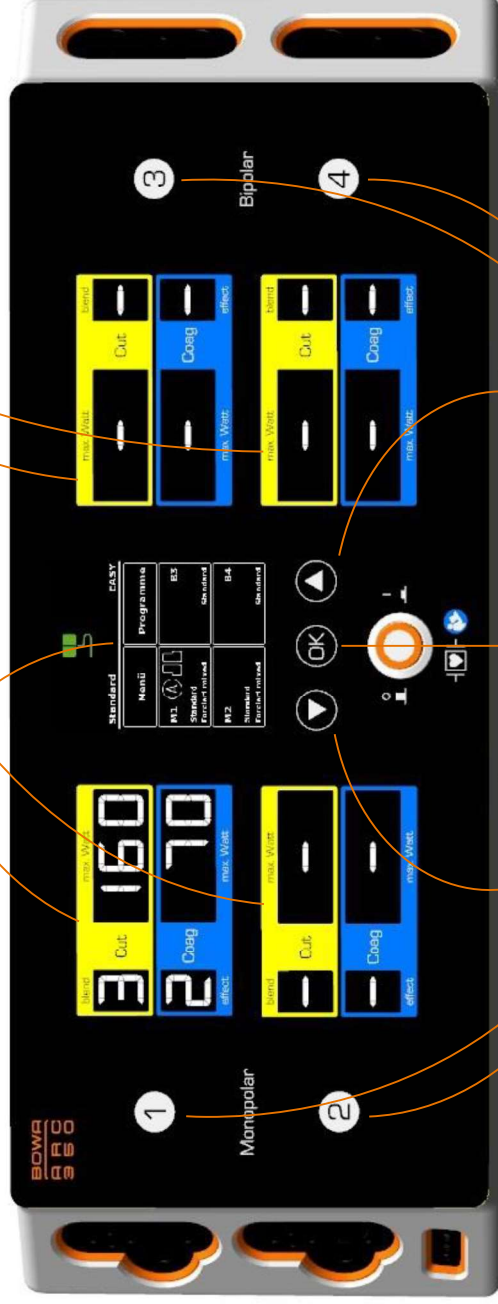

7 PCT-buttons  
Projective-Capacitive-Touchscreen





# Introduction


## Sockets

- Max. 4 sockets
- Monopolar:
  - 2 International/  
Erbe/Storz
- Bipolar:
  - 2 International/  
Erbe/Storz

Erbe configuration



# Configuration Sheet



**BOWA**  
EINFACH SICHER

REF 900-351

**CONFIGURATION ARC 350**

Complets overview of configurations


<p><b>Voltage range (± 10%)</b></p> <input type="checkbox"/> 220-240V <input type="checkbox"/> 100-120V	<p><b>Mains cable 5 m type</b></p> <input type="checkbox"/> F <input type="checkbox"/> G <input type="checkbox"/> L <input type="checkbox"/> B <input type="checkbox"/> N
<p><b>Equipotential bonding cable (extra charge)</b></p> <input type="checkbox"/> without cable <input type="checkbox"/> 0,5 m <input type="checkbox"/> 1,5 m <input type="checkbox"/> 5 m	<p><b>Socket configuration monopolar</b></p> <input type="checkbox"/> Version 1: Erbe 5mm <input type="checkbox"/> Version 2: Bovie
<p><b>Service Manual</b></p> <input type="checkbox"/> DE <input type="checkbox"/> EN	<p><b>Operating Manual</b></p> <input type="checkbox"/> DE <input type="checkbox"/> EN <input type="checkbox"/> FR <input type="checkbox"/> ES <input type="checkbox"/> IT <input type="checkbox"/> PL <input type="checkbox"/> RU <input type="checkbox"/> TR <input type="checkbox"/> PT <input type="checkbox"/> HU <input type="checkbox"/> ZH
<p><b>Options ARC 350 (extra charge)</b></p> <input type="checkbox"/> REF 900-395 <b>Bipolar Resection/COMFORT</b> Special current for bipolar resectoscope/hysteroscopes	<p><b>Options ARC 350 (extra charge)</b></p> <input type="checkbox"/> REF 900-396 <b>LIGATION/COMFORT</b> Bipolar tissue and vessel sealing
<p>FEL688            Vers. 01-EN            Dat.: 2018-05-23</p>	<p>BOWA-elektronik GmbH &amp; Co. KG            Heinrich-Hertz-Strasse 4-10            72810 Gomaringen   Germany            Phone: +49 (0) 7172-6002-0            Fax: +49 (0) 7172-6002-33            info@bowa.de   bowa-medical.com</p>

Individual socket configuration  
-International/Bovie  
-Erbe/Storz


COMFORT  
only with option

# Sockets

## MONOPOLAR

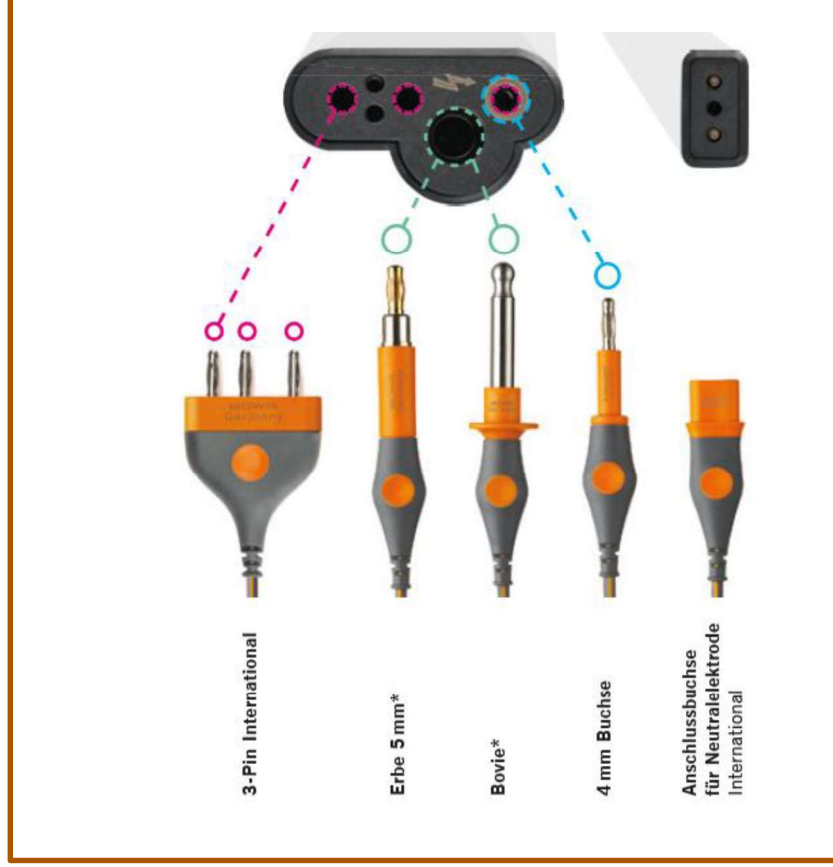

## BIPOLAR

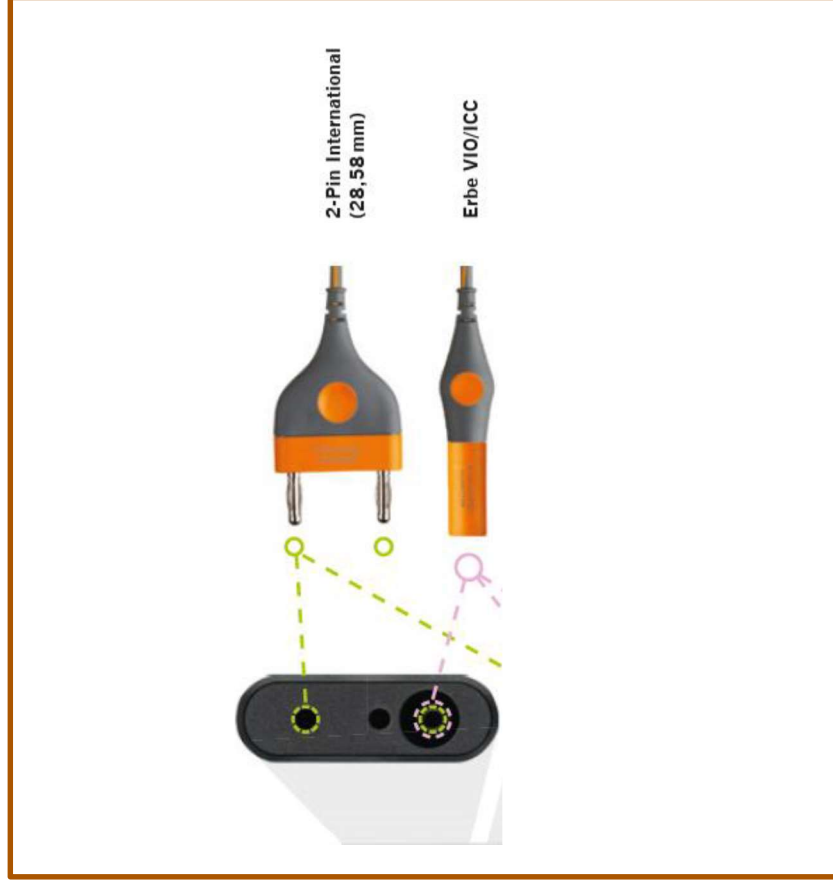



## Interfaces

- Updates via USB
- Connections:
  - Footswitches
  - Light wires
  - Equipotential bonding
  - Audio (IN)
  - CAN interface





- System integration
- System analysis
- ➞ Future and investment safe

## Differences compared to ARC 400

- Different touch displays
- Sockets (max. 4) and configuration
- GastroCut option, RFID<sup>light</sup> incl. with every option
- No bipolar SimCoag available
- No Dr. Dongle available
- No favorites
- No mode icons with preview
- Short instruction animation
- No training video incl.

## Short instruction animation



- After startup

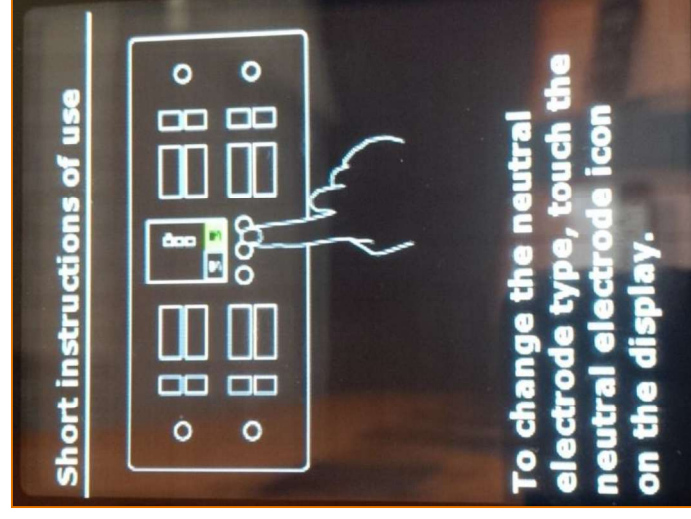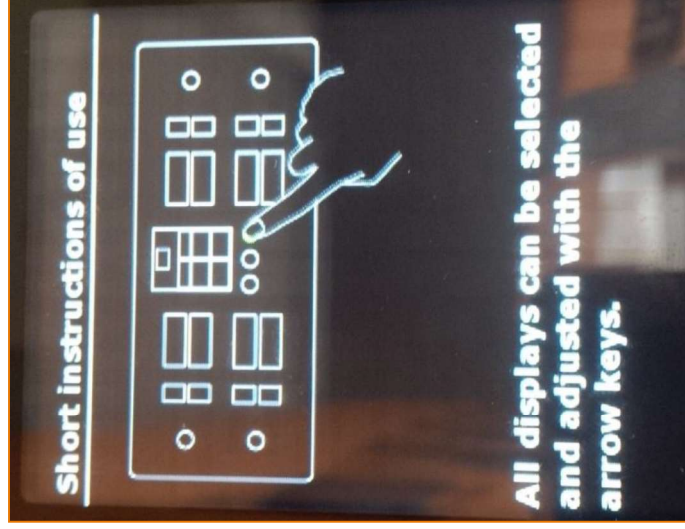

- Skip with tapping a button outside the display




## Power and effect settings

- LCD indicators directly selectable
- Fast setup holding the arrow keys

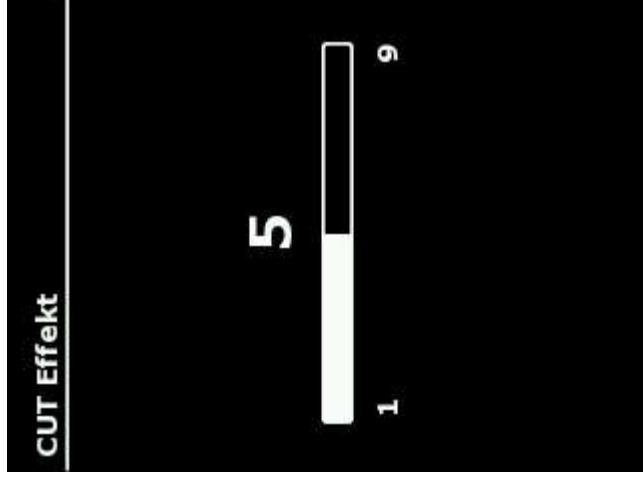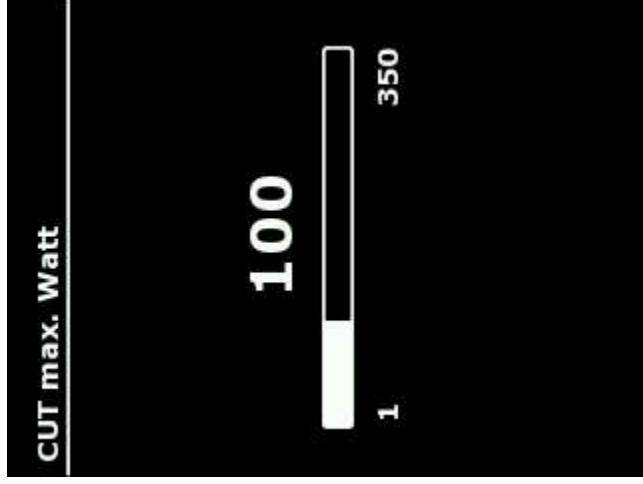

## 350 program spaces

- Selected within the menu
- Currently loaded program with orange arrow
- Sort programs i.a. by favorites, alphabet, ...

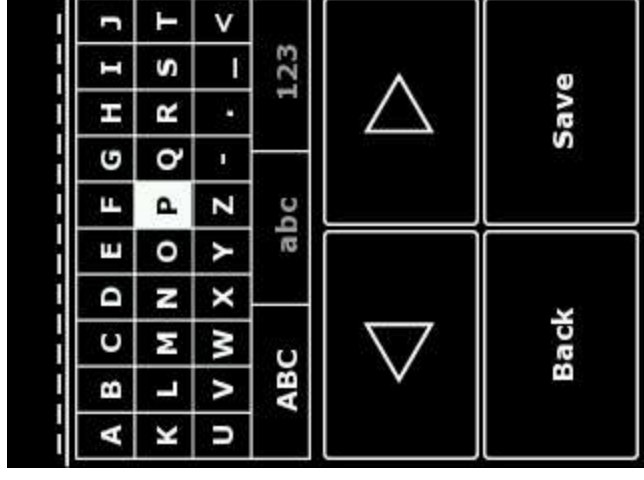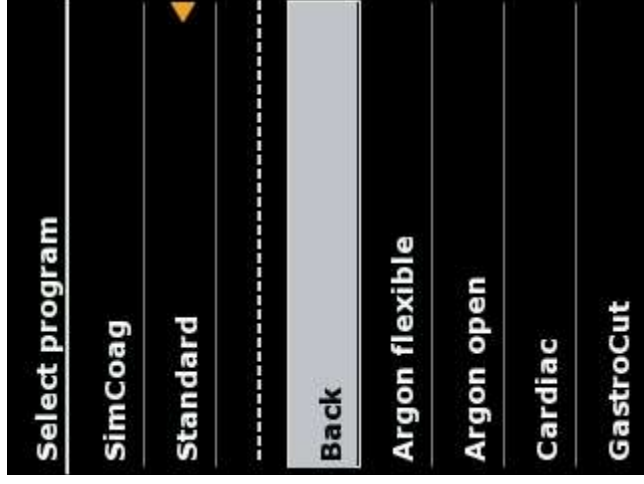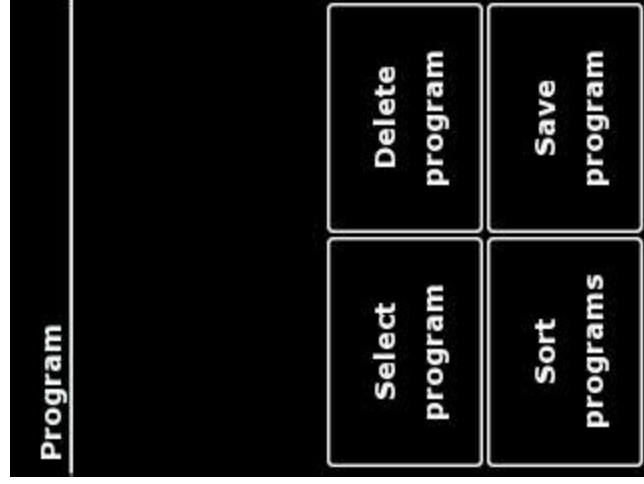



## Footswitch assignment

- Attribution to the whole socket
- 2 Footswitches with button
  - REF 901-032, DP+
  - REF 901-031, DP
  - REF 901-011, EP
- deactivatable



## Mode selection

- Socket menu for selection
- Indication of active mode with orange arrow

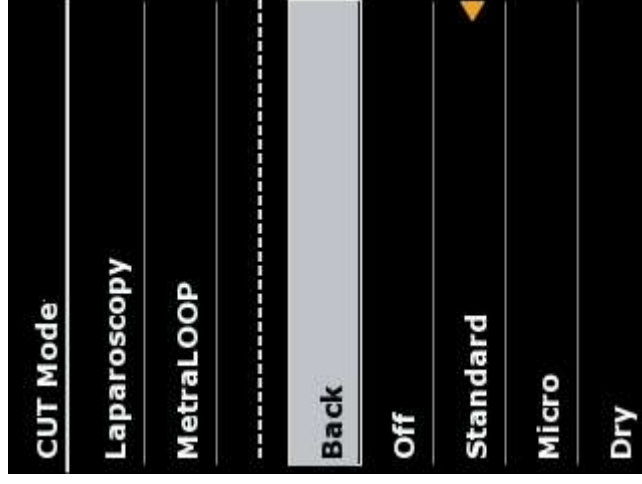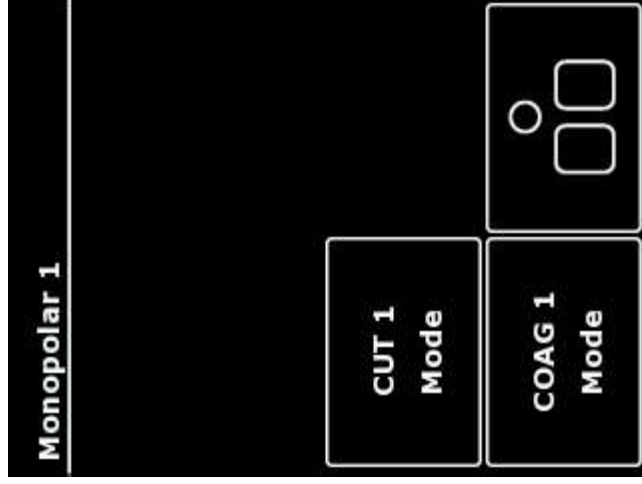

## Plug'n Cut & socket illumination

- Connection with detection of standard accessories
  - Protection against false activation without cable
  - Activation of the input fields of the respective socket
- ➔ Interactive input assistance



## Neutral electrode

- EASY BABY MONO
- Contact indicator with colour indicator

