

MT 01 Foam Mattress – User & Cleaning Manual

Manufacturer: Schröder Medikal Ticaret A.Ş.

Model: MT-01 Foam Mattress

Intended Use: Hospital examination rooms, emergency rooms, and general patient care environments.

1. Product Description

The MT-01 Foam Mattress is a one-piece, CNC-cut foam mattress designed to reduce pressure points and enhance patient comfort. It features integrated air passages for ventilation and heat dispersion, and is covered with a waterproof, breathable, and antibacterial PU-coated polyester fabric.

2. Technical Specifications

Parameter	Value
Dimensions	85 x 195 cm
Thickness	12cm standard thickness, thickness can be customised under customer request
Foam Density	28 kg/m ³
Material	Grey/White PU Foam
Cover Fabric	100% Polyester + PU coating
Properties	Breathable, Antiallergic, Fire-resistant, Liquid-proof
Closure	Zipper with protective flap
Cleaning Compatibility	Autoclavable, washable at $\geq 70^{\circ}\text{C}$, dryer-safe

3. Cleaning & Disinfection Instructions

- Clean the mattress surface using a **soft, damp cloth** with a **0.5% sodium hypochlorite solution** or hospital-grade disinfectant.
- Allow disinfectant to remain for **5–10 minutes**, then wipe with clean water and dry with individual cloths.
- The **removable cover** can be machine-washed at **up to 70°C** and dried in a dryer.
- Avoid prolonged exposure to liquids or excessive moisture.
- Do not use high-pressure hoses, steam cleaners, or water above 70°C.

4. Storage Conditions

- Store in a **dry, ventilated area**, away from direct sunlight and heat sources.
- Do not fold or compress the mattress during storage.
- Avoid placing heavy objects on top of the mattress.

5. Safety Precautions

- Ensure the mattress is used on compatible bed frames.
- Do not expose to open flames or explosive environments.
- Do not use if the cover is damaged or unzipped.

6. Warranty & Support

2 years warranty. Please contact with us for warranty terms and technical support.

SCHRÖDER MEDİKAL TİC. A.Ş.

Tel. 0850 202 77 50 • e-mail: finans@schroderhealth.de

Başpınar (Organize) OSB Mahallesi O.S.B. 5. Bölge

83548 Nolu Cad. No:6/1 SEHİTKAMİL/GAZİANTEP

Ticaret Sicil No: 69216

Sehirkamil V.D. 757 098 8720

SCHRÖDER HEALTH PROJECTS

Schröder Medikal Ticaret A.Ş.

BASPINAR MAH. 5. O.S.B 83548 NOLU CAD NO:6/1/0 SEHİTKAMİL GAZİANTEP

Ph : +90 850 202 77 50 Fax: +90 342 230 15 83 www.schroder.com.tr info@schroder.com.tr

Factory : BASPINAR (ORGANIZE) OSB MAH. OSB 5.BOLGE 83548 NO CAD. NO:6 SEHİTKAMİL GAZİANTEP TURKEY

