

ALKALINE CLEANING SOLUTION FOR AUTOMATIC INSTRUMENT WASHER MACHINES

DETRO MATIC A – USER MANUAL

- **ALKALINE CLEANING PRODUCT**
- **CONCENTRATE**

Product Properties

Compatible For All Washer Machines

Compatible For All Water Hardness

Cleaning and Disinfection of Heat-Resistant Instruments at 93 °C

Cleaning of Thermolabile Instruments at 40-60 °C

Protection Against Corrosion

Low Usage Concentration

Powerful Effect

Usage Areas

Alkaline cleaning of instruments made of metal, glass, ceramic, porcelain, plastic materials in automatic washing machines.

Ingredient

>5% Potassium Hydroxide (CAS no: 1310-58-3, surfactants, silicate, complexing agents, corrosion inhibitors)

Dosing

It is used at %0,1-0,2 concentration. (10-20 ml product into per 10 lt water). This level can be increased according to contamination level.

Warning

Read usage instruction before usage. Don't use for cleaning of hand, face and food. Keep out of reach children. Irritant agent. Don't contact with skin and mucous membranes. In the case of eye contact, wash plenty of water immediately.

Storage Conditions

Keep away from direct sunlight. Store containers tightly closed and in a well ventilated area at 0-30 °C

Package

5 L

DETRO HEALTHCARE KİMYA SANAYİ A.Ş.

☎ 212 659 77 62 📠 212 659 77 63

✉ info@detrox.com.tr – www.detrox.com.tr