SPECIAL INSTRUCTIONS

BAUSCH & LOMB
PACKAGING/GRAPHIC ENG.
UNIT 424/425 INDUSTRIAL
PARK, WATERFORD. IRELAND
TEL: 353 51 859791

THE INFORMATION LISTED BELOW MUST APPEAR ON ALL VENDOR PROOFS (COLORKEYS, MATCHPRINTS/CHROMALINS) OR THEY WILL BE REJECTED

PLACED PICTURES:

AW # 4032701 DESCRIPTION 1300 mPa.s Silicone Oil / 5700 mPa.s Silicone Oil DFU

NUMBER OF PRINTING PLATES 1

TOTAL NUMBER OF PMS COLORS 1

PAGE 1 OF 1

PMS COLORS

1. Pantone Process Black C

Bausch & Lomb		
Oxane®1300/5700 Silicone Oil		
EN Product Information		
CURVE OF SPECTRAL TRANSMITTANCE		
110		
COMPOSITION 1ml contains highly pure sterile silicone oil, 1000-1500mPas or 5000-5900mPas. %T —		
Refractive index = 1.40.		
300 400 600 800 1000 1100		
Wavelength [nm]		
with massive proliferative vitreo-retinopathy, traumatic detachment, giant tears as well as other retinal detachments, known STORAGE INSTRUCTIONS: store Silicone Oil below		
use of silicone oils in cases of central foramina with detachment or in cases of severe diabetic retinopathies is at presultand outer surface simultaneously in a steam ster		
ent under discussion. CONTRAINDICATIONS: Intraocular lenses made from silicone can be opacified by silicone oil. Therefore, silicone oil should not be applied in eyes with library lenses made from silicone oil should not be applied in eyes with library lenses made from silicone oil should not be applied in eyes with		
METHOD OF USAGE AND DOSAGE: Oxane® is injected intra- ocularly in a dose to be determined by the ophthalmic sur-		
geon. The selection of patients, the choice of surgical procedure and the length of the indwelling period of the sill-cone oil in the eye are the sole responsibility of the ophthalmic surgical procedures and the sill-cone oil in the eye are the sole responsibility of the ophthalmic surgical procedures and the ophthalmic surgical procedures and the ophthalmic surgical procedures and the ophthalmic surgical procedures are the sole responsibility of the ophthalmic surgical procedures are the ophthalmic surgical procedures and the ophthalmic surgical procedures are the ophthalmic surgical procedures and the ophthalmic surgical procedures are the ophthalmic surgical procedures are the ophthalmic surgical procedures and the ophthalmic surgical procedures are the ophthalmic surgical procedures		
Do not reuse; do not resterilise; do not use if sterile barrier is breached; verify the expiry date; do not overfill the post-		
erior segment with Oxane®; Oxane® is free from preservatives, do not use for more than one patient: left-over fluid must be disposed. WARNINGS: Heavy liquids like all types		
of fluorocarbon liquids have to be removed completely from the vitreous cavity before silicone oil can be filled into the eye. Following the use of silicone oils that were not highly		
purified, the occurrence of corneal decompensation, cataract, secondary glaucoma and emulsification has been reported. Although the casual relationship with the purity of the silicone oil has not yet been established, the incidence		
of these complications appears to be lower following the use of highly pure silicone oil. The duration of the indwell-		
ing period of the silicone oil in the eye must be determined by the treating physician. Some reports indicate a period of use of more than two years. It has been shown that on removal of the silicone oil in the post-post-post-pix of the silicone oil in the post-post-post-pix of the silicone oil in the eye must be determined by the reading physician. Some reports indicate a period of use of more than two years. It has been shown that on the post-post-pix of the silicone oil in the eye must be determined by the reading physician. Some reports indicate a period of use of more than two years. It has been shown that on the post-post-post-pix of the silicone oil in the eye must be determined by the reading physician. Some reports indicate a period or use of use of more than two years. It has been shown that on the post-post-post-pix of the post-post-pix of the post-post-post-pix of the post-post-pix of the post-post-post-pix of the post-post-pix of the post-post-post-pix of the post-post-post-pix of the post-post-post-pix of the post-post-post-pix of the post-post-pix of the post-post-post-pix of the post-post-post-pix of the post-post-pix of the post-post-post-pix of the post-post-post-pix of the post-post-pix of the post-post-post-pix of the post-post-pix of the post-post-post-pix of the post-post-post-pix of the post-post-post-pix of the post-post-pix of the post-post-post-pix of the post-post-pix of the post-post-post-pix of t		
removal of the silicone oil in the post-operative phase, dur- ing which the visual acuity neither increases nor further 4032701		

	Artwork History		
4032701	Previous artwork revision updated for rebranding and new artwork number assigned	PD	04th Apr '06
4032701 @ A	Updated to replace Mpas with mPa.s on Lionel's request		25th May '06
4032701 @ B	Updated to remove Mpas in front of word silicone oil throughout document on Lionel request. Also change precautions to correct wording from ISO 15798 (do not use if sterile barrier is breached) & include new symbol.	PD	27th Jun '06
4032701 @ C	Updated Brand sub brand & descriptor to identity standards & adapted to new keyline		11th Oct '06
4032701 @ D	Address noted as Rochester address and Waterford as Manufacturer Updated TM/R mark sentence to change requested from legal trademark to include or its affiliates	PD	21/12/2006

Viewed/Printed: 05 Oct 2015 **GDMS Copy-Use per procedure** 4032701 Rev A

BAUSCH & LOMB CONTROLLED DOCUMENT SIGNATURE PAGE

Document Name: 4032701

Document Title: Oxane 1300/5700 DFU

Date (GMT)	Signed by
6/25/2007 11:36:57	Bowe David P
For Department	
Justification	Author Confirmation
7/3/2007 22:45:49	Izzo Christopher
For Department	
Justification	Quality
7/9/2007 07:56:44	Jones Janet
For Department	
Justification	DocControl Release
For Department	
Justification	
For Department	
Justification	
For Department	
Justification	
For Department	
Justification	

B&L CONTROLLED DOCUMENT Status: Effective Effective Date: 10/Jul/2007

BAUSCH & LOMB CONTROLLED DOCUMENT SIGNATURE PAGE

For Department	
Justification	
For Department	
Justification	
For Department	
Justification	
For Department	
For Department	
For Department	
For Department	
For Department	