

# Medical Laminated Protective Overall TYPE 3B/4B/5B/6B

## Product description:

Medical laminate protective overall, disposable, made of laminated spunbond, with full-length protective tape, density 60 gr./m<sup>2</sup>. It has hood with elastic and sewn chin flap fastened with adhesive tape; zipper with a protective flap and adhesive tape; booties - integral part of the overall, additionally a pair of protective plastic shoes made of spunbonded material; a sleeve fastened with a finger loop; elastic at the hood, sleeves, ankles and waist. Provides an effective barrier against fluids (waterproof).

## Instructions for use:


The equipment is worn over the work coat. The zipper opens, and the coverall is dressed starting with the feet, then the hands are inserted into the sleeves of the coverall, the elastic ends are arranged correctly, the hood is positioned on the head and, at the end, the zipper is closed and the protective flap is fixed with adhesive tape.

## Undressing instructions:

The coverall will be undressed in the designated area, away from pollution and dangers; the help of another person is recommended, avoiding contamination of skin and clothing. Carefully remove the self-adhesive flap for the zipper, unzip the coverall and remove the hood making sure the gloves do not touch the inside of the coverall; then the coverall is totally removed. The gloves should be removed according to the manufacturer's instructions and with safety procedures. Dispose of the coverall in a suitable waste container.

## Storage instructions:

Storage is done in the original packaging. Store at a temperature between 0 ° C and + 20 ° C; in a dry and clean place, away from moisture; away from direct sunlight, any flame, water droplets and solvent vapors.

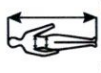

EN 13688

EN 14605



TYPE 3B

EN 14605



TYPE 4B

ISO 13982-1



TYPE 5B

EN 13034




TYPE 6B

EN 14126



2841

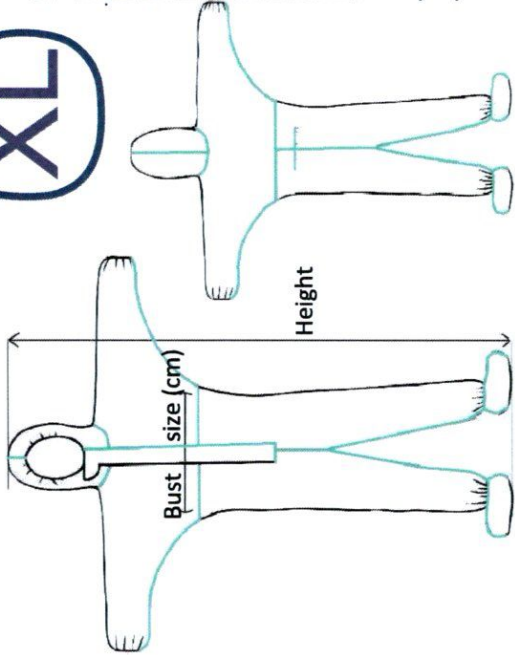

Registration number:

Size	Height (cm)	Bust size (cm)
L	170-176	132-136
XL	176-182	136-140
2XL	182-188	140-144
3XL	188-194	144-148



Do not iron



Do not bleach



Do not wash




Do not dry clean



Do not Tumble dry



09.2021

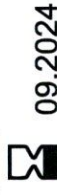

09.2024

Manufacturer:



YFS GIDA TEMİZLİK AMBALAJ MEDİKAL TEKSTİL

SANAYİ TİCARET LİMİTED ŞİRKETİ

Adress: BAĞCILAR MAH.1084.SOK.ENDER 6 APT 1/1

BAĞLAR/DIYARBAKIR

Tel: +90 412 251 8526

YFS GIDA TEMİZLİK AMBALAJ MEDİKAL TEKSTİL SAN. TİC. LİM. ŞİRK. BAĞLAR/DIYARBAKIR MAH. BAĞCILAR MAH. 1084. SOK. ENDER 6 APT 1/1 T.C. İÇİŞİLERİ BAKANLIĞI GIDA VE İLAHİYAT GENEL MÜDÜRLÜĞÜ GIDA VE İLAHİYAT GENEL MÜDÜRLÜĞÜ GIDA VE İLAHİYAT GENEL MÜDÜRLÜĞÜ GIDA VE İLAHİYAT GENEL MÜDÜRLÜĞÜ

# Medical Laminated Protective Overall TYPE 3B/4B/5B/6B

## Product description:

Medical laminate protective overall, disposable, made of laminated spunbond, with full-length protective tape, density 60 gr./m2. It has hood with elastic and sewn chin flap fastened with adhesive tape; zipper with a protective flap and adhesive tape; booties - integral part of the overall, additionally a pair of protective plastic shoes made of spunbonded material; a sleeve fastened with a finger loop; elastic at the hood, sleeves, ankles and waist. Provides an effective barrier against fluids (waterproof).

## Instructions for use:


The equipment is worn over the work coat. The zipper opens, and the overall is dressed starting with the feet, then the hands are inserted into the sleeves of the overall, the elastic ends are arranged correctly, the hood is positioned on the head and, at the end, the zipper is closed and the protective flap is fixed with adhesive tape.

## Undressing instructions:

The overall will be undressed in the designated area, away from pollution and dangers; the help of another person is recommended, avoiding contamination of skin and clothing. Carefully remove the self-adhesive flap for the zipper, unzip the overall and remove the hood making sure the gloves do not touch the inside of the overall; then the overall is totally removed. The gloves should be removed according to the manufacturer's instructions and with safety procedures. Dispose of the overall in a suitable waste container.

## Storage instructions:

Storage is done in the original packaging. Store at a temperature between 0 ° C and + 20 ° C; in a dry and clean place, away from moisture; away from direct sunlight, any flame, water droplets and solvent vapors.

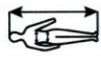

EN 14605



TYPE 3B

EN 14605



TYPE 4B

ISO 13982-1



TYPE 5B

EN 13034



TYPE 6B

EN 14126




CE

2841

MADE IN Turkey  
Discover the potential

Model  
YFS-1

L

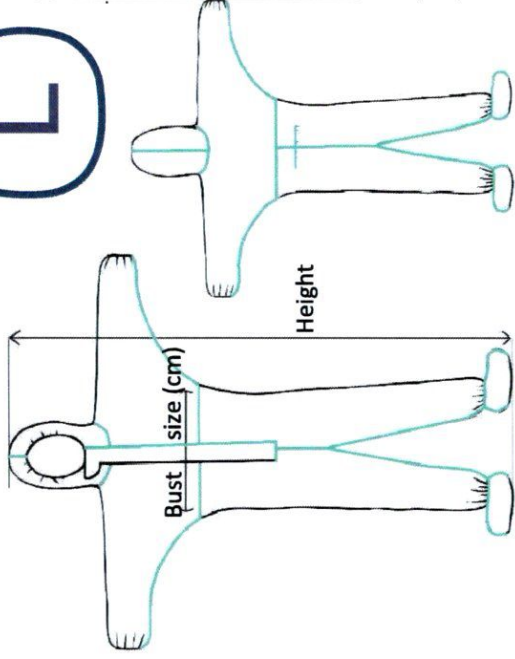

Registration number:

Size	Height (cm)	Bust size (cm)
L	170-176	132-136
XL	176-182	136-140
2XL	182-188	140-144
3XL	188-194	144-148



Do not bleach



Do not wash



Do not dry clean



Do not Tumble dry



09.2021



09.2024

Manufacturer:



YFS GIDA TEMİZLİK AMBALAJ MEDİKAL TEKSTİL

SANAYİ TİCARET LİMİTED ŞİRKETİ

Address: BAĞCILAR MAH.1084.SOK.ENDER 6 APT 1/1

BAĞLAR/DIYARBAKIR

Tel: +90 412 251 8526

Y.F.S. GIDA TEMİZLİK AMBALAJ MEDİKAL TEKSTİL SAN. TİC. LTD. ŞTİ.  
TEK. ÇİFT. ZARFI  
Büyükdere Mah. 1084. Sok. Ender 6 Apt 1/1  
Bağlar/Diyarbakir

# Medical Laminated Protective Overall TYPE 3B/4B/5B/6B

## Product description:

Medical laminate protective overall, disposable, made of laminated spunbond, with full-length protective tape, density 60 gr./m2. It has hood with elastic and sewn chin flap fastened with adhesive tape; zipper with a protective flap and adhesive tape; booties - integral part of the overall, additionally a pair of protective plastic shoes made of spunbonded material; a sleeve fastened with a finger loop; elastic at the hood, sleeves, ankles and waist. Provides an effective barrier against fluids (waterproof).

## Instructions for use:

The equipment is worn over the work coat. The zipper opens, and the overall is dressed starting with the feet, then the hands are inserted into the sleeves of the overall, the elastic ends are arranged correctly, the hood is positioned on the head and, at the end, the zipper is closed and the protective flap is fixed with adhesive tape.

## Undressing instructions:

The overall will be undressed in the designated area, away from pollution and dangers; the help of another person is recommended, avoiding contamination of skin and clothing. Carefully remove the self-adhesive flap for the zipper, unzip the overall and remove the hood making sure the gloves do not touch the inside of the overall; then the overall is totally removed. The gloves should be removed according to the manufacturer's instructions and with safety procedures. Dispose of the overall in a suitable waste container.

## Storage instructions:

Storage is done in the original packaging. Store at a temperature between 0 ° C and + 20 ° C; in a dry and clean place, away from moisture; away from direct sunlight, any flame, water droplets and solvent vapors.




EN 13688




EN 14605

TYPE 3B

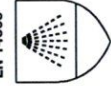

EN 14605

TYPE 4B

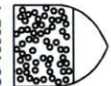

ISO 13982-1

TYPE 5B



EN 13034

TYPE 6B


EN 14126



2841



2XL

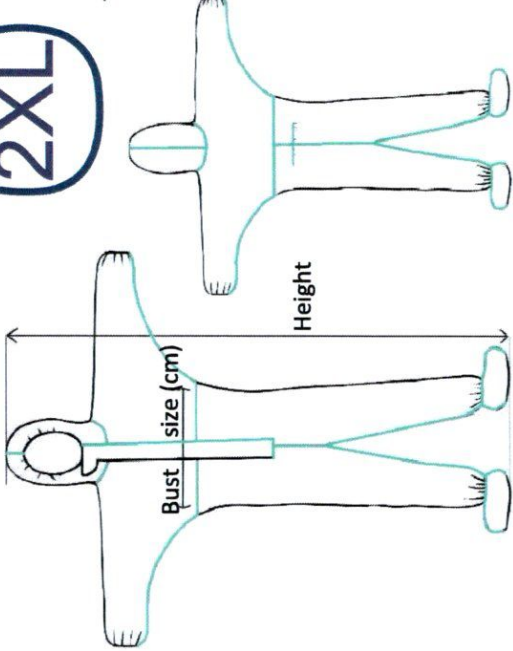

Registration number:

Size	Height (cm)	Bust size (cm)
L	170-176	132-136
XL	176-182	136-140
2XL	182-188	140-144
3XL	188-194	144-148



09.2021



09.2024

## Manufacturer:



YFS GIDA, TEMİZLİK, AMBALAJ, MEDİKAL, TEKSTİL  
SANAYİ TİCARET LİMİTED ŞİRKETİ  
Adres: BAĞCILAR MAH.1084.SOK.ENDER 6 APT 1/1  
BAĞLAR/DIYARBAKIR  
Tel: +90 412 251 8526

YFS GIDA, TEMİZLİK, AMBALAJ, MEDİKAL, TEKSTİL, SAN. TİC. LTD. ŞTİ.  
Tic. Sic. No: 272200, Şişli, İstanbul, Türkiye  
Gıda, Temizlik, Ambalaj, Medikal, Tekstil  
Gıda, Temizlik, Ambalaj, Medikal, Tekstil

# Medical Laminated Protective Overall TYPE 3B/4B/5B/6B

## Product description:

Medical laminate protective overall, disposable, made of laminated spunbond, with full-length protective tape, density 60 gr./m2. It has hood with elastic and sewn chin flap fastened with adhesive tape; zipper with a protective flap and adhesive tape; booties - integral part of the overall, additionally a pair of protective plastic shoes made of spunbonded material; a sleeve fastened with a finger loop; elastic at the hood, sleeves, ankles and waist. Provides an effective barrier against fluids (waterproof).

## Instructions for use:


The equipment is worn over the work coat. The zipper opens, and the coverall is dressed starting with the feet, then the hands are inserted into the sleeves of the coverall, the elastic ends are arranged correctly, the hood is positioned on the head and, at the end, the zipper is closed and the protective flap is fixed with adhesive tape.

## Undressing instructions:

The coverall will be undressed in the designated area, away from pollution and dangers; the help of another person is recommended, avoiding contamination of skin and clothing. Carefully remove the self-adhesive flap for the zipper, unzip the coverall and remove the hood making sure the gloves do not touch the inside of the coverall; then the coverall is totally removed. The gloves should be removed according to the manufacturer's instructions and with safety procedures. Dispose of the coverall in a suitable waste container.

## Storage instructions:

Storage is done in the original packaging. Store at a temperature between 0 ° C and + 20 ° C; in a dry and clean place, away from moisture; away from direct sunlight, any flame, water droplets and solvent vapors.

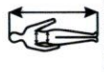

EN 14605



TYPE 3B

EN 14605



TYPE 4B

ISO 13982-1



TYPE 5B

EN 13034



TYPE 6B

EN 14126




2841

MADE IN Turkey  
Discover the potential

Model  
YFS-1

3XL

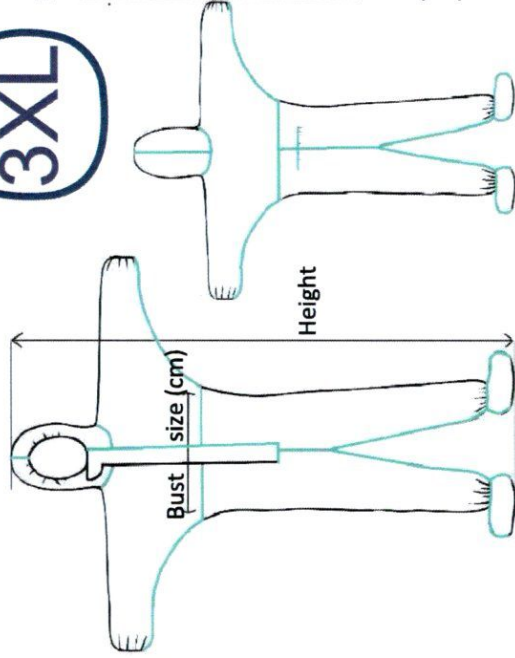

Registration number:

Size	Height (cm)	Bust size (cm)
L	170-176	132-136
XL	176-182	136-140
2XL	182-188	140-144
3XL	188-194	144-148



Do not bleach



Do not wash



Do not dry clean



Do not Tumble dry



09.2021



09.2024

Manufacturer:



YFS GIDA TEMİZLİK AMBALAJ MEDİKAL TEKSTİL  
SANAYİ TİCARET LİMİTED ŞİRKETİ  
Address: BAĞCILAR MAH.1084.SOK ENDER 6 APT 1/1  
BAĞLAR/DIYARBAKIR  
Tel: +90 412 251 8526

YFS GIDA TEMİZLİK AMBALAJ MEDİKAL TEKSTİL SANAYİ TİCARET LİMİTED ŞİRKETİ  
Address: BAĞCILAR MAH.1084.SOK ENDER 6 APT 1/1  
BAĞLAR/DIYARBAKIR  
Tel: +90 412 251 8526