

2.2 Operation Panel

The function of each button found on the operation panel is explained. The buttons on the rear operation panel located on the back surface of the equipment perform the identical functions.

Their functions are explained as per the standard arrangement. If the arrangement is changed, read the explanation with respect to that new arrangement.

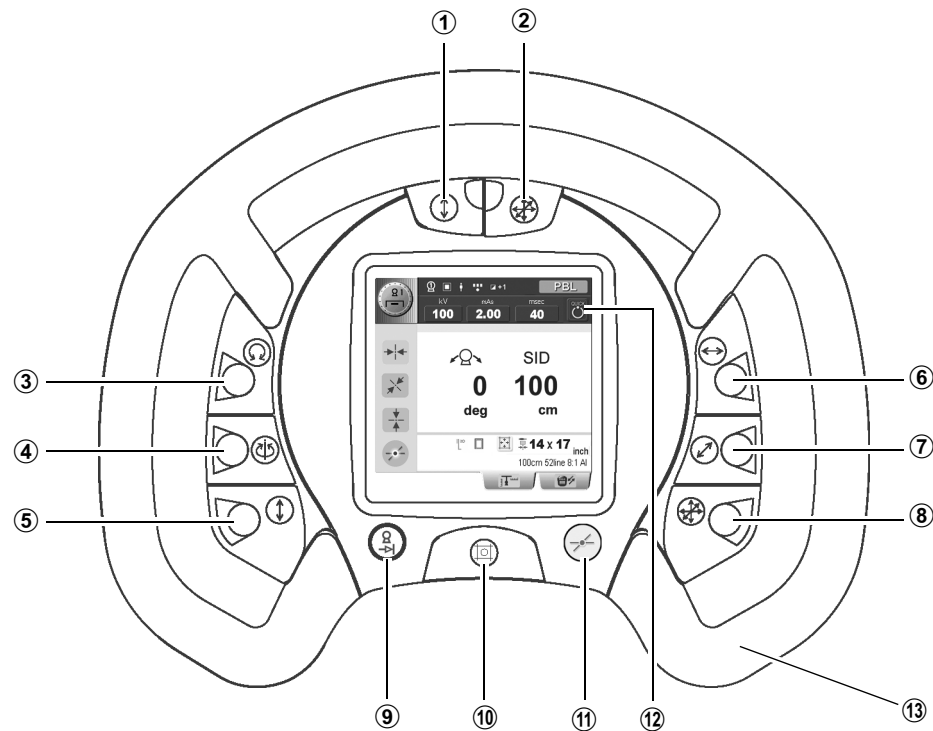


Fig. 2.3 Operation Panel

① (Vertical release) button

This button releases the lock when moving the X-ray tube support vertically.

② (3-directional release) button

This button simultaneously releases the locks for the vertical, longitudinal and transversal directions.

③ (H-axis rotation release) button

This button is to release the lock when rotating the X-ray tube support around the horizontal axis.

④ (V-axis rotation release) button

This button is to release the lock when rotating the X-ray tube support around the vertical axis.