










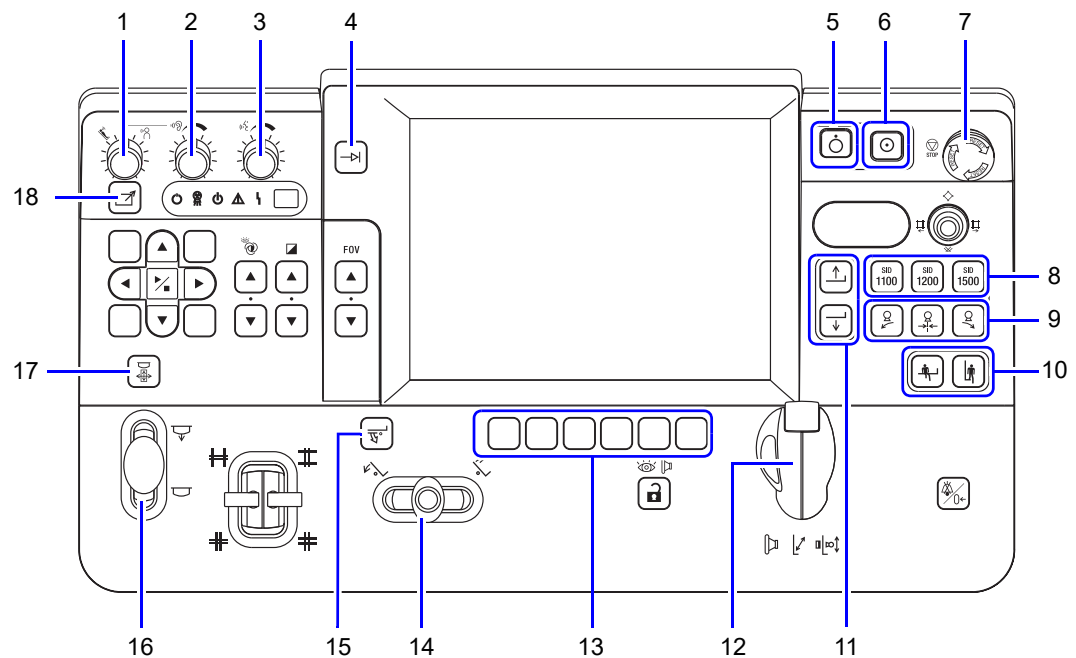

Controls that Operate the X-ray Diagnostic Table



No.	Form	Name	Function
1		Balance adjustment dial	Adjusts the mixing balance between the microphone installed in the tabletop and the auxiliary microphone. Mixing is increased for the tabletop microphone when the dial is turned towards  , and increased for the auxiliary microphone when the dial is turned towards  .  Reference P.4-83
2		Control room speaker volume adjustment dial	Adjusts the volume of the control room speaker. Turning it to the right increases the volume, and turning it to the left lowers the volume.  Reference P.4-83
3		Examination room speaker volume adjustment dial	Adjusts the volume of the examination room speaker. Turning it to the right increases the volume, and turning it to the left lowers the volume.  Reference P.4-83
4		Set button *1	Used with the position memory function.  Reference P.4-66 Moves the X-ray tube and FPD to the radiography start point in the SLOT radiography (option).  Reference P.4-181
5		Power OFF button	Turns the system's power off.  Reference P.4-40
6		Power ON button	Turns the system's power on.  Reference P.4-39
7		Emergency stop button	Brings the movement of the X-ray diagnostic table to an emergency stop.  Reference P.5-2