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EC Declaration of Conformity

In accordance with Directive 98/79/EC

Legal Manufacturer: *Zhejiang Orient Gene Biotech Co., Ltd*

Legal Manufacturer Address: *3787#, East Yangguang Avenue, Dipu Street,
Anji 313300, Huzhou, Zhejiang, China*

Declares, that the products
Product Name and Model(s)

Troponin I Rapid Test Cassette (Whole Blood/Serum/Plasma)	GDTRO-402a
Troponin I Rapid Test Cassette (Whole Blood/Serum/Plasma)	GDTRO-402b

Classification: *Other*
Conformity assessment route: *Annex III (EC DECLARATION OF CONFORMITY)*

We, the Manufacturer, herewith declare with sole responsibility that our product/s mentioned above meet/s the provisions of the Directive 98/79/EC of the European Parliament and of the Council on In-Vitro Diagnostic Medical Devices.

We hereby explicitly appoint

EC Representative's Name: *Shanghai International Holding Corp. GmbH (Europe)*

EC Representative's Address: *Eiffestrasse 80, 20537 Hamburg, Germany*

to act as our European Authorized Representative as defined in the aforementioned Directive.

I, the undersigned, hereby declare that the medical devices specified above conform with the directive 98/79/EC on in vitro diagnostic medical devices and pertinent essential requirements

Date Signed: August 11, 2020

Name of authorized signatory: *Joyce Pang*
Position held in the company: *Vice-President*

Certificate

No. Q5 092305 0001 Rev. 01

Holder of Certificate: **Zhejiang Orient Gene Biotech Co., Ltd.**
3787#, East Yangguang Avenue, Dipu Street Anji
313300 Huzhou, Zhejiang
PEOPLE'S REPUBLIC OF CHINA

Certification Mark:

Scope of Certificate: **Design and Development, Production and Distribution of In Vitro Diagnostic Reagent and Instrument for the Detection of Drugs of Abuse, Fertility, Infectious Diseases, Oncology, Biochemistry, Cardiac Diseases, Allergic Disease based on Rapid Test, PCR and Liquid Biochip Method.**

The Certification Body of TÜV SÜD Product Service GmbH certifies that the company mentioned above has established and is maintaining a quality management system, which meets the requirements of the listed standard(s). All applicable requirements of the testing and certification regulation of TÜV SÜD Group have to be complied with. For details and certificate validity see: www.tuvsud.com/ps-cert?q=cert:Q5 092305 0001 Rev. 01

Report No.: SH2198802

Valid from: 2022-04-11

Valid until: 2024-03-16

Date, 2022-04-11

Christoph Dicks

Head of Certification/Notified Body

Certificate

No. Q5 092305 0001 Rev. 01

Applied Standard(s):

EN ISO 13485:2016
Medical devices - Quality management systems -
Requirements for regulatory purposes
(ISO 13485:2016)
DIN EN ISO 13485:2016

Facility(ies):

Zhejiang Orient Gene Biotech Co., Ltd.
3787#, East Yangguang Avenue, Dipu Street Anji, 313300
Huzhou, Zhejiang, PEOPLE'S REPUBLIC OF CHINA

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STATEMENT

We, Zhejiang Orient Gene Biotech Co., Ltd , having a registered office at 3787#, East Yangguang Avenue, Dipu Street Anji 313300, Huzhou, Zhejiang, China assign SRL SANMEDICO having a registered office at A. Corobceanu street 7A, apt. 9, Chişinău MD-2012, Moldova, as non-exclusive authorized representative for Orient Gene Brand product in correspondence with the conditions of directive 98/79/EEC.

We declare that the company mentioned above is authorized to register, notify, renew or modify the registration of medical devices on the territory of the Republic of Moldova.

This Statement letter will be valid from Feb.21th,2023 to Feb.20th, 2024.

Zhejiang Orient Gene Biotech Co., Ltd

General Manager:

Date:2023/2/21

[Handwritten signature in blue ink]

地址：浙江省湖州市安吉县递铺镇阳光大道东段 3787 号
Add: 3787#, East Yangguang Avenue, Dipu Street Anji 313300, Huzhou, Zhejiang, China
电话 Tel:+86-572-5226111 传真 Fax: +86-572-5226222 邮编 P.C.:313300

Troponin I

Troponin I Rapid Test Device (Whole Blood/Serum/Plasma) Package Insert

A rapid visual immunoassay for the qualitative presumptive detection of cardiac Troponin I in human whole blood, serum, or plasma specimens.
For professional in vitro diagnostic use only.

INTENDED USE

The Troponin I Rapid Test Device (Whole Blood/Serum/Plasma) is a rapid visual immunoassay for the qualitative presumptive detection of cardiac Troponin I in human whole blood, serum, or plasma specimens. This kit is intended to be used as an aid in the diagnosis of myocardial infarction (MI).

SUMMARY

Cardiac Troponin I (cTnI) is a protein found in cardiac muscle with a molecular weight of 22.5 kDa.¹ Troponin I is part of a three subunit complex comprising of Troponin T and Troponin C. Along with tropomyosin, this structural complex forms the main component that regulates the calcium sensitive ATPase activity of actomyosin in striated skeletal and cardiac muscle.² After cardiac injury occurs, Troponin I is released into the blood 4-6 hours after the onset of pain. The release pattern of cTnI is similar to CK-MB, but while CK-MB levels return to normal after 72 hours, Troponin I remains elevated for 6-10 days, thus providing for a longer window of detection for cardiac injury. The high specificity of cTnI measurements for the identification of myocardial damage has been demonstrated in conditions such as the perioperative period, after marathon runs, and blunt chest trauma.³ cTnI release has also been documented in cardiac conditions other than acute myocardial infarction (AMI) such as unstable angina, congestive heart failure, and ischemic damage due to coronary artery bypass surgery.⁴ Because of its high specificity and sensitivity in the myocardial tissue, Troponin I has recently become the most preferred biomarker for myocardial infarction.⁵

PRINCIPLE

The Troponin I Rapid Test Device (Whole Blood/Serum/Plasma) has been designed to detect cardiac Troponin I through visual interpretation of color development in the strip. The membrane was immobilized with anti-cTnI antibodies on the test region. During the test, the specimen is allowed to react with colored anti-cTnI antibodies colloidal dog conjugates, which were precoated on the sample pad of the test. The mixture then moves on the membrane by a capillary action, and interact with reagents on the membrane. If there were enough cTnI in specimens, a colored band will form at the test region of the membrane.

Presence of this colored band indicates a positive result, while its absence indicates a negative result. Appearance of a colored band at the control region serves as a procedural control. This indicates that proper volume of specimen has been added and membrane wicking has occurred.

PRECAUTIONS

- For professional In Vitro diagnostic use only.
- Warning: the reagents in this kit contain sodium azide which may react with lead or copper plumbing to form potentially explosive metal azides. When disposing of such reagents, always flush with large volumes of water to prevent azide build-up.
- Do not use it if the tube/pouch is damaged or broken.
- Test is for single use only. Do not re-use under any circumstances.
- Handle all specimens as if they contain infectious agents. Observe established standard procedure for proper disposal of specimens
- Wear protective clothing such as laboratory coats, disposable gloves and eye protection when specimens are assayed.
- Humidity and temperature can adversely affect results

STORAGE AND STABILITY

All reagents are ready to use as supplied. Store unused test device unopened at 2°C-30°C. If stored at 2°C-8°C, ensure that the test device is brought to room temperature before opening. The test is not stable out off the expiration date printed on the sealed pouch. Do not freeze the kit or expose the kit over 30°C.

SPECIMEN COLLECTION AND PREPARATION

- The Troponin I Rapid Test Device (Whole Blood/Serum/Plasma) is intended only for use with human whole blood, serum, or plasma specimens.
- Only clear, non-hemolyzed specimens are recommended for use with this test.
- Serum or plasma should be separated with soonest possible opportunity to avoid hemolysis.
- Perform the testing immediately after the specimen collection. Do not leave the specimens at room temperature for prolonged periods. Specimens may be stored at 2-8°C for up to 3 days. For long term storage, specimens should be kept below -20°C.
- Bring specimens to room temperature prior to testing. Frozen specimens must be completely thawed and mixed well prior to testing. Avoid repeated freezing and thawing of specimens.

- Pack the specimens in compliance with applicable regulations for transportation of etiological agents, in case they need to be shipped.
- Icteric, lipemic, hemolyzed, heat treated and contaminated sera may cause erroneous results.
- There is a slight possibility that some whole blood specimens with very high viscosity or which have been stored for more than 2 days may not run properly on the test device. Repeat the test with a serum or plasma specimen from the same patient using a new test device.

MATERIALS

Materials Provided

- Test devices
- Buffer

- Disposable Droppers
- Package insert

Materials Required But Not Provided

- Specimen collection containers
- Centrifuge (for plasma only)

- Clock or Timer

DIRECTIONS FOR USE

Allow test device, specimen, buffer and/or controls to equilibrate to room temperature (15-30°C) prior to testing.

- Remove the test from its sealed pouch, and place it on a clean, level surface. Label the device with patient or control identification. To obtain a best result, the assay should be performed within one hour.

- Transfer 2 drops of serum or plasma to the specimen well of the device with a disposable pipette provided in the kit, and then start the timer.

OR

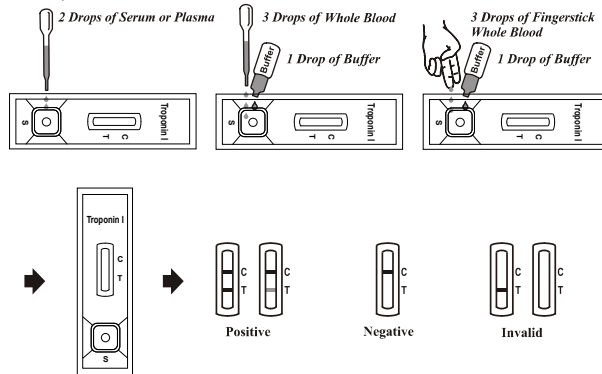
Transfer 3 drops of whole blood specimen to the specimen well of the device with a disposable pipette provided in the kit, then add 1 drop of buffer, and start the timer.

OR

Allow 3 hanging drops of fingerstick whole blood specimen to fall into the center of the specimen well (S) on the device, then add 1 drop of buffer, and start the timer. Avoid trapping air bubbles in the specimen well (S), and do not drop any solution in observation window.

As the test begins to work, you will see color move across the membrane.

- Wait for the colored band(s) to appear. The result should be read at 10 minutes. Do not interpret the result after 20 minutes.

INTERPRETATION OF RESULTS

(Please refer to the illustration above)

POSITIVE: Two colored bands appear on the membrane. One band appears in the control region (C) and another band appears in the test region (T).

NEGATIVE: Only one colored band appears in the control region (C). No apparent colored band appears in the test region (T).

INVALID: Control band fails to appear. Results from any test which has not produced a control band at the specified reading time must be discarded.

Please review the procedure and repeat with a new test. If the problem persists, discontinue using the kit immediately and contact your local distributor.

NOTE:

- The intensity of the color in test region (T) may vary depending on the concentration of aimed substances present in the specimen. Therefore, any shade of color in the test region should be considered positive. Besides, the substances level can not be determined by this qualitative test.
- Insufficient specimen volume, incorrect operation procedure, or performing expired tests are the most likely reasons for control band failure.

QUALITY CONTROL

Internal procedural controls are included in the test. A colored band appearing in the control region (C) is considered an internal positive procedural control. It confirms sufficient specimen volume and correct procedural technique.

External controls are not supplied with this kit. It is recommended that positive and negative controls be tested as a good laboratory practice to confirm the test procedure and to verify proper test performance.

LIMITATIONS

- The Troponin I Rapid Test Device (Whole Blood/Serum/Plasma) is for professional in vitro diagnostic use, and should be used for the qualitative detection of cardiac Troponin I only. There is no meaning attributed to linen color intensity or width.
- The Troponin I Rapid Test Device (Whole Blood/Serum/Plasma) will only indicate the presence of Troponin I in the specimen and should not be used as the sole criteria for the diagnosis of tuberculosis.
- If the test result is negative and clinical symptoms persist, additional testing using other clinical methods is recommended. The test cannot detect less than 0.5 ng/mL of cTnI in specimens. Thus, a negative result does not at anytime rule out the existence of Troponin I in blood, because the antibodies may be absent or below the minimum detection level of the test.
- Like with all diagnostic tests, a confirmed diagnosis should only be made by a physician after all clinical and laboratory findings have been evaluated.
- Some specimens containing unusually high titers of heterophile antibodies or rheumatoid factor (RF) may affect expected results. Even if the test results are positive, further clinical evaluation should be considered with other clinical information available to the physician.

PERFORMANCE CHARACTERISTICS

Table: Troponin I Rapid Test vs. EIA

Method	Troponin I Rapid Test Device		Total Results
	Results	Positive	Negative
EIA	Positive	138	2
	Negative	1	315
Total Results		139	317
		456	

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STATEMENT

We, XEMA LLC, as a manufacturer of in vitro diagnostic medical devices, having a registered office at Akademika Yefremova St. 23, Kyiv, Ukraine assign SRL SANMEDICO having a registered office at A. Corobceanu Street 7A, apt. 9, Chişinău MD-2012, Moldova, as authorized representative in correspondence with legislative requirements of the Republic of Moldova.

We declare that the company mentioned above is authorized to register, notify, renew, or modify the registration of medical devices on the territory of the Republic of Moldova.

This Statement shall come into force on the date of its signing. The duration of this Statement is 3 years from the date of signing.

Date: 06.09.2023

Signature:

Director Xema LLC
Oleksandra Zavaliei

Certificate

Of Marketing Authorization of Medical Product

within Germany, the member states of the European Union
and the other states having a contractual agreement with the European Economic Area

Nr. **AR/IVD/XEMA LLC/01/2023**

Issued on the basis of the Declaration of conformity and registration taking into account Article 11 of Regulation (EU) 2017/746 (IVDR) on In Vitro Diagnostic, and Medical Device Implementing Act (MPDG)

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EC DECLARATION OF CONFORMITY
(Annex III, except point 6, Directive 98/79/EC)
in connection with article 110(3) IVDR

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BfArM Federal Institute for Drugs and Medical Devices
DMIDS (German Medical Device Information and Database System)

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Anhang zum Zertifikat Nr.:

AR/IVD/XEMA LLC/01/2023

The following medical devices can be placed on the market in the Federal Republic of Germany, in the member states of the European Economic Community (EEC) and in the other contract states of the agreement about the European Economic Area.

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