

- 1 or more digits / 1 or more special characters (which can be entered using the keyboard)

3.2.4 Server Options

Item	Description
Overwrite file which is same SOP instance UID	Overwrites the image when receiving the same SOP Instance UID.
Response Received Presentation Context Only	Sends Transfer Syntax that can respond to the requested Presentation Context.
Enable Strict Attribute Matching (DICOM Q/R)	Searches by the exactly corresponding property condition when conducting DICOM Q/R.
Start Server at System Sign In	QXLink Server starts automatically when signed into the system.
Enable Chiropractic Features	Activates measurement tools related to chiropractic.
Enable Orthopedic Features	Activates measurement tools related to orthopedics.
Enable Podiatry Features	Activates measurement tools related to podiatry.
Enable Veterinary Features	Activates the veterinary measurement tools.



- With the option of **Response Received Presentation Context Only**, only the first Transfer Syntax is responsive among the Transfer Syntaxes requested by SCU. Therefore, the rest of Transfer Syntax images are not transmitted when executing DICOM Q/R with C-Get.



- **Enable Veterinary Features** is enabled only if you have registered a veterinary license.
- **Enable Chiropractic Features** is enabled only if you have registered a human license.
- **Enable Podiatry Features** is enabled only if you have registered a human license.
- **Enable Orthopedic Features** is enabled only if you have registered a human license.

3.2.5 Path

Item	Description
Local STORAGE Path	Sets the image storage folder or drive.
Warning Notification	Displays a warning message if the space is less than the configured spare capacity.



- If the storage is insufficient, the following message is displayed.

