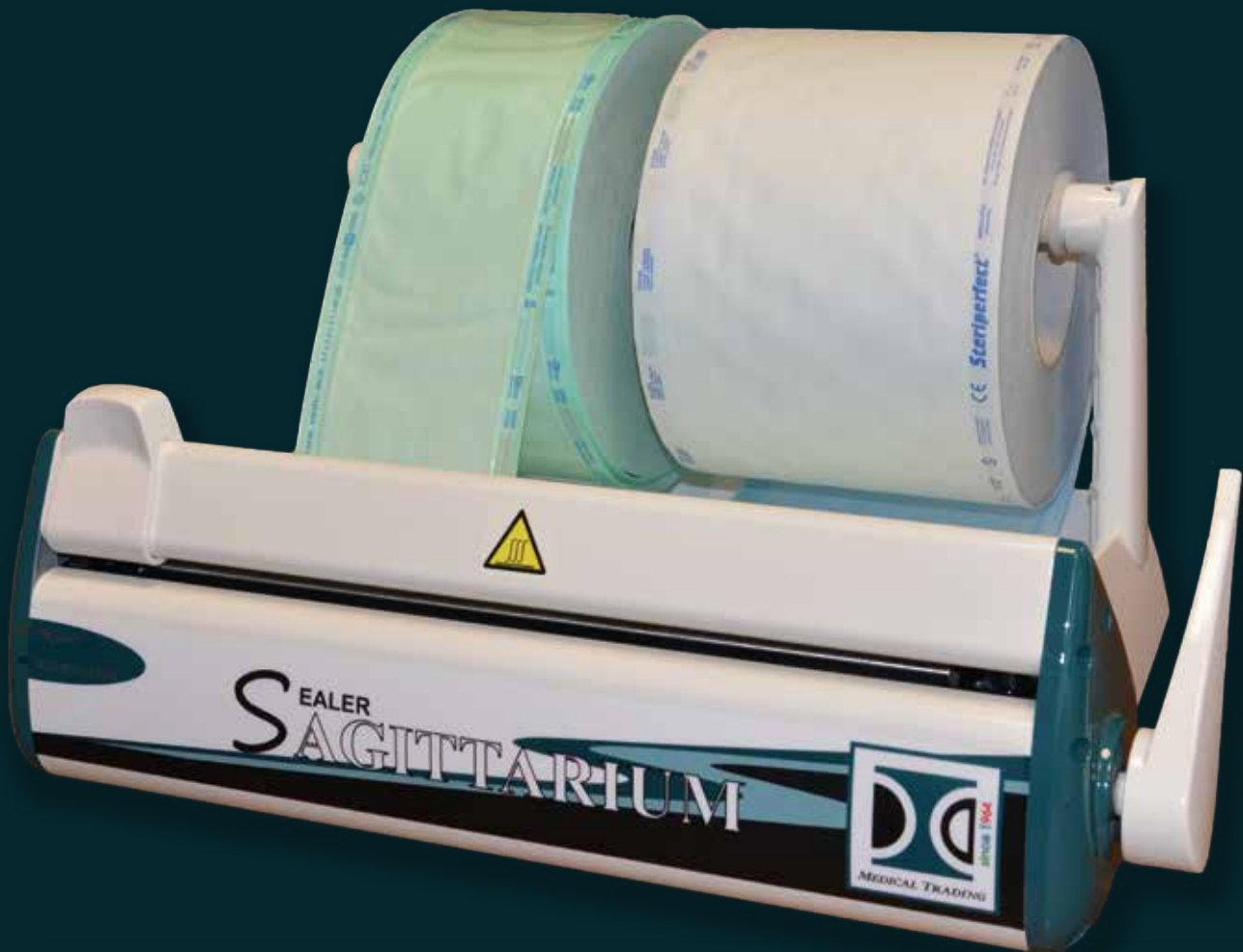
MEDICAL TRADING
ITALIAN MANUFACTURER
OF AUTOCLAVE
FOR STERILIZATION

SEALER SAGITTARIUM

EN
IT
FR

Dear Customer,

We thank you for choosing a quality product manufactured by *GUARISCO MEDICAL STERILIZER S.R.L.*

This unit has been manufactured according to the international Safety Regulations. There isn't any risk for the operator if this unit is used in accordance with the instructions provided in this manual.

IMPORTANT :

- **Read the instructions carefully and look at the pictures, then switch the unit on.**
- **Keep these instructions always at hand.**
- **Only use distilled water to avoid damages to the autoclave and to the instruments to be sterilized.**
- **Follow carefully the instructions in this manual.**

WARNING :

This symbol warns of the need to carefully read the instructions.

We inform that pictures and any other document enclosed with the unit is *GUARISCO MEDICAL STERILIZER S.R.L.*'s property and reserve all the rights. Pictures and Documents cannot be made available at third parties. Reproduction of manual or pictures, even if partial, is forbidden.

CONTENTS

- 1)	GENERAL WARNINGS	PAG. 3
- 2)	EQUIPMENT	PAG. 4
- 3)	SAFETY	PAG. 5
- 4)	TECHNICAL DATA	PAG. 6
- 5)	DESCRIPTION OF THE MACHINE	PAG. 7
- 6)	ELECTRICAL CONNECTION	PAG. 8
- 7)	OPERATION COMMAND	PAG. 9
- 8)	MAINTENANCE AND CLEANING	PAG. 10
- 9)	BOX STORAGE	PAG. 11
- 10)	CONTACT	PAG. 11

1

GENERAL WARNINGS

Carefully read the manual

Warning High Temperature

Grounding

Disposal in accordance with 2002/96/EC

This symbols shown on the product, accessories or instruction manual indicates that the product and its accessories should not be disposed of with household waste at the end of its life in accordance with Directive 2002/96/EC.

To prevent damage to the environment or human health from uncontrolled waste disposal, please separate these items from other waste and recycle them responsibly to promote the sustainable reuse of material resources.

Users should consult the dealer or service center for disposal.

This product and its accessories should not be mixed with commercial waste and / or household for disposal.

EQUIPMENT**GENERAL NOTES ON DELIVERY**

Upon receipt of the machine check that the packaging is not damaged!

Open the box and check that:

- The supply matches with the order (see shipping document)
- There is no evident damage.
- Equipment is included

In case of damage or missing parts, please notify immediately and in details the carrier and *MEDICAL TRADING S.R.L.* or its local representatives.

Do not throw the packaging away, it will be useful if you return the autoclave for repair and / or maintenance.

Photo n° 1

POWER CORD

3

SAFETY

- The unit has not been manufactured to be used in presence of gas or other explosive.
- Do not pour water or other liquids on the unit in order to avoid short circuit and corrosion
- Before any maintenance or cleaning, it is necessary to remove the electricity.

WARNING

If you can not disconnect the electricity and the power switch is far or not visible from the person doing the maintenance, you must affix the sign **WORK IN PROGRESS** on the power switch after placing the switch on OFF position (see Fig. 20). Make sure that the system is equipped with grounding and EMC requirements in the country of installation (see Figure 19).

- Do not remove any label or sign, in case of need ask for a new one.
- Only use original spare parts.

Failure to comply with what above will void Medical Trading's liability

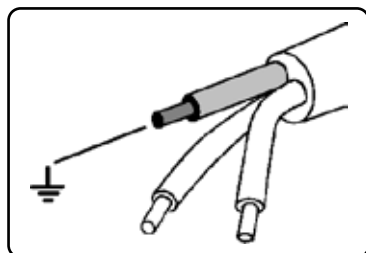
Photo n° 19


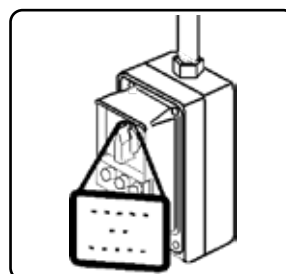
Photo n° 20

The device is built according to current standards and meets all the requirements protection with regard electromagnetic compatibility (EMC).

TECHNICAL DATA

Type of seal:	Constant heat
Welding height:	12 mm
Length of sealing bar:	320 mm
Roll holder width:	320 mm.
Cutting system:	Incorporated
Temperature	200±10 °C (Automatic control)
Warm-up time	< 2 minutes
Sealing time	3 seconds (Can be increased as needed)
Sealing information completed	Light and sound alerts
Sealing speed	7.5 m/Min
Sealing material	Puches/Rolls EN 868-5 Puches/Rolls EN 868-4 Puches/Rolls gusseted
Dimension: L x H x W	365x245x310 mm.
Weight	Net 4,8 kg. Gross 7,0 kg
Power	400 Watt
Voltage	230/240 V.
Frequency	50/60 Hz
Structure material	Steel / ABS injection

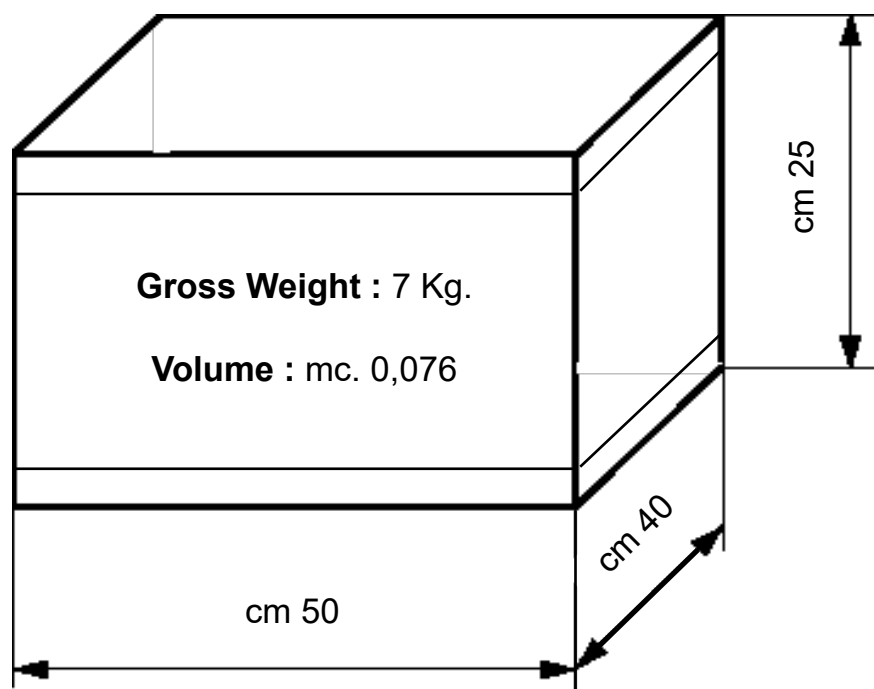
Sealer SAGITTARIUM


5

DESCRIPTION OF THE MACHINE

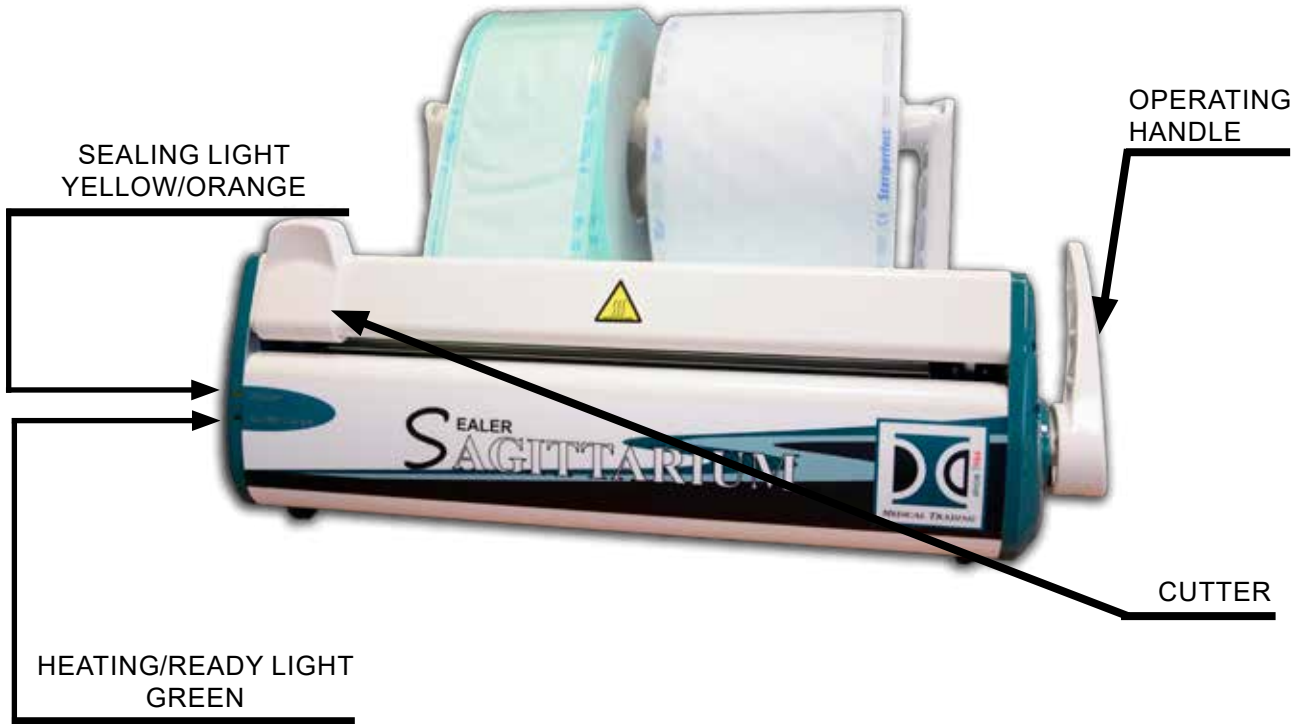FRONT STRUCTURE DESCRIPTION

BACK SIDE DESCRIPTION

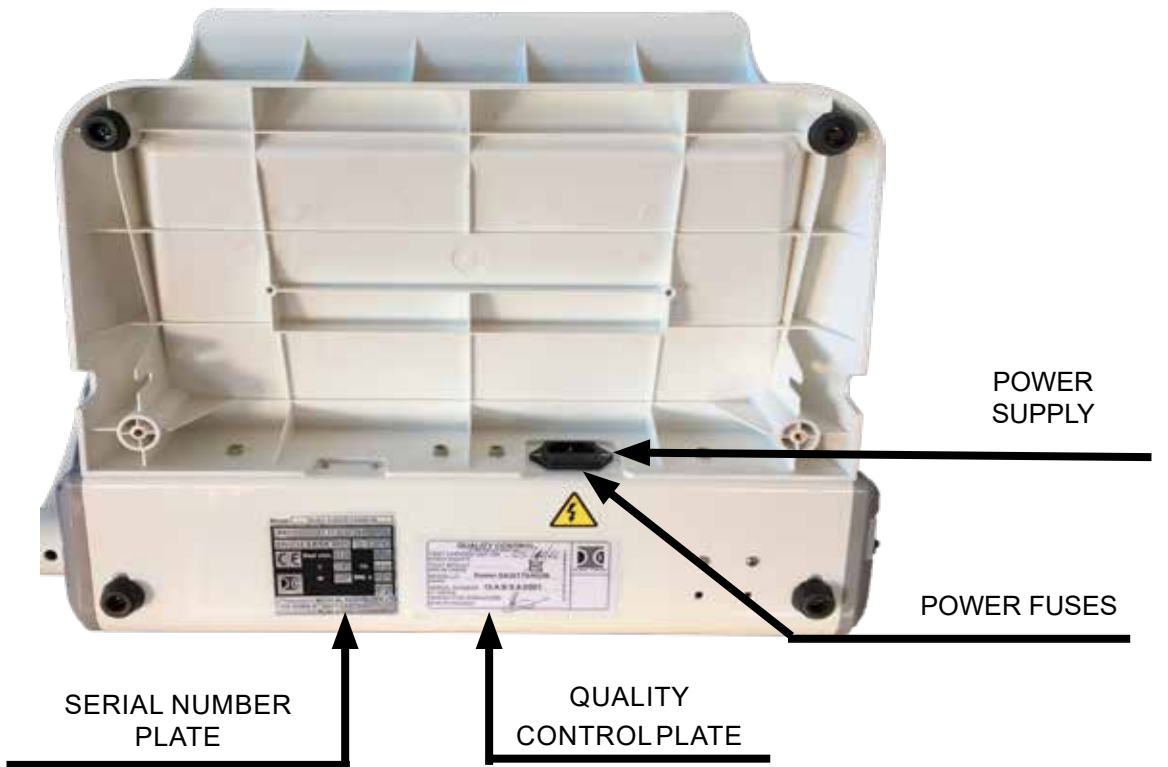ELECTRICAL CONNECTION

Check that the voltage indicated on the label (1) corresponds to that available at the place of installation (see photo no. 32).

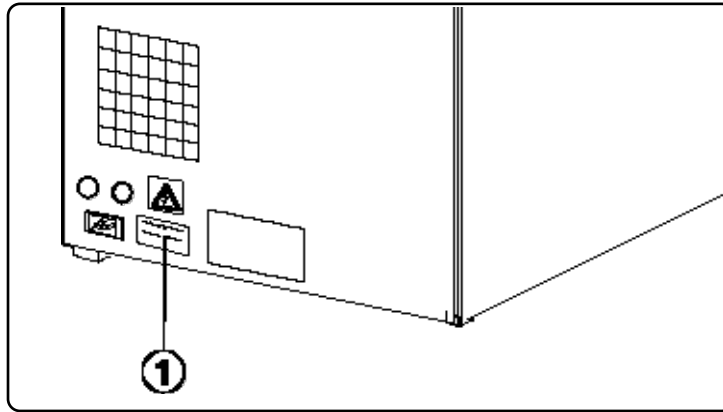
Photo n° 32

The sealer must be connected by a protect switch (2) to a system equipped with properly grounding in accordance with the country safety regulations (see photo No. 33).

ATTENTION

Do not bend the power cord, do not place objects on top of it.

NOTE VALID FOR ITALY :

The installation must be performed according to CEI 64-4 and a differential switch (3) with the following characteristics must be installed before the power supply (4) (see photo N° 34) with the following characteristics:

- **Rated current: 10 A**
- **Differential sensitivity: 0.03**

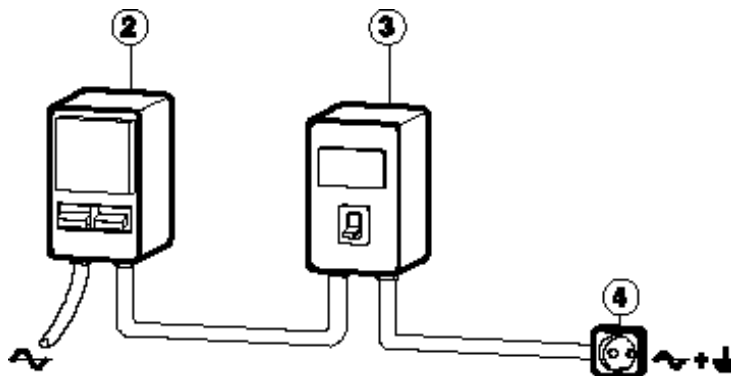
Photo n° 33

7

OPERATION COMMAND

- 1 - Place the rolls in the roll holder
- 2 - Insert the power cord
- 3 - Turn on the power switch
- 4 - At this point the heating light (GREEN) will start flashing.
- 5 - Wait until the GREEN light turn ON(stop to flashing) to indicate that the sealer is ready. (around 2 minutes)
- 6 - Put the roll of the measures required. Make sure that you have insert the rolls in the right way: with the plastic part up and the paper down
- 7 - Lower the lever and wait 3 second.

After 3 second there will a "beep" to indicate the sealing complete, and the yellow light keep on. If you don't release the handle for 6 second the machine will alarm you by constant sound alarm.

- 8 - Cut the roll with the appropriate cutter
- 9 - Lift the lever
- 10 - Insert the instrument to be sterilized in the bag
- 11 - Repeat the procedure from operation n ° 7

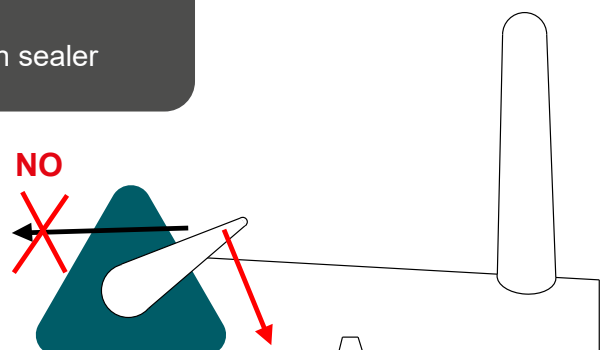
ATTENTION

Sealer is always warming up, to turn off please push the power button and unplug the power cord

RECOMMENDED

Push the HANDLE towards, to the back of the sealer

MAINTENANCE AND CLEANING

The required controls consist of an executable routine maintenance by the operator and preventive and / or corrective executable by Customer Service.

Like all electrical equipment, this machine requires a proper use and also a regular maintenance and control.

This precaution will ensure a continuous operation, safe and efficient apparatus. To prevent any possibility danger, to the end user, is necessary that the equipment is subjected to regular control and maintenance by the technical assistance.

**ATTENTION:**

In case of replacement of parts which have a direct or indirect influence ON SAFETY please use original spare parts only.

ROUTINE MAINTENANCE AND CLEANING

- For a good maintenance of the sealer, clean all the external parts periodically using a wet cloth with neutral detergent which is not corrosive or abrasive.
- Do not use abrasive cloths or metallic brushes for cleaning.

**ATTENTION:**

Before making any operation, switch off the autoclave and allow it to cool down.

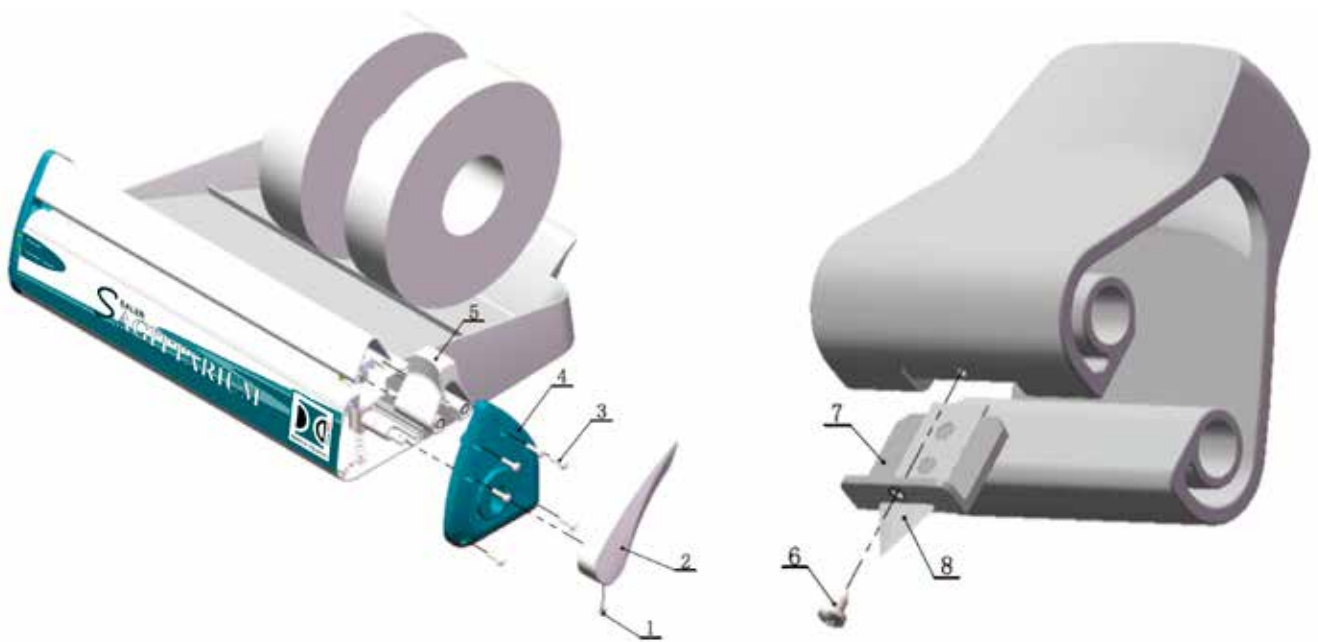
CLEANING

Keep clean the bars of sealing and always check that there are no objects, pieces of the bag between them.

Make cleaning and checking periodically, at least once a week, or when necessary.

REPLACEMENT BLADE CUTTER

- a) Unscrew the handle fixing by Allen wrench **1** - **2**
- b) Unscrew the rest 6 screw by CROSS screwdriver **3**
- c) Take out side plastic **4**
- d) Take out the cutter group **5**
- e) Unscrew the screw by little CROSS screwdriver **6**
- f) Take out the cutter **7**
- g) Replace the blade of the cutter **8**
- h) Assemble all the parts back

9 - INFORMATION FOR THE SHIPMENT

If is necessary to ship the sealer for repair, inspection, overhaul, follow the following informations:

- Use the original packaging, if that is no longer in your possession using adequate packaging. The goods travel with the risk borne by the sender.
- Send only the sealing (do not put any component contained in the accessory kit).
- Thoroughly clean the sealing.
- Indicate in writing, packaging and insert a document where you indicate precisely the anomaly or service that you intend to qualify.
- Also specify in writing the following information: address, name of the study, the opening hours.

CAUTION:

All non-original packaging that are received there will be disposed of.
The autoclaves will be back with new and original packaging, this to ensure maximum protection to your product during transport.
If not present at the machine receiving the packaging cost you will be charged.

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