La Procedurile administrative pentru notificarea dispozitivelor medicale care dețin marcajul CE

Către Agenția Medicamentului și Dispozitivelor Medicale

### **NOTIFICARE**

pentru înregistrarea dispozitivelor medicale în Registrul de stat al dispozitivelor medicale nr. TR-50 din 09.06.2023

III. <u>IR-50</u> dii	1 09.06.2023
	RL_, cu sediul <u>or. Chişinău str. Grenoble 193</u> (adresa)
solicit înregistrarea în Registrul de	022768462 , e-mail triumf.motiv@mail.ru, stat al dispozitivelor medicale a următoarelor cale pentru introducerea și punerea la dispoziție
Ear, Nose & Throat Treatment Units, Endoscope	e G60
Se anexează următoarele acte:  DECLAȚIE DE CONFORMITATE CE  CERTIFICATUL DE CONFORMITATE  Scrisoare de autorizare de la produc	
Data <u>09.06.2023</u>	Semnătura
- · · · · · · · · · · · · · · · · · · ·	ionare a notificării în momentul depunerii notificării de către
Comentarii cu privire la acceptul/refuzul recepționării notificării, inclusiv motivul refuzului	
Data/nr. de ordine atribuit notificării de către Agenție (în cazul acceptării recepționării)	
Numele, prenumele, funcția persoanei responsabile de recepționarea dosarului	
Semnătura persoanei responsabile	

Către Agenția Medicamentului și Dispozitive Medicale

### **DECLARAȚIE PE PROPRIE RĂSPUNDERE**

Solicitant:	TRIUMF M	OTIV SRL		, cu
sediul <u>or. Chi</u>	şinău str	. Grenoble	193	
declar pe propri	e răspundei	re, cunoscá	and prevederile	art. <b>352<sup>1</sup></b> , Codul Penal al
Republicii Moldova cu	ı privire la f	alsul în de	claraţii, că docu	ımentele și datele furnizate
pentru notificarea di	spozitivului	medical:		
Ear, Nose & Throat Treatm	nent Units, End	doscope	G60	
Sunt autentic	e și cores <sub>l</sub>	ound reali	ității.	
	Nume	ele, prenur	nele și funcția	Jighili Tatiana, Administrator Semnătura
				Data <u>09.06.2023</u>



# DOCUMENTATION REVIEW

S/N: 003285

### No.: ICR Polska/DR/HS220809

Applicant: Hunan Fude Technology Co., Ltd.

address: D3 Building, No 106, Xujiazhou Road, Yuelu Street, Yuelu District, Changsha, Hunan,

China

Manufacturer: Hunan Fude Technology Co., Ltd.

D3 Building, No 106, Xujiazhou Road, Yuelu Street, Yuelu District, Changsha, Hunan,

China

Review goal: Verification of the presence of Technical Documentation compatible with the Medical

Devices Regulation (EU) 2017/745 Annexes II & III

Product: Ear, Nose & Throat Treatment Units, Endoscope

Model(s): G65, G60, G35, G30, G80, G85, G90, G95, Pioneer, G010, G011, G012,

HA37, HG37, HW37, HU37T, FD-60L, FD-52L, FD-50L, FD-32L, FD-29L, FD-28L,

FD-26L, FD-49S, FD-48S, FD-46S, FD-45S, FD-58A, FD-56A, FD-54A

Product trademark: FUDE

Declared Class II (Not Sterile according to the Manufacturer's declaration – does not require

classification: the participation of a NB)

Technical ENTTCF0802-MDR

Documentation: prepared by Hunan Fude Technology Co., Ltd.

Date of review: 24.08.2022 Expiration date 23.08.2027

### **Review output:**

- This document was issued voluntarily and at the request of the applicant.
- The above-mentioned Technical Documentation provided to us by the applicant is compatibile in terms of the general requirements for the documentation of this type of product, but its scope of review is not equivalent to Notified Body assessment and such shall be carried independently.
- The manufacturer is responsible for the CE marking process and is not exempt from carrying out all necessary activities related to the legal compliance.



ICR Polska Co. Ltd. www.icrpolska.com icrpolska@icrqa.com



Director: Rafał Kalinowski



### EC DECLARATION OF CONFORMITY



### 1. Name and address of the manufacturer:

Hunan Fude Technology Co., Ltd.

D3 Building, No 106, Xujiazhou Road, Yuelu Street, Yuelu District, Changsha, Hunan, China.

### 2. EC Representative:

Tech4med SRL

Strada Nufarului 30, Bacau, Romania

Tel: +40 750 473 423

### 3. Description of the machine

*Name:* Ear, Nose & Throat Treatment Units, Endoscope

### Model/type:

G65,G60,G35,G30,G80,G85,G90,G95,Pioneer,G010,G011,G012,HA37,HG37,HW37,HU3

7T,FD-60L,FD-52L,FD-50L,FD-32L,FD-29L,FD-28L,FD-26L,FD-49S,FD-48S,FD-46S,FD-

45S,FD-58A,FD-56A,FD-54A

Parameters: -

### 4. Declared classification: Class II b

Meets all the provisions of the directive MDR 2017/745 Annexes II & III which apply to it.

### 5. Harmonized standard applied

EN 60601-1-2:2015+A1:2021, EN 55011:2016+A1:2017+A11:2020+A2:2021, EN IEC 61000-3-2:2019+A1:2021, EN 61000-3-3:2013+A1:2019+A1:2021

6. Notified body: ICR Polska Co, Ltd.,

Plac Przymierza 6, 03-944, Warszawa

**POLAND** 

7. Identification of subscriber and on behalf of Hunan Fude Technology Co., Ltd.

Surname and family name:..湖.. 南.. 复. . 德.. 科. 技 有 限

General Manager:....

Authorized Signature(s)

ulie Shu



### Hunan Fude Technology Co,. Ltd. | 南 复 德 科 技 有 限 公 司

D3 Building, No 106, Xujiazhou Road, Yuelu Street, Yuelu District, Changsha, Hunan, China. Tel: 0086 18773728649 Email: info@fude-techgroup.com Web: www.fdmedical.cn

### **LETTER OF AUTHORIZATION**

We, Hunan Fude Technology Co., Ltd., based in D3 Building, No 106, Xujiazhou Road, Yuelu Street, Yuelu District, Changsha, Hunan, China, assign Triumf Motiv SRL, based in StrGrenoble 193, Chisinau MD -2011, Moldova, as **authorized representative** in correspondence with the conditions if directive 93/42/EEC, 98/79/EEC and 90/385/EEC. We declare that the company mentioned above is authorized to register, notify, renew or modify the registration of medical devices on the territory of the Republic of Moldova.

Place: Hunan, China Date: June 9, 2023

Signed:



# Certificate of Registration

Certificate NO: 101385650

This is to Certify that the Quality Management System of

# Hunan Fude Technology Co., Ltd.

Building D3, No 106, Xujiazhou Road, Yuelu Street, Yuelu District, Changsha, Hunan, China

Has been audited to the following Quality Management System standard:

## ISO9001:2015

This system is valid for the

Production and sales of Ear, Nose & Throat Treatment Units, ENT Patient Chair, Endoscope, Endoscopic Camera System, LED light source, Microscope, Dental Unit and accessories, Steam Sterilizers, X ray, Audiometer, Hearing aids, LED headlight, Surgical Loupes, Electrosurgical Unit, Surgical light, Teeth whitening system, Dental handpiece, LED curing light, ECG machine, Ultrasound, Ni-Ti Alloy Self-expanding Stent System, Ventilators, Mesh Nebulizer, High Flow Heated Respiratory Humidifiers, Oxygen Concentrator, Injection Needle, Ligature Endoscopic Loops, Plastic Stent, Preloaded Stent, Balloon Dilatation Catheters, Biopsy Forceps, Guidewires, Sphincterotome, Cytology Brush, Cleaning Brush, Drainage Catheters, Disposable Pressure Transducers, Medical Diode Laser System, Medical Intense Pulsed Light Treatment System, YAG Laser Therapy Systems, Medical Photodynamic Therapy(PDT) Equipment and Medical Fractional CO2 Laser Therapy System, Skin Analysis System etc medical equipment

Date of issue: Aug. 12, 2022

Date of expiry: Aug. 11, 2025

This certificate will not remain valid only if the certified organization accepts at least one surveillance audit annually within the validity period of the certificate in which the surveillance audit conforming mark is in the designated position on the certificate.





Issued by:

12months 24months

CERTIFICATION EUROPE(HONGKONG)LIMITED WWW.CCE-HK.CN



REGISTRATION NO. 04718Q10000136

# CERTIFICATE OF QUALITY MANAGEMENT SYSTEM FOR MEDICAL DEVICES

This is to certify that the quality management system of

Hunan Fude Technology Co., Ltd.

Registered Address: D3 Building, No 106, Xujiazhou Road, Yuelu Street, Yuelu District, Changsha, Hunan, China.
Manufacturing Address: D3 Building, No 106, Xujiazhou Road, Yuelu Street, Yuelu District, Changsha, Hunan, China.

Has been assessed and conformed to the following standard(s) YY/T 0287-2017 idt ISO 13485:2016

The certificate is valid for the following scope:

The design, Development, Production of ENT Treatment Unit, ENT Patient Chair, Endoscopic camera system, Endoscope, Electrosurgical Unit, Microscope, LED headlight, Surgical Loupes, Dental Unit, Teeth whitening system, Dental lamp, Dental handpiece, ECG machine, Patient monitor, Medical stent, Audiometer, hearing aid, Laparoscope set with instrumentation, X ray, Airway Clearance System

Date of issue: April 13,2022 Date of expiry: April 12,2025

General Manager:

BEIJING HUA GUANG CERTIFICATION OF MEDICAL DEVICES CO., LTD.

# INSTRUCTIONS G60

**ENT Treatment Unit** 

**Hunan Fude Technology Co., Ltd.** 

# Content

Co	ntent	1
Saf	fety information	2
1.	Description	2
	Structure	
3.	Capability presentation	3
	Working condition	
	Range of application	
6.	Equipment installation	4
7.	Operating instruction	6
8.	The fuse model and rated value	g
9.	Disinfect and clean	9
10.	Maintenance	9
11.	Usual malfunctions and obviation	10
	Tab and logo	
13.	Delivery and store	12
	Notice	
15	Diagram	13

### Safety information

Please read manual carefully, to operate and keep ear & nose &throat treatment unit well and save, manual to have a reference in the future, please connect with local agent of manufacturer if need any special requirement

### **Notice**

- 1. Please do not put equipment in wet place to reduce possibility of fire and Levin, please do not open box if you are not the careen man to avoid touch some spare parts with high voltage, please look for high careen man if need repair machine.
- 2. Please cut off the power and connect with local agent or after sale service place, if machine smocking or other unusual thing, to avoid possibility of fire.
- 3. Please cut off the power and contact with local agent or after sale service place if water go into machine, as it lead to fire happen.
- 4. Please cut off power and contact with local agent or after sale service place if some unusual situation happen.
- 5. Please do not teardown or re-install equipment, otherwise it will destroy spare parts or Levin happen.

### Ps: Do not re-install machine! It's out of maintain if something destroyed.

- 6. Power supply wire can't be destroyed, please cut off the power and do not touch destroyed wire directly, then contact with local agent and replace destroyed wire
- 7. Ear& nose & throat treatment unit should be work under a.c220V/50HZ, it's very dangerous to operate under other voltage
- 8. Please mind the change of temperature and cut off power in case of raining,
- 9. Do not touch power supply pin when your hand is wet, please cut off power when machine out of usage.
- 10. Do not clean equipment with insecticide, alcohol, thinner and chemic. It will make surface corrosively.

Please record machine No	
Model: (G60 ENT treatment unit)	
Please keep this manual well	

### One: Description

Ear, Nose, Throat treatment unit is a new range ENT treatment unit that according to the development of home and abroad ENT equipment, it widely use in ENT sections, this series machine mainly including curing table (curing system), lighting system, chair curing system use the stand-alone compressor and vacuum pumps for spray guns and sucking guns, its gas flow rate stability, low noise, lights can adjusts to any height, any angle and reliable angles, making your work more handy.

Dangerous: power must have reliable earth wire


Attention: cut off the power before clean and repair the machine

Attention: cut off the power before you leave the office

### **Two: Structure**

1. The structure: consists of treatment table, operating light system, the treatment table including: spray gun, sucking gun, blowing gun, pre-heater, wasting system, the operating system including the film viewer and cool light (excluding the options)

### 2. Diagram

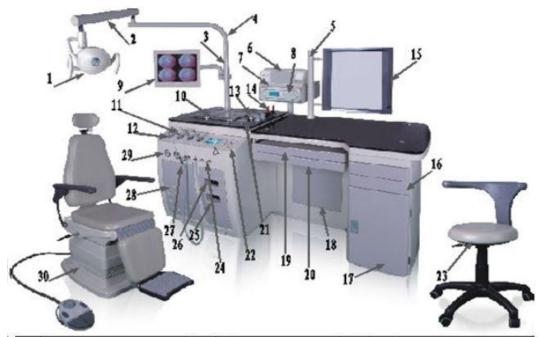

1	Operating light	11	spray gun	21	preheater
2	Operating light arm	12	blowing gun	22	control panel
3	lampstandard	13	instrument cup	23	doctor stool
4	bend lamp pole	14	medical bottle	24	air pressure regulating valve
5	folm viewer	15	film viewer	25	caton cup
6	printer	16	right drawer	26	waste instrument place
7	camera bracket	17	computer drawer door	27	sucking gun
8	camera	18	inner door	28	front door
9	monitor	19	keyboard splint	29	air pressure meter
10	instrument tray	20	middle drawer	30	motor patient chair

### **Three: Capability presentation**

- 1. The surface of machine is easy to clean and disinfect
- 2. Freestanding motor, sucking gun and spray gun can work at the same time to extend air's compressor working tine
- 3. Operating light can adjust to any angle and any height, its light can be refract into ear, nose, throat, and other place by reverberator
- 4. Positive-negative regulating valve can adjust the sprayer and suction
- 5. Positive-negative manometer show the power of sprayer and suction
- 6. Control panel is easy to operate

- 7. The capacity of the waste bottle are 2500CC
- 8. Medicine bottle with a capacity of 30CC, instrument cup's diameter are 80 and 100 mm
- 9. The cold light source (built-out), two fiber joints can be connected to be fiber otoscope, endoscope, fiber larynx illuminating rod etc.
- 10. **spray gun**:3 spray guns(spraying equipment: the medicine bottle in spray gun can hold difference medicine for curing snufflen, shrinking nose catarrh, nose congest, laryngitis and anaesthesia before operation)
  - **sucking gun**: 1 sucking gun(plug attractive device, in addition to nasal absorption of the gun, the gore wound, pus and so on, to ensure the operation and affected part clear),
  - **blowing gun**: 1 blowing gun (can be used for the ear blowing, and the stem of wound)
- 11. Preheating device: baking indirect laryngoscope, nasopharyngeal mirror, making the mirror suitable for the body temperature(indirect laryngoscope, nasopharyngeal, mirror nasopharynx can check whether throat retained have another thing or not

### Four: Working condition

To make sure the machine working normally, its working condition's technical equipment as follow:

- 1. Power source:AC220V ,50Hz
- 2. Environment temperature: 5°C-40°C
- 3. Relative humidity: ≤85%


### Five: Range of application

The ENT treatment unit is used to checking and curing in ENT department of hospital


### Six: Equipment installation

Notice: make sure every installation wire is connected firmly. And the protective casing of wire is in good condition

- 1. Put the unit in the preserved place (when flitting ENT unit, handle the button but not the top of the unit table. Make sure all balance nuts touch floor well, screw up the nuts and wheels touch the floor then you can move the ENT unit
- 2. Installation of the lamp standard and operating light
- (1) Open the package of the lamp standard, lamp arm, light
- (2) Installation of the : open the door of cabinet, connecting two connector of the lamp standard and the screw thread on the right of cabinet back door and fixed it by the spanner
- (3) Put electrical wire with the" operating light" logo through the lamp standard from down to up, as the "picture 1" showed, install the lamp arm in turn, take out the operating light, connecting electrical pin of the operating and the electrical pin of the front lamp arm, and put it into operating light fixed axes. Then out the decorating ring on fixed axes, put the operating light's fixed axes into the lamp tandard and fixed behind M5X12 nut.

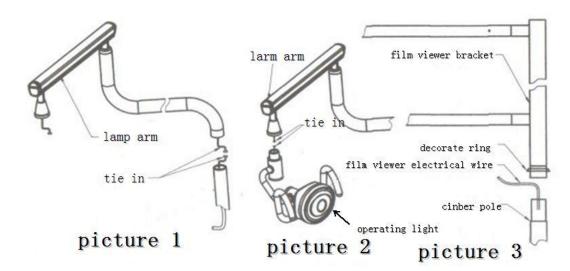

- 3. Installation of the film viewer
- (1) Take out the film viewer, film viewer bracket, film viewer standard from the packing box
- (2) According to the structure diagram, open the door of the cabinet first, fixing the film viewer standard, film viewer bracket in turn as "picture 3" showing, then hang the film viewer on the two M5 nuts of the bracker and fixed it by the spanner, last put the film viewer spint and fix it
- 4. Installation of the air compressor and vacuum pump
- (1) Open the package and take out the air compressor and vacuum pump
- (2) Put the air compressor on the right of cabinet, insert the air outlet with "air compressor" logo to the joint of tie-in and fix it.

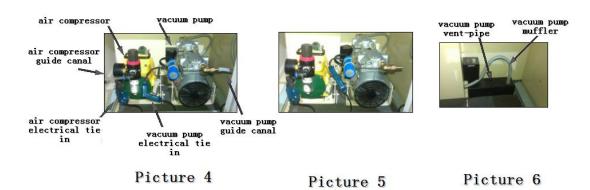

- (3) Put the vacuum pump on the left of the cabinet, insert the air outlet with the "vacuum pump" logo to the tie-in with the dame logo and fix button
- (4) Respectively insert the plug of vacuum pump and air compressor to the with the logo to the pin stand
- 5. Installation of the waste bottle
- (1) Open the package and take out the sucking waste bottle
- (2) Open the cabinet side board and put the waste bottle on the bracket, and cover

waste bottle with "waste water rubber cover" to make sure the pressurization between the cover and the bottle

Notice: When you use the suction function, please drainage waste water if the machine alarm, otherwise, waste water will into vacuum pump.

6. Please move active block which fixed on the active block leader to normal working situation (the active block will fixed on the top of leader to avoid active block will dragoon control switch on the spray gun on transportation)




active piece normal working situation

picture 7


picture 8

### **Seven: Operating instruction**

- 1. Main electrical switch: control all the operation
- 2. Sucking gun: Hanging equipment, you can use it as you take it on, it will stop working when you put it back
- 3. Spray gun: you can collocate the different medicine into the bottle built-in the spray gun, as the picture showing, fit screw the locking nut, second, screw the medicine bottle and fit the medicine needed, last, fit it in turn, air compressor begin to work as soon as you take the spray gun on, press the button and spray gun can work,( spray gun normal working pressure:0.18mpa-0.26mpa) the regulation ralve on the right of spray gun can regulating the working pressure.
- 4. Blowing gun; can be used for blowing the ent throat, the drying of the wound.

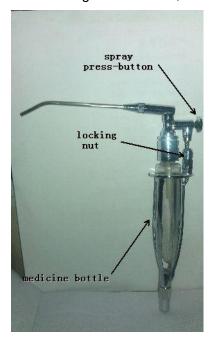

### 5. The operating illustration of the control panel adjuster of cold light

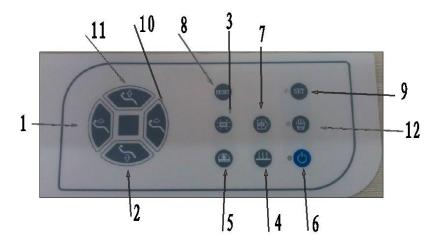



### (1) Backrest reclining key

Press the key, the backrest will move forward to the required position, release the key, it will stop working at once, press on the key all the time, the backrest will reach backward to top point and then stop move automatically



### (2)Cushion declining key

Press the key, the cushion will move downward to the required position, release the key, the cushion will stop working at once, press the key all the time, the cushion will decline downward to the lowest point and then stop moving automatically.



### (3) The control key of film viewer

The light will on by press the key, then you can view x ray piece, the light off by pressing the key again.



### (4) The control key of pre-heater

The pre-heater will working by pressing the key, such as baking the sinuscope and laryngoscope etc.(the pre-heater will stop working after 0-180 seconds by itself, as showed: pre-heater time setting key) press the key when it needed.



### (5) The operating light's control key

The light will on by pressing key, then for a check, the light will work off by pressing the key again



### (6) Power supply indicator light

This key is switch for power supply. Connect power supply wire and open power supply switch, press this key for three seconds and buzzer will sound, power supply indicator light will be on and LCD show all function on normal work situation, press this key for three seconds when machine working and buzzer will sound again, power supply indicator light will off.



### (7) Cold light key

Press the key, you can open the Cold light source lamp, then press the button to close.



### (8) Rest key

The chair to be the lowest place to limited place by press this key (it will stop working by press any key)



### (9) Set key

### 1. Pre-heating time setting

Press the key 3 seconds, you can hear the buzzer sound, press the pre-heating key setting the time what you need, and press the key once again

### 2. Operation light (ten degree lightness)

Press the set key and hear buzzer sound and see the key bright, press the operating light key (weak/ middle/ strong light) to comfortable what you need, press set key again and hear buzzer sound, the situation were between open and close of the operation light.

### 3. Cold light setting (ten degree lightness)

Press the key 3 second, you can hear the buzzer sound, press the cold light key (weak/middle/ strong light) to comfortable what you need, press set key again and hear buzzer sound. The situation were between open and close of the cold light.



### (10) Backrest fore rake key

Press the key, the backrest will move forward to the required position, release the key, the backrest will stop working, press on the key all the time, the backrest will move forward to the lowest point and then stop automatically.



### (11) Cushion uprising key

Press the key, the cushion will move upwards to the required position, release the key, it will stop working at once, press the key all the time, the cushion will catch the top point and stop working automatically



### (12) Clean system

The water heater heat automatically once open the machine (the indicator light bright

when the water temperature between 36-39  $^{\circ}$ C, then you can use it. The heater closed automatically under the 36  $^{\circ}$ C water temperature)

### Remark

- 1. The system can cut off the power supply automatically without operating within 4 and half hours
- 2. The chair can stop by any key when move automatically
- 3. All setting parameter with self-protect function
- 4. When machine rework after power off, everything will safe besides operating stop
- 5. The machine can check all function when power supply light on
- 6. Air compressor with self-protect function (air compressor can stop working after 20 minutes)

### Eight: The fuse model and rated value

1. Fuse model: BGXP.

2. Rated value: 1.Operating light fuse rated value: F1A250V,

2. Cold light fuse rated value: F1A/250V .

3. in220V Power fuse rated value: F10AL/250

### Nine: Disinfect and clean

- **1.Spray gun:** 3 pcs, the medicine bottle was Removable and replaced medication, the nozzle and pipe was Removable, please using up 75% alcoholic disinfect the spray gun after using.
- 2. Suction gun: 1pcs, it's equipped 5 pcs nozzle which 3 pcs 2.55mm and 2 pcs 3mm
- 3. Blowing gun: 1 pcs which used for dry surface.
- 4. Please cleaning the dirt collecting bottle every day to avoid twice contaminate.

### **Ten: Maintenance**

In the daily maintenance, the unit should keep clean, the pipeline should keep smooth and the leaking should be banned, the rotation should add the lube in time to make the equipment work in the normal condition

- 1. Clean the spry gun in time, in case the mouth built up
- 2. Check the waste water in sucking bottle if extend the imitation or mot, don't
- 3. The operating light piece should keep in the normal temperature and clean with the cloths.

### Eleven: Usual malfunctions and obviation

Phenomena	Possible causes	Solutions	Remark
-----------	-----------------	-----------	--------

	① The power supply can	①connect the power	
	not connect	supply switch	
	② The power supply	②Replace the fuse	
	switch fuse destroyed	③replace the	
	③ Equipment fuse	equipment inside	
Equipment	destroyed	fuse	
Out of work	4 Equipment main switch	4) open the main	
	not to/be open	switch	
	①bulbs are burned out		
	②Lamp is loose connect	①replace the bulbs	
	③Fuse in the	in the same models	
	second-level power,	②revise and then	
	transformer is burned out	inset	
Operating light	4 The inner wire in the	③replace the fuse in	
off	movable joints are twist	the same	
	off	4)change new lines	
	①Bulbs are burned out	①replace the bulbs	
	②Lamp is in loose	in the same models	
	connect	②revise and then	
	③Fuse in the	inset	
	second-level power	③replace the fuse in	
	transformer is burned out	the same	
Cold light off	④The control panel is	④The control panel	
J S S S S S S S S S S S S S S S S S S S	broken	is broken	
	①The electric fan's switch	①Open the switch	
	no be open	②Insert the pin well	
	②The power supply pin	again	
	not to be inserted well	③Change the same	
	③The electric fan broken	model goods	
Pre-heater(ele	④The control panel is	④The control panel	
ctric fan) off	broken	is broken	

Dhonomono	Possible causes	Solutions	Remark	
Phenomena	Possible causes	Solutions	Remark	

The sucking gun out of work	①the power pin of vacuum pump not to be inserted well ②the sucking bottle not to be covered well ③windpipe tie-in not to be inserted well and destroyed ④the journey switch of sucking is not to be inserted well and destroyed ⑤the vacuum pump broken	①insert the vacuum pump pin again ②cover the sucking bottle well ③lock the windpipe tie-in tighten or change the windpipe ④check the wire and change the journey switch ⑤change the same model vacuum pump	
Spray gun is out of work or the pressure meter no pressure	①the power supply pin of the air compressor is not to be inserted well ②the switch of air compressor is close ③the regulation valve of spray gun is turned to the lowest ④the journey switch of spry gun not to be inserted well and destroyed ⑤windpipe tie-in not to be inserted well and destroyed ⑥air compressor is broken	①insert the compressor vacuum well again ②open the air compressor switch ③turn the regulation valve to the 0.20-0.25MP ④check the wire or change the journey switch ⑤lock the wire tie-in tighten or change the windpipe ⑥change same model air compressor	
Air compressor out of work	①the block on the pole is locked that can't control the switch ②journey destroyed	①look for the reason why the block is locked and find out the solution ②find out the destroy journey and change a new one	

Twelve: Label and packing identifier

1. Product name: ear& nose& throat treatment unit

2. Goods: ENT unit

3. Product register: 25408694. Product license: 20122168

5. Power: a,c.220V/50Hz Input power:1200VA



: Please consult the documents



: Avoid under raining



: Fragile, move carefully



: Should be placed up 90 degree


### Thirteen: Delivery and store

- **1.** Please keep production placed up 90 degree, without under raining and move carefully when you transport.
- **2.** The production should be storage under  $-40^{\circ}\text{C} \sim 55^{\circ}\text{C}$ , without non-corrosiveness gas and well-ventilated.

### **Fourteen: Notice**

- 1. The product just for the qualified professional doctor
- 2. The doctor just shut down the power and unplug the power cord plugs to ensure product safety when they off work or not to use the product everyday
- 3. Do not allow sewage enter into the vacuum pump
- 4. We should cut off all power before replace electronic original
- 5. We must cut off the power supply before maintenance and clean the equipment

Fifteen: Diagram

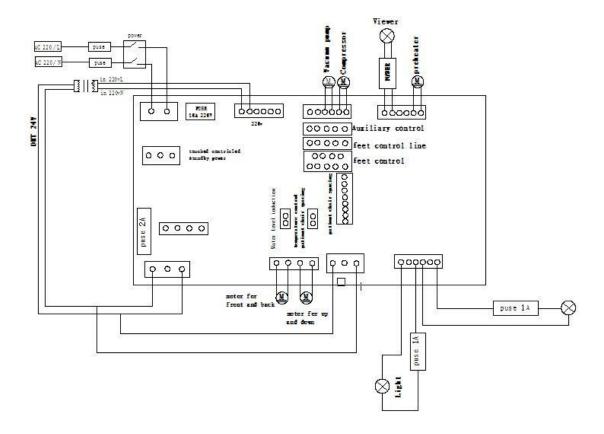

### Hunan Fude Technology Co., Ltd.

D3 Building, No 106, Xujiazhou Road, Yuelu Street, Yuelu District, Changsha, Hunan, China

Cell: +86 18773728649, +86 13365870198

Website: www.fdmedical.cn