# ALP

**mindray**

## Alkaline Phosphatase Kit (IFCC Modified Method)

### Order Information

| Cat. No.      | Package size              |
|---------------|---------------------------|
| 105-000816-00 | R1: 4×35 mL + R2: 2×18 mL |
| 105-000855-00 | R1: 6×40 mL + R2: 2×32 mL |
| 105-004593-00 | R1: 6×58 mL + R2: 3×32 mL |

### Intended Purpose

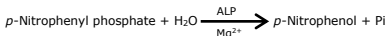
In vitro test for the quantitative determination of Alkaline phosphatase (ALP) activity in human serum and plasma on Mindray BS series chemistry analyzers. It is intended to be used for aiding to diagnose hepatobiliary diseases.

### Summary<sup>1-3</sup>

Alkaline phosphatase (EC 3.1.3.1, ALP), a hydrolytic enzyme acts optimally at alkaline pH, is formed in liver and is existed in almost all tissues of the body. Serum ALP measurements is commonly used for the investigation of hepatic disease. Under some conditions, such as gestation, budding children, high ALP activities are normal physiological phenomena. Pathologic high ALP activities may exist in hepatobiliary diseases, bone diseases, bone metastases and hyperparathyroidism. Decreased ALP activity occurs uncommonly and is only observed in about 0.2% old people.

### Assay Principle

International Federation of Clinical Chemistry and Laboratory Medicine (IFCC) modified method

By the action of ALP and magnesium ions, p-Nitrophenyl phosphate is catalyzed to p-Nitrophenol, and the absorbency increase is directly proportional to the activity of ALP.

### Reagents Components

|            |                                                          |            |
|------------|----------------------------------------------------------|------------|
| <b>R1:</b> | Na <sub>2</sub> -EDTA                                    | 150 mmol/L |
|            | Magnesium acetate                                        | 2 mmol/L   |
| <b>R2:</b> | Na <sub>2</sub> -EDTA                                    | 38 mmol/L  |
|            | Magnesium acetate                                        | 2 mmol/L   |
|            | 4-nitrophenyl phosphate, disodium salt·6H <sub>2</sub> O | 115 mmol/L |
|            | sodium azide                                             | 0.01%      |

**Storage and stability**

Up to expiration date indicated on the label, when stored unopened at 2-8°C and protected from light.

On board in use, the reagents are stable for 7 days when refrigerated on the analyzer.

Contamination must be avoided.

Do not freeze the reagent.

**Specimen collection and preparation****■ Specimen types**

Serum and lithium heparin plasma are suitable for samples.

**■ Preparation for Analysis**

1. Use the suitable tubes or collection containers and follow the instruction of the manufacturer; avoid effect of the materials of the tubes or other collection containers.
2. Centrifuge samples containing precipitate before performing the assay.
3. Specimens should be tested as soon as possible after sample collection and pre-analytical treatment.

**■ Sample Stability<sup>4</sup>**

7 days at 15-25°C

7 days at 2-8°C

2 months at (-25)-(-15)°C (Only frozen once)

For longer storage periods, samples should be frozen at (-20°C)<sup>5</sup>. Sample stability claims were established by manufacturer and/or based upon references, each laboratory should establish its own sample stability criteria.

**Reagent Preparation**

R1 and R2 are ready to use.

Please perform scheduled maintenance and standard operation including calibration and analysis to assure the performance of measurement system.

**Materials required but not provided**

1. General laboratory materials: NaCl solution 9 g/L (saline), distilled/deionized water.
2. Calibrator and Control: Please check the section of reagent instruction of Calibration and Quality Control.
3. Mindray BS series chemistry analyzers and General laboratory equipment.

**Assay procedure**

| Parameters Item                                                                                                                                             | BS-800 chemistry analyzers |
|-------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------|
| Assay type                                                                                                                                                  | Kinetic                    |
| Wavelength (Primary/Secondary)                                                                                                                              | 412/546 nm                 |
| Reaction direction                                                                                                                                          | Increase                   |
| R1                                                                                                                                                          | 200 $\mu$ L                |
| Sample or Calibrator                                                                                                                                        | 4 $\mu$ L                  |
| Mix, incubate at 37°C for 1-2 min, then add:                                                                                                                |                            |
| R2                                                                                                                                                          | 50 $\mu$ L                 |
| Mix thoroughly, after incubate at 37°C for 1 min,<br>measure the absorbance value continuously within further 3 min<br>Then calculate $\Delta A/\text{min}$ |                            |

Parameters may vary in different chemistry analyzers, may adjust in proportion if necessary. For Mindray BS series chemistry analyzers, Reagent Parameters is available on request. Please refer to the appropriate operation manual for the analyzers.

**Calibration**

1. It is recommended to use the Mindray Calibrator (Multi Sera Calibrator: 105-001144-00 or other suitable calibrators) and 9 g/L NaCl (saline) for two-point calibration. Traceability of the Mindray Multi Sera Calibrator can refer to the calibrator instructions for use of Mindray Company.

**2. Calibration frequency**

Calibration is stable for approximately 7 days on BS-800 chemistry analyzers. The calibration stability may vary on different instruments, each laboratory should set a calibration frequency in the instrument parameters appropriate to their usage pattern.

Recalibration may be necessary when the following occur:

- As changed reagent lot.
- As required following quality control procedures or out of control.
- As executes specific maintenance or troubleshooting procedure of chemistry analyzers.

3. The calibrator values are lot-specific with the matched models listed in the value sheet.

**Quality control**

1. It is recommended to use the Mindray Control (ClinChem Multi Control: 105-009119-00, 105-009120-00 or other suitable controls) to verify the performance of the measurement procedure; other suitable control material can be used in addition.

2. Two levels of control material are recommended to analyze each batch of samples. In addition, the control should be run with each new calibration, with each new reagent cartridge, and after specific maintenance or trouble shooting procedures as detailed in the appropriate system manual.
3. Each laboratory should establish its own internal quality control scheme and procedures for corrective action if control doesn't recover within the acceptable tolerances.

### Calculation

The BS series chemistry analyzer detects the change of absorbance ( $\Delta A/\text{min}$ ) and calculates the ALP activity of each sample automatically with a specified calibration curve from calibration process.

Conversion factor of conventional units (U/L) into S.I. units ( $\mu\text{kat/L}$ ):

$$1 \text{ U/L} = 16.67 \times 10^{-3} \mu\text{kat/L}$$

$$1 \mu\text{kat/L} = 60 \text{ U/L}$$

### Dilution

If the value of serum/plasma sample exceeds 800 U/L, the sample should be diluted with 9 g/L NaCl solution (saline) (e.g. 1+9) and rerun; the result should be multiplied by 10.

### Expected values

| Sample Type      |                       | Units                           |
|------------------|-----------------------|---------------------------------|
| Serum/<br>Plasma | Adults <sup>6</sup>   | Male: 45-125 U/L                |
|                  |                       | Female: 20-49 years: 35-100 U/L |
|                  |                       | 50-79 years: 50-135 U/L         |
|                  | 28 days - <6 months   | 98-532 U/L                      |
|                  | 6 months - <1 year    | 106-420 U/L                     |
|                  | 1 - <2 years          | 128-432 U/L                     |
|                  | 2 - <9 years          | 143-406 U/L                     |
|                  | 9 - <12 years         | 146-500 U/L                     |
|                  | Children <sup>7</sup> | Male: 160-610 U/L               |
|                  |                       | Female: 81-454 U/L              |
|                  | 14 - <15 years        | Male: 82-603 U/L                |
|                  |                       | Female: 63-327 U/L              |
|                  | 15 - <17 years        | Male: 64-443 U/L                |
|                  |                       | Female: 52-215 U/L              |
|                  | 17 - <18 years        | Male: 51-202 U/L                |
|                  |                       | Female: 43-130 U/L              |

The expected value is provided from reference, and Mindray has verified it by 482 serum samples of people from China.

Each laboratory should establish its own reference intervals based upon its particular locale and population characteristics since expected values may vary with geography, race, sex and age.

## Performance Characteristics

### ■ Analytical Sensitivity

The Alkaline phosphatase Kit has an analytical sensitivity of 5 U/L on BS-300. Analytical sensitivity is defined as the lowest concentration of analyte that can be distinguished from a sample that contains no analyte. It is calculated as the value lying 3 standard deviations above that of the mean from 20 replicates of an analyte-free sample.

### ■ Measuring range

The Mindray BS series systems provide the following linearity range:

| Sample Type  | Units     |
|--------------|-----------|
| Serum/Plasma | 5-800 U/L |

A high ALP activity sample (approximately 800 U/L) is mixed with a low activity sample (<5 U/L) at different ratios, generating a series of dilutions. The ALP activity of each dilution is determined using Mindray System, the linearity range is demonstrated with the correlation coefficient  $r \geq 0.990$ . The reportable range is 5-8000 U/L.

### ■ Precision

Precision was determined by following CLSI Approved Guideline EP05-A3<sup>8</sup>, each sample was assayed 2 times per run, 2 runs per day, a total of 20 days. The precision data of controls on BS-300 are summarized below\*.

| Specimen Type<br>(N=80) | Mean<br>(U/L) | Repeatability |      | Within-Lab |      |
|-------------------------|---------------|---------------|------|------------|------|
|                         |               | SD (U/L)      | CV % | SD (U/L)   | CV % |
| Control Level 1         | 193.19        | 0.24          | 0.12 | 0.54       | 0.28 |
| Control Level 2         | 385.54        | 0.32          | 0.08 | 0.84       | 0.22 |

\*Representative data, results in different instruments, laboratories may vary.

### ■ Analytical Specificity

The samples with different concentration interfering substance were prepared by addition of interferent to human serum pools, and recovers are within  $\pm 10\%$  of the corresponding control value to be considered as no significant interference.

No significant interference was observed when the following substances were tested for interference with this methodology. The data of interference studies on BS-300 are summarized below.

| Interfering Substance | Interferent Concentration (mg/dL) | Analyte activity (U/L) | Relative Deviation (%)* |
|-----------------------|-----------------------------------|------------------------|-------------------------|
| Ascorbic acid         | 30                                | 199.63                 | -0.35                   |
| Hemoglobin            | 500                               | 192.30                 | -2.72                   |
| Bilirubin             | 40                                | 201.90                 | +2.74                   |
| Intralipid            | 500                               | 192.80                 | +0.46                   |

\*Representative data, results in different instruments, laboratories may vary.

In very rare cases gammopathy, in particular type IgM, may cause unreliable results<sup>9</sup>.

### ■ Method Comparison

Correlation studies were performed using CLSI Approved Guideline EP09-A3<sup>10</sup>. The Mindray System (Mindray BS-2000/Mindray ALP Reagent) (y) was compared with comparison system (Roche cobas c701/ Roche ALP Reagent) (x) using the same serum specimens. The statistical data obtained by linear regression are shown in the table below \*:

| Regression Fit     | Correlation Coefficient (r) | Sample (N) | Activity Range (U/L) |
|--------------------|-----------------------------|------------|----------------------|
| $y=0.9809x-1.0972$ | 0.9999                      | 100        | 40.88-753.95         |

\*Representative data, results in different instruments, laboratories may vary.

### Result interpretation

The results could be affected by drugs, disease, or endogenous substances<sup>9,11</sup>. When the reaction curve is abnormal, it is recommended to retest and check the result.

### Warnings and precaution

1. For in vitro diagnostic use only. For laboratory professional use.
2. Please take the necessary precautions for handling all laboratory reagents.
3. Please confirm the integrity of the package before use. Do not use the kits with damaged packages. The reagents avoid direct exposure to sunlight and freezing. The results can't be assured when stored at inappropriate condition.
4. If unintentionally opened before used, store the reagents tightly capped at 2-8°C and protected from light, and the stability is equally to in-use stability.
5. Do not mix reagents with different lots and bottles.  
Do not use the reagents beyond the expiration date and the in-use date.

Do not mix fresh reagents with in-use reagents.

Avoid the formation of foam.

6. Instability or deterioration should be suspected if there are visible signs of leakage, precipitates or microbial growth, or if calibration or controls do not meet the insert and/or the Mindray System criteria.
7. Reliability of assay results cannot be guaranteed if the instructions in this package insert are not followed.
8. Preservative contained. Do not swallow. Avoid contact with skin and mucous membranes.
9. When the reagents accidentally enter the eyes and mouth, or contact with the skin, immediately wash with plenty of water. If necessary, visit the doctor for further medical treatment.
10. Safety data sheet is available for professional user on request.
11. Disposal of all waste material should be in accordance with local guidelines.
12. All human material should be considered potentially infectious.
13. All identified risks have been reduced as far as possible without adversely affecting the benefit-risk ratio, and the overall residual risk is acceptable.
14. Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.
15. This kit contains components classified as follows in accordance with the Regulation (EC) No 1272/2008:

### Warning

|      |                                      |
|------|--------------------------------------|
| H317 | May cause an allergic skin reaction. |
|------|--------------------------------------|

### Prevention:

|      |                                              |
|------|----------------------------------------------|
| P280 | Wear protective gloves, protective clothing. |
|------|----------------------------------------------|

|      |                                     |
|------|-------------------------------------|
| P261 | Avoid breathing mist/vapours/spray. |
|------|-------------------------------------|

|      |                                                                        |
|------|------------------------------------------------------------------------|
| P272 | Contaminated work clothing should not be allowed out of the workplace. |
|------|------------------------------------------------------------------------|

### Response:

|           |                                        |
|-----------|----------------------------------------|
| P302+P352 | IF ON SKIN: Wash with plenty of water. |
|-----------|----------------------------------------|

|           |                                                                  |
|-----------|------------------------------------------------------------------|
| P333+P313 | If skin irritation or rash occurs: Get medical advice/attention. |
|-----------|------------------------------------------------------------------|

|                  |                                                                                                                                  |
|------------------|----------------------------------------------------------------------------------------------------------------------------------|
| P362+P364        | Take off contaminated clothing and wash it before reuse.                                                                         |
| <b>Disposal:</b> |                                                                                                                                  |
| P501             | Dispose of contents/container to authorised hazardous or special waste collection point in accordance with any local regulation. |

## References

1. Thomas L. Clinical Laboratory Diagnostics: Use and Assessment of Clinical Laboratory Results. 1st ed. Frankfurt: TH-Books Verlagsgesellschaft 1998; 36-46.
2. Burtis CA, Ashwood ER, Bruns DE, eds. Tietz Textbook of Clinical Chemistry and Molecular Diagnostics, 5th ed. Elsevier Saunders 2012; 577-580.
3. McPherson RA, Pincus MR. Henry's Clinical Diagnosis and Management by Laboratory Methods, 22nd ed. Elsevier Saunders 2007;289-290.
4. Ehret W, Heil W, Schmitt Y, Töpfer G, Wisser H, Zawta B, et al. Use of Anticoagulants in Diagnostic Laboratory Investigations and Stability of Blood, Plasma and Serum Samples. WHO/DIL/LAB/99.1 Rev.2:21pp.
5. CLSI. Procedures for the handling and processing of blood specimens; Approved Guideline-Third Edition. CLSI document H18-A3. Wayne, PA: Clinical and Laboratory Standards Institute; 2004.
6. WS/T 404.1-2012. Reference intervals for common clinical biochemistry tests-Part 1: Serum alanine aminotransferase, aspartate aminotransferase, alkaline phosphatase and  $\gamma$ -glutamyltransferase. National Health Commission of the People's Republic of China, 2012.
7. WS/T 780-2021. Reference intervals of clinical biochemistry tests commonly used for children. National Health Commission of the People's Republic of China, 2021.
8. CLSI. Evaluation of Precision of Quantitative Measurement Procedures; Approved Guideline-Third Edition. CLSI document EP05-A3. Wayne, PA: Clinical and Laboratory Standards Institute; 2014.
9. Bakker AJ, Mucke M. Gammopathy interference in clinical chemistry assays: mechanisms, detection and prevention. Clin Chem Lab Med, 2007,45(9):1240-1243.
10. CLSI. Measurement Procedure Comparison and Bias Estimation Using Patient Sample; Approved Guideline-Third Edition. CLSI document EP09-A3. Wayne, PA: Clinical and Laboratory Standards Institute; 2013.
11. Young DS. Effects of Drugs on Clinical Laboratory Tests, 5th ed. Washington, DC: AACC Press; 2000:3-36,3-49.

## Graphical symbols

In Vitro Diagnostic  
medical device

Unique device  
identifier

European  
Conformity

Consult Instructions  
For use

Use-by  
date

Authorized representative in  
the European Community

Batch Code

Temperature  
limit

Manufacturer

Catalogue  
number

Keep away from sunlight

Indicates a medical device that needs protection from light sources

© 2022-2023 Shenzhen Mindray Bio-Medical Electronics Co., Ltd. All rights Reserved

**Manufacturer:** Shenzhen Mindray Bio-Medical Electronics Co., Ltd.

**Address:** Mindray Building, Keji 12th Road South, High-Tech Industrial Park, Nanshan, Shenzhen, 518057, P. R. China

**E-mail Address:** service@mindray.com

**Website:** www.mindray.com

**Tel:** +86-755-81888998; **Fax:** +86-755-26582680

**EC-Representative:** Shanghai International Holding Corp. GmbH(Europe)

**Address:** Eiffestraße 80, Hamburg 20537, Germany

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1. Thomas L. Clinical Laboratory Diagnostics: Use and Assessment of Clinical Laboratory Results. 1st ed. Frankfurt: TH-Books Verlagsgesellschaft 1998; 36-46.
2. Burtis CA, Ashwood ER, Bruns DE, eds. Tietz Textbook of Clinical Chemistry and Molecular Diagnostics, 5th ed. Elsevier Saunders 2012; 577-580.
3. McPherson RA, Pincus MR. Henry's Clinical Diagnosis and Management by Laboratory Methods, 22nd ed. Elsevier Saunders 2007; 289-290.
4. Ehret W, Heil W, Schmitt Y, Töpfer G, Wisser H, Zawta B, et al. Use of Anticoagulants in Diagnostic Laboratory Investigations and Stability of Blood, Plasma and Serum Samples. WHO/DIL/LAB/99.1 Rev.2:21pp.
5. CLSI. Procedures for the handling and processing of blood specimens; Approved Guideline-Third Edition. CLSI document H18-A3. Wayne, PA: Clinical and Laboratory Standards Institute; 2004.
6. WS/T 404.1-2012. Reference intervals for common clinical biochemistry

- tests-Part 1: Serum alanine aminotransferase, aspartate aminotransferase, alkaline phosphatase and  $\gamma$ -glutamyltransferase. National Health Commission of the People's Republic of China, 2012.
- 7.WS/T 780-2021. Reference intervals of clinical biochemistry tests commonly used for children. National Health Commission of the People's Republic of China, 2021.
- 8.CLSI. Evaluation of Precision of Quantitative Measurement Procedures; Approved Guideline-Third Edition. CLSI document EP05-A3. Wayne, PA: Clinical and Laboratory Standards Institute; 2014.
- 9.Bakker AJ, Mucke M. Gammopathy interference in clinical chemistry assays: mechanisms, detection and prevention. Clin Chem Lab Med, 2007;45(9):1240-1243.
- 10.CLSI. Measurement Procedure Comparison and Bias Estimation Using Patient Sample; Approved Guideline-Third Edition. CLSI document EP09-A3. Wayne, PA: Clinical and Laboratory Standards Institute; 2013.
- 11.Young DS. Effects of Drugs on Clinical Laboratory Tests, 5th ed. Washington, DC: AACC Press; 2000:3-36,3-49.

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**Referências**

1. Thomas L. Clinical Laboratory Diagnostics: Use and Assessment of Clinical Laboratory Results. 1st ed. Frankfurt: TH-Books Verlagsgesellschaft 1998; 36-46.
2. Burtis CA, Ashwood ER, Bruns DE, eds. Tietz Textbook of Clinical Chemistry and Molecular Diagnostics, 5th ed. Elsevier Saunders 2012; 577-580.
3. McPherson RA, Pincus MR. Henry's Clinical Diagnosis and Management by Laboratory Methods, 22nd ed. Elsevier Saunders 2007; 289-290.
4. Ehret W, Heil W, Schmitt Y, Töpfer G, Wisser H, Zawta B, et al. Use of Anticoagulants in Diagnostic Laboratory Investigations and Stability of Blood, Plasma and Serum Samples. WHO/DIL/LAB/99.1 Rev.2:21pp.
5. CLSI. Procedures for the handling and processing of blood specimens; Approved Guideline-Third Edition. CLSI document H18-A3. Wayne, PA: Clinical and Laboratory Standards Institute; 2004.
6. WS/T 404.1-2012. Reference intervals for common clinical biochemistry tests-Part 1: Serum alanine aminotransferase, aspartate aminotransferase, alkaline phosphatase and  $\gamma$ -glutamyltransferase. National Health Commission of the People's Republic of China, 2012.
7. WS/T 780-2021. Reference intervals of clinical biochemistry tests commonly used for children. National Health Commission of the People's Republic of China, 2021.
8. CLSI. Evaluation of Precision of Quantitative Measurement Procedures; Approved Guideline-Third Edition. CLSI document EP05-A3. Wayne, PA: Clinical and Laboratory Standards Institute; 2014.
9. Bakker AJ, Mucke M. Gammopathy interference in clinical chemistry assays: mechanisms, detection and prevention. Clin Chem Lab Med, 2007;45(9):1240-1243.
10. CLSI. Measurement Procedure Comparison and Bias Estimation Using Patient Sample; Approved Guideline-Third Edition. CLSI document EP09-A3. Wayne, PA: Clinical and Laboratory Standards Institute; 2013.
11. Young DS. Effects of Drugs on Clinical Laboratory Tests, 5th ed. Washington, DC: AACC Press; 2000:3-36,3-49.

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**Fabricante:** Shenzhen Mindray Bio-Medical Electronics Co., Ltd.

**Morada:** Mindray Building, Keji 12th Road South, High-Tech Industrial Park, Nanshan, Shenzhen, 518057, R. P. China

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**Referencias**

1. Thomas L. Clinical Laboratory Diagnostics: Use and Assessment of Clinical Laboratory Results. 1st ed. Frankfurt: TH-Books Verlagsgesellschaft 1998; 36-46.
2. Burtis CA, Ashwood ER, Bruns DE, eds. Tietz Textbook of Clinical Chemistry and Molecular Diagnostics, 5th ed. Elsevier Saunders 2012; 577-580.
3. McPherson RA, Pincus MR. Henry's Clinical Diagnosis and Management by Laboratory Methods, 22nd ed. Elsevier Saunders 2007; 289-290.
4. Ehret W, Heil W, Schmitt Y, Töpfer G, Wissner H, Zawta B, et al. Use of Anticoagulants in Diagnostic Laboratory Investigations and Stability of Blood, Plasma and Serum Samples. WHO/DIL/LAB/99.1 Rev.2:21pp.
5. CLSI. Procedures for the handling and processing of blood specimens; Approved Guideline-Third Edition. CLSI document H18-A3. Wayne, PA: Clinical and Laboratory Standards Institute; 2004.
6. WS/T 404.1-2012. Reference intervals for common clinical biochemistry tests-Part 1: Serum alanine aminotransferase, aspartate aminotransferase, alkaline phosphatase and  $\gamma$ -glutamyltransferase. National Health Commission of the People's Republic of China, 2012.
7. WS/T 780-2021. Reference intervals of clinical biochemistry tests commonly used for children. National Health Commission of the People's Republic of China, 2021.
8. CLSI. Evaluation of Precision of Quantitative Measurement Procedures; Approved Guideline-Third Edition. CLSI document EP05-A3. Wayne, PA: Clinical and Laboratory Standards Institute; 2014.
9. Bakker AJ, Mucke M. Gammopathy interference in clinical chemistry assays: mechanisms, detection and prevention. Clin Chem Lab Med, 2007, 45(9):1240-1243.
10. CLSI. Measurement Procedure Comparison and Bias Estimation Using Patient Sample; Approved Guideline-Third Edition. CLSI document EP09-A3. Wayne, PA: Clinical and Laboratory Standards Institute; 2013.
11. Young DS. Effects of Drugs on Clinical Laboratory Tests, 5th ed. Washington, DC: AACC Press; 2000:3-36,3-49.

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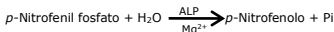
**Principio del test**

Metodo IFCC (International Federation of Clinical Chemistry and Laboratory Medicine) modificato

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## Bibliografia

1. Thomas L. Clinical Laboratory Diagnostics: Use and Assessment of Clinical Laboratory Results. 1st ed. Francoforte: TH-Books Verlagsgesellschaft 1998; 36-46.
2. Burtis CA, Ashwood ER, Bruns DE, eds. Tietz Textbook of Clinical

- Chemistry and Molecular Diagnostics, 5th ed. Elsevier Saunders 2012; 577-580.
3. McPherson RA, Pincus MR. Henry's Clinical Diagnosis and Management by Laboratory Methods, 22nd ed. Elsevier Saunders 2007;289-290.
  4. Ehret W, Heil W, Schmitt Y, Töpfer G, Wissner H, Zawta B, et al. Use of Anticoagulants in Diagnostic Laboratory Investigations and Stability of Blood, Plasma and Serum Samples. WHO/DIL/LAB/99.1 Rev.2:21pp.
  5. CLSI. Procedures for the handling and processing of blood specimens; Approved Guideline-Third Edition. CLSI document H18-A3. Wayne, PA: Clinical and Laboratory Standards Institute; 2004.
  6. WS/T 404.1-2012. Reference intervals for common clinical biochemistry tests-Part 1: Serum alanine aminotransferase, aspartate aminotransferase, alkaline phosphatase and  $\gamma$ -glutamyltransferase. National Health Commission of the People's Republic of China, 2012.
  7. WS/T 780-2021. Reference intervals of clinical biochemistry tests commonly used for children. National Health Commission of the People's Republic of China, 2021.
  8. CLSI. Evaluation of Precision of Quantitative Measurement Procedures; Approved Guideline-Third Edition. CLSI document EP05-A3. Wayne, PA: Clinical and Laboratory Standards Institute; 2014.
  9. Bakker AJ, Mucke M. Gammopathy interference in clinical chemistry assays: mechanisms, detection and prevention. Clin Chem Lab Med, 2007,45(9):1240-1243.
  10. CLSI. Measurement Procedure Comparison and Bias Estimation Using Patient Sample; Approved Guideline-Third Edition. CLSI document EP09-A3. Wayne, PA: Clinical and Laboratory Standards Institute; 2013.
  11. Young DS. Effects of Drugs on Clinical Laboratory Tests, 5th ed. Washington, DC: AACC Press; 2000:3-36,3-49.

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**Indirizzo:** Mindray Building, Keji 12th Road South, High-Tech Industrial Park, Nanshan, Shenzhen, 518057, Repubblica popolare cinese

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## Referanslar

- 1.Thomas L. Clinical Laboratory Diagnostics: Use and Assessment of Clinical Laboratory Results. 1st ed. Frankfurt: TH-Books Verlagsgesellschaft 1998; 36-46.
- 2.Burtis CA, Ashwood ER, Bruns DE, eds. Tietz Textbook of Clinical Chemistry and Molecular Diagnostics, 5th ed. Elsevier Saunders 2012; 577-580.
- 3.McPherson RA, Pincus MR. Henry's Clinical Diagnosis and Management by Laboratory Methods, 22nd ed. Elsevier Saunders 2007;289-290.
- 4.Ehret W, Heil W, Schmitt Y, Töpfer G, Wisser H, Zawta B, et al. Use of Anticoagulants in Diagnostic Laboratory Investigations and Stability of Blood, Plasma and Serum Samples. WHO/DIL/LAB/99.1 Rev.2:21pp.
- 5.CLSI. Procedures for the handling and processing of blood specimens; Approved Guideline-Third Edition. CLSI document H18-A3. Wayne, PA: Clinical and Laboratory Standards Institute; 2004.
- 6.WS/T 404.1-2012. Reference intervals for common clinical biochemistry tests-Part 1: Serum alanine aminotransferase, aspartate aminotransferase, alkaline phosphatase and  $\gamma$ -glutamyltransferase. National Health Commission of the People's Republic of China, 2012.
- 7.WS/T 780-2021. Reference intervals of clinical biochemistry tests commonly used for children. National Health Commission of the People's Republic of China, 2021.
- 8.CLSI. Evaluation of Precision of Quantitative Measurement Procedures; Approved Guideline-Third Edition. CLSI document EP05-A3. Wayne, PA: Clinical and Laboratory Standards Institute; 2014.
- 9.Bakker AJ, Mucke M. Gammopathy interference in clinical chemistry assays: mechanisms, detection and prevention. Clin Chem Lab Med,

2007,45(9):1240-1243.

- 10.CLSI. Measurement Procedure Comparison and Bias Estimation Using Patient Sample; Approved Guideline-Third Edition. CLSI document EP09-A3. Wayne, PA: Clinical and Laboratory Standards Institute; 2013.
- 11.Young DS. Effects of Drugs on Clinical Laboratory Tests, 5th ed. Washington, DC: AACC Press; 2000:3-36,3-49.

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**Üretici:** Shenzhen Mindray Bio-Medical Electronics Co., Ltd.

**Adres:** Mindray Building, Keji 12th Road South, High-Tech Industrial Park, Nanshan, Shenzhen, 518057, P. R. Çin

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**Références**

1. Thomas L. Clinical Laboratory Diagnostics: Use and Assessment of Clinical Laboratory Results. 1<sup>re</sup> éd. Francfort : TH-Books Verlagsgesellschaft 1998; 36-46.
2. Burtis CA, Ashwood ER, Bruns DE, eds. Tietz Textbook of Clinical Chemistry and Molecular Diagnostics, 5th ed. Elsevier Saunders 2012; 577-580.
3. McPherson RA, Pincus MR. Henry's Clinical Diagnosis and Management by Laboratory Methods, 22nd ed. Elsevier Saunders 2007; 289-290.
4. Ehret W, Heil W, Schmitt Y, Töpfer G, Wisser H, Zawta B, et al. Use of Anticoagulants in Diagnostic Laboratory Investigations and Stability of Blood, Plasma and Serum Samples. WHO/DIL/LAB/99.1 Rev.2:21pp.
5. CLSI. Procedures for the handling and processing of blood specimens; Approved Guideline-Third Edition. CLSI document H18-A3. Wayne, PA: Clinical and Laboratory Standards Institute; 2004.
6. WS/T 404.1-2012. Reference intervals for common clinical biochemistry tests-Part 1: Serum alanine aminotransferase, aspartate aminotransferase, alkaline phosphatase and  $\gamma$ -glutamyltransferase. National Health Commission of the People's Republic of China, 2012.
7. WS/T 780-2021. Reference intervals of clinical biochemistry tests commonly used for children. National Health Commission of the People's Republic of China, 2021.
8. CLSI. Evaluation of Precision of Quantitative Measurement Procedures; Approved Guideline-Third Edition. CLSI document EP05-A3. Wayne, PA: Clinical and Laboratory Standards Institute; 2014.
9. Bakker AJ, Mucke M. Gammopathy interference in clinical chemistry assays: mechanisms, detection and prevention. Clin Chem Lab Med, 2007;45(9):1240-1243.
10. CLSI. Measurement Procedure Comparison and Bias Estimation Using Patient Sample; Approved Guideline-Third Edition. CLSI document EP09-A3. Wayne, PA: Clinical and Laboratory Standards Institute; 2013.
11. Young DS. Effects of Drugs on Clinical Laboratory Tests, 5th ed. Washington, DC: AACC Press; 2000:3-36,3-49.

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© 2022-2023 Shenzhen Mindray Bio-Medical Electronics Co., Ltd. Tous droits réservés
**Fabricant** : Shenzhen Mindray Bio-Medical Electronics Co., Ltd.
**Représentant en Europe** : Shanghai International Holding Corp. GmbH (Europe)