



ugy/m²	-
dgy/cm²	-
Language	Applies DICOM Language (16 languages).
Default	Not applying DICOM Language Encoding .
Status	Checks the processing condition of image exporting and its result.
<div>  <ul style="list-style-type: none"> The following patient information is processed to be anonymous. <ul style="list-style-type: none"> Patient ID, Patient Name, Other Patient ID, Other Patient Name, Patient Address, Patient Telephone Number The following hospital information is processed to be anonymous. <ul style="list-style-type: none"> Institution Name, Institution Address, Referring Physician Name, Performing Physician Name, Operator Name </div>	

8.12.2 DICOM Setting Options for Exporting Images

Item	Description
Include Portable Viewer	Sets whether to include DICOM Portable Viewer . <ul style="list-style-type: none"> This item activates only by checking DICOM or DICOMDIR.
Compression	Sets JPEG2000 compression way of DICOM image. <ul style="list-style-type: none"> This item activates only by checking DICOM or DICOMDIR.
No Compression	Not apply the JPEG2000 compression way.
JPEG2000 Lossless	Compress the DICOM image as JPEG2000 without a loss.
JPEG2000 Lossy	Compress the DICOM image as JPEG2000 with a loss.
Compression Quality	You can set the JPEG2000 compression ratio of the DICOM image. <ul style="list-style-type: none"> This item activates only when selecting JPEG2000 Lossy Compression. You can input the compression ratio directly, or control the slide bar.
	<ul style="list-style-type: none"> The image can be damaged seriously when the compression ratio of JPEG2000 Lossy is higher, and this can affect to the image quality.