


NOTE: If the probe cable slackens, adjust it to prevent it from being caught between the casters.

5. Fix the monitor in place.

NOTE: This prevents the monitor from moving while the system is moved.


For a 22-inch monitor (OLED):

- a. Push Arm #1 to the rear.
- b. Adjust Arm #3 and the base so that they are parallel.

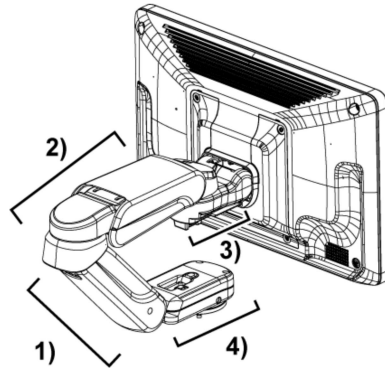


- 1) Arm #1
- 2) Arm #2
- 3) Arm #3
- 4) Base

- c. Tilt the monitor so that it faces upward.
- d. Lower Arm #2, and insert a lock block.
→ Arm #3 locks. When this happens, you will hear a click.

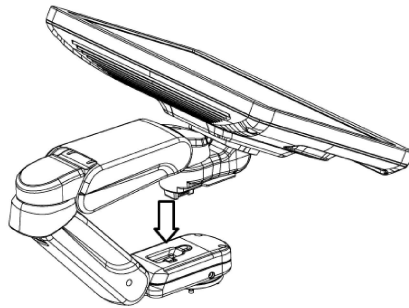


- e. Rotate Arm #1 so that it appears straight when viewed from the front of the system.
→ Arm #1 locks. When this happens, you will hear a click.

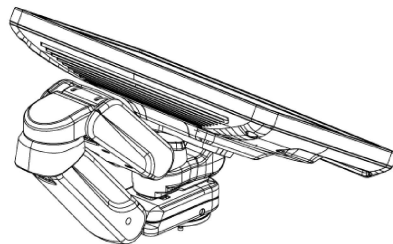

For a 23-inch monitor (LCD):

- a. Adjust Arm #3 and the base so that they are parallel.

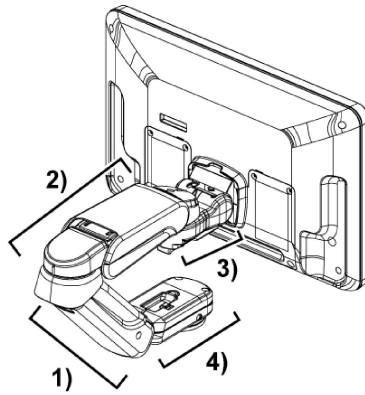


- 1) Arm #1
- 2) Arm #2
- 3) Arm #3
- 4) Base

- b. Tilt the monitor so that it faces upward.


- c. Lower Arm #2, and insert a lock block.

→ Arm #3 locks. When this happens, you will hear a click.

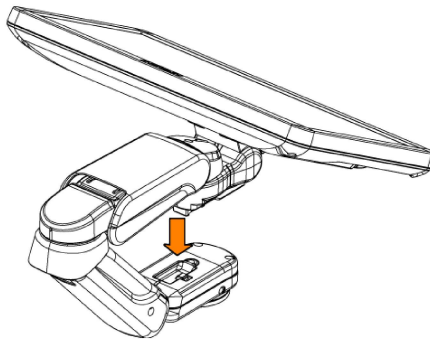

- d. Rotate Arm #1 so that it appears straight when viewed from the front of the system.

→ Arm #1 locks. When this happens, you will hear a click.

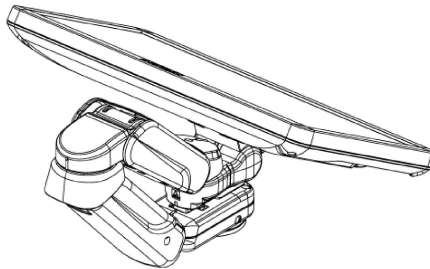


For a 21.5-inch monitor (LCD) :

- a. Turn the lock lever while pressing it to lift the lock arm.


- b. Face Arm #1 to the front.

→ Arm #1 is locked.

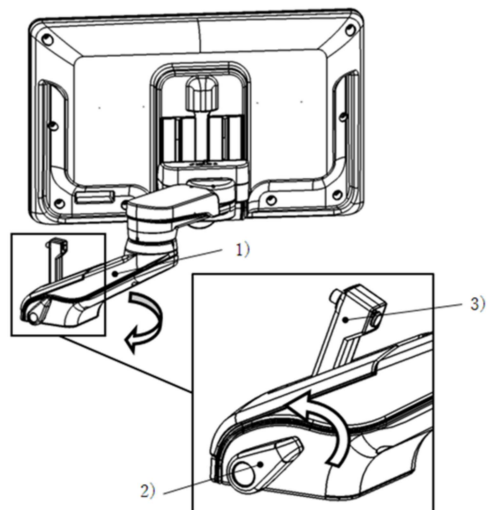


- 1) Arm #1
- 2) Lock lever
- 3) Lock arm

- c. Align the position of Arm #2 and Arm #3 to the position of the mark so that they are parallel to Arm #1.

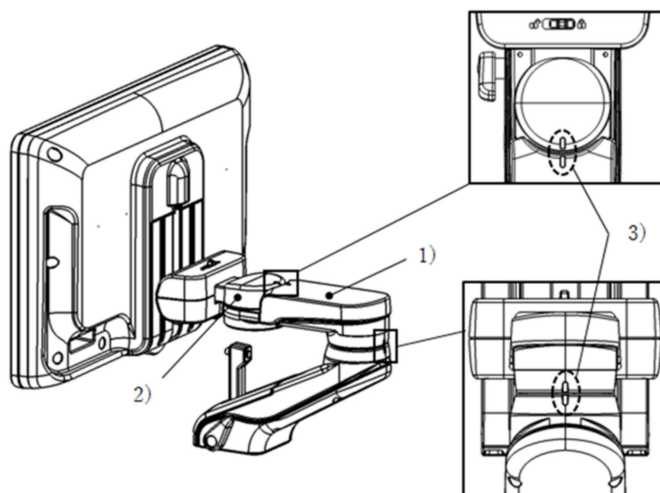

- 1) Arm #2
- 2) Arm #3
- 3) Mark


- d. Move the monitor to the highest position.

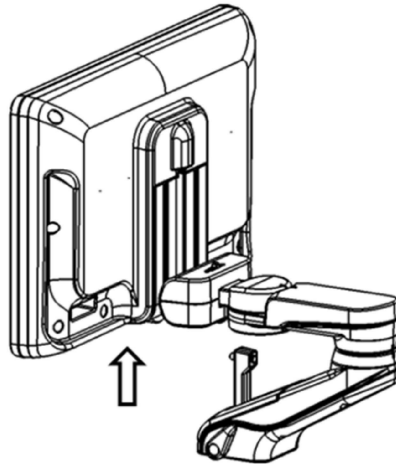

- e. Slide the tilt lever to tilt the monitor forward.

If the position of the lock is not aligned with the position of lock receiver:


Adjust the position of the monitor so that the pin on the lock arm is inserted into the lock receiver.

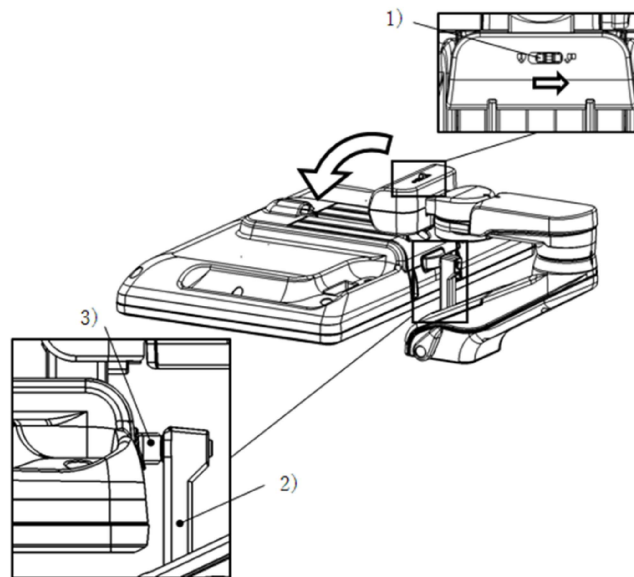

- 1) Tilt lever
2) Lock arm
3) Lock receiver

6. Unlock the casters.


Depress the caster locks for the front wheels. Depress the lock release pedals for the rear wheels.

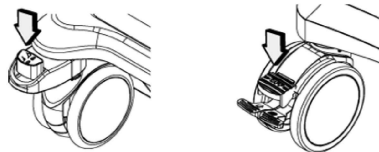

Left: Front wheel
Right: Rear wheel

Step in the direction of the arrow to release the casters.

7. To move the system, firmly grasp the handle on the back of the system with both hands.
NOTE: Grasp the handle on the back of the system, not the operation panel or its handle.
NOTE: Do not put anything on top of the monitor.

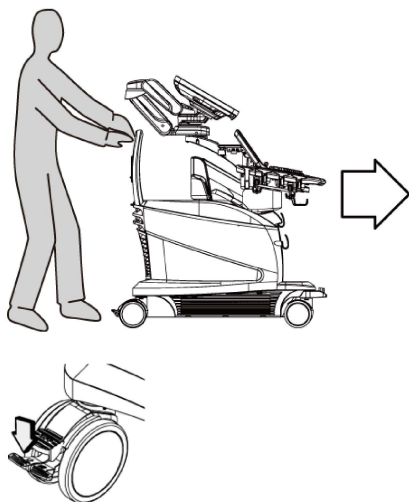

When transporting the system over long distances or up and down slopes:
Depress the swing lock pedals for the rear wheels in the direction of the arrow.

3.1.4 Installing the system

DANGER



Do not use this system in a flammable atmosphere.
Use of this system in a flammable atmosphere might cause an explosion.

CAUTION



When using this system together with other medical electrical devices, position the system and its cables (probe cables, ECG cables, I/O cables, etc.) as far away as possible from other medical electrical devices and their cables.

Note that electromagnetic interference generated by this system might prevent the normal operation of other medical electrical devices that are used together with the system. If such interference occurs, immediately stop using the devices together.

Procedure

1. At the installation location, make fine adjustments to the position of the system.
2. Once the position and orientation of the system are fixed, lock the casters.
Depress the caster locks for the front wheels. Depress the lock pedals for the rear wheels.