


3.4 Detent mode


This function is to stop the X-ray tube assembly at the detent position in longitudinal/transversal direction. Detent position should be set by installing.

Also, a function to stop at 0° , $\pm 90^\circ$ and -180° only in detent mode can be set by installing.

3.4.1 Moving to the longitudinal/transversal detent positions

3

Press the  (detent) button to operate.

- 1 Press the  button on the operation panel.

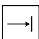



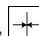

 light up and the detent mode will be activated.



Figure 3.10 Detent button

- 2 While pressing the , , or  button on the operation panel, hold the grip and move the X-ray tube assembly.

When the X-ray tube assembly reached the longitudinal detent position,  lights up and the assembly stops.

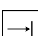
When the X-ray tube assembly reached the transversal detent position,  lights up and the assembly stops.

To move the assembly from the detent position, lift your finger off the release buttons once and press again to release the lock.



The assembly may pass the detent position when it moves too fast. In such case, try the operation slowly again.

- 3 Press  button.

Detent mode is deactivated, and  lights off.