



AKADENT EXTRA - USER MANUAL

AKADENT EXTRA PRECLEANING SOLUTION AND DISINFECTANT

- **CLEANING AND DISINFECTION**
- **SUIT FOR MANUAL, ULTRASONIC AND ENDOSCOPE WASHERS**

Product Properties

- Effects in 15 min at %0,5 concentration
- Economic due to very low usage concentration
- Effective cleaning of instruments and endoscopes
- Has corrosion inhibitors

Usage Areas

- It is used for pre-cleaning and disinfection of general surgical equipments, endoscopes and anesthesia equipments etc.

Usage form

- Add 5 ml product into 1 lt water.
- Immerse instruments and wait during 15 min. If possible, wash a little bit instruments before immersing.
- After effect time, take them out and rinse with plenty of water.

Effect time: 15 min.

Microbiological efficiency:

%0,5 concentration in 15 min; Bactericidal, fungicidal, tuberculocidal

%0,25 concentration in 15 min; Virucidal

%1 concentration in 15 min; Sporocidal

Preparation of Product Concentration

WATER AMOUNT	%0,5 SOLUTION	EFFECT TIME
1 L	5 ml	15 min.
5 L	25 ml	15 min.
10 L	50 ml	15 min.
15 L	75 ml	15 min.
20 L	100 ml	15 min.

Ingredients

Quarternary ammonium compounds, Amine groups, Auxiliary substances

First aid informations:

- Eyes; In the case of eye contact, wash plenty of water around 5 min.
- Skin: Wash your skin immediately.
- Ingestion: Drink water and apply your doctor.

DETRO HEALTHCARE KİMYA SANAYİ A.Ş.

☎ 212 659 77 62 📞 212 659 77 63

✉ info@detrox.com.tr – www.detrox.com.tr