Proxima v

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Compact Solution

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ProXima V is a modular x-ray system with the standard configuration of floor-mounted tube stand, vertical bucky stand and bucky table. The positioning features, both manual and automatic, enable such a high workflow to make ProXima V the best choice in terms of price-performance ratio.

Proxima V is a compact system suitable to be installed in small environments. It can be configured in a ultra-basic configuration without table and with stretcher, or with fixed height floating table, or with elevating height floating table or with OPEN model, elevating height floating table with extended longitudinal movement of 120 cm.

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Smart buckies are available on both table and vertical stand, with the possibility to rotate and remove the grid. ProXima V supports both wifi or fixed detectors of 35x43 cm and 43x43cm.

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ProXima V can automatically position itself according to the exam and the projection to make. The automatic positioning control can be activated from the console or from a dedicated remote control or from the tube side console.

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Automatic tracking and keeping of focal distance

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X-ray tube and digital flat panel detectors are kept automatically aligned for faster and more correct exam execution, even during oblique projections. At the same time, selected SID is automatically maintained also during table elevation. Once the exam, the patient size and the SID are selected, the collimator is automatically set.

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With the optional stitching package, it is possible to acquire full-spine and full-leg images. Images are acquired through a fully automatic and combined movement of tube and detector. The series of 2,3 or 4 images are then automatically stitched into one single image by the processing software.

Stitching procedure is quite fast to minimize any patient's movement. A wheeled patient stabilization tool with radio-opaque ruler is supplied as well.

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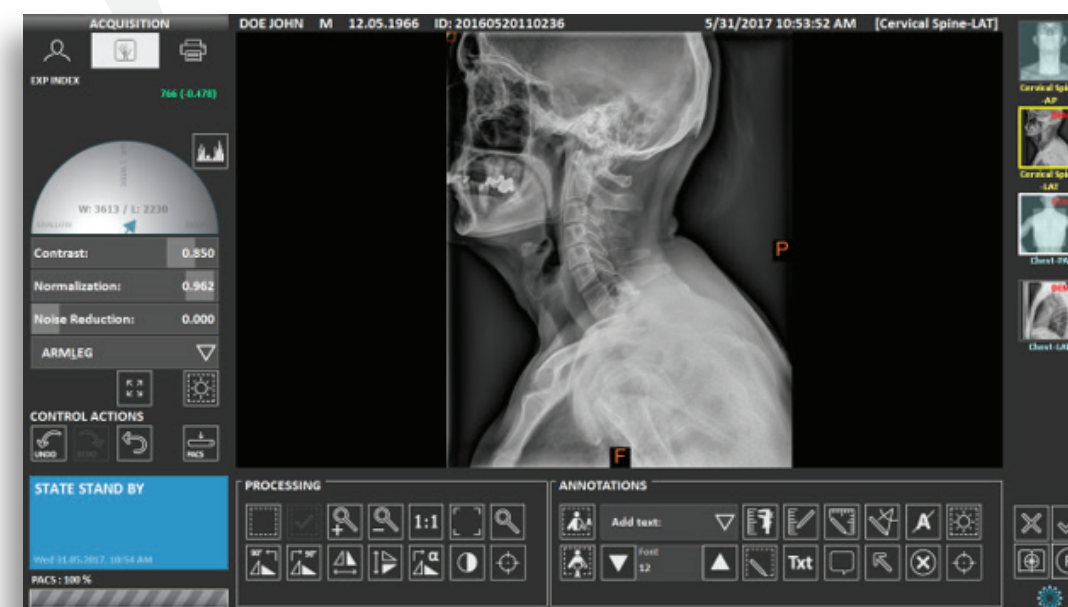
*High-automated integrated
management console*

*Control of the stands,
generator setting, collimator
opening, grids on/off,
detector orientation.*

*Images acquisition and
processing console*

*Simple and intuitive GUI,
exam is performed with two
clicks only.*

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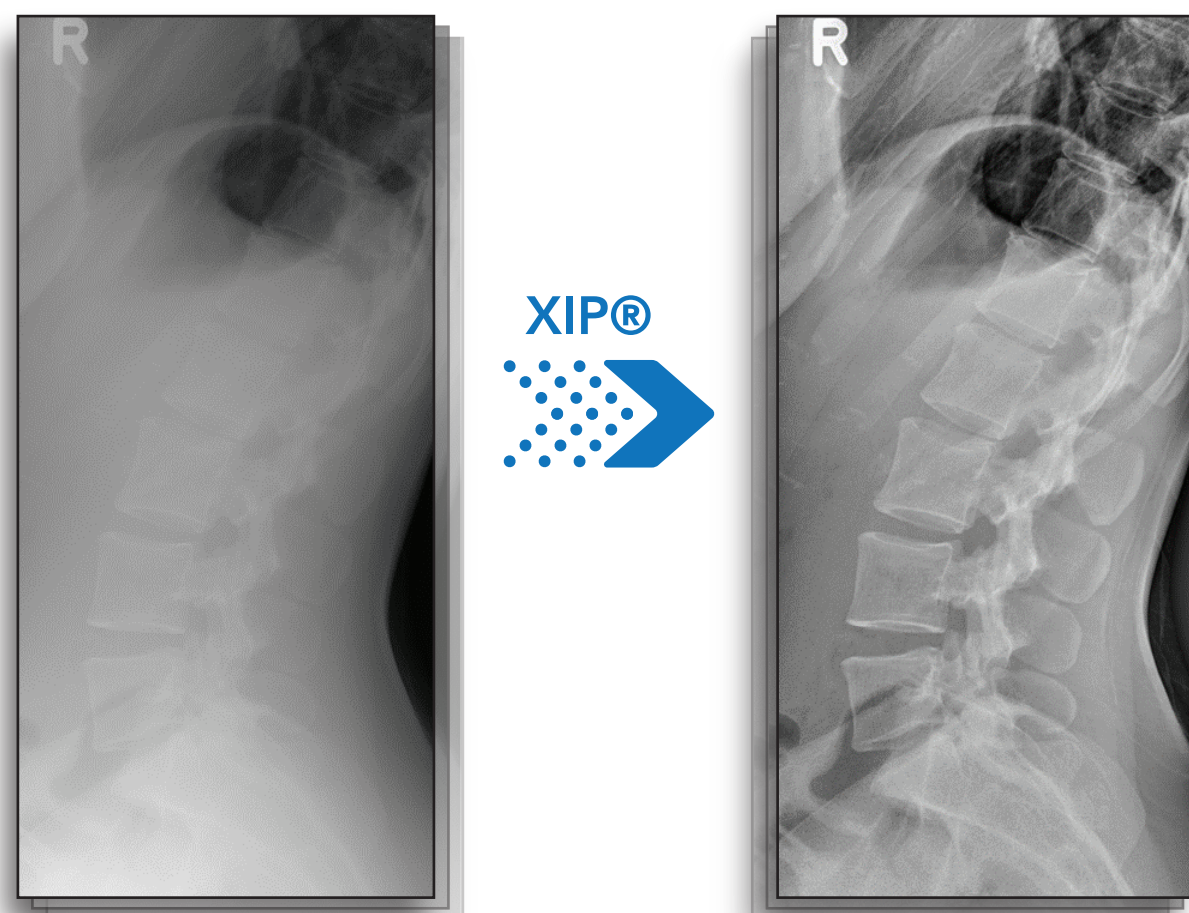
The XIP® algorithm automatically optimizes acquired images based on exam type and anatomical region. The software enhances the content details in both high attenuation image areas (shoulders, abdomen) and low attenuation images areas (lungs, cavities).

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The Avanse DR software includes a full dicom package whici is compatible with any RIS, PACS or Dicom Priter. The daily worklist is automatically downloaded from the RIS. The avanse DR software automatically sets all parameters for the selected patient. Once the study is completed, it can be sent to any PACS or DICOM Printer.

Radiographic Images

Dicom Worklist / MPPS
Dicom Query / Retrieve
CD/DVD Drive

Dicom DoseSR
CD/DVD Burner
DVD Recorder
PACS Network
HIS/RIS
Dicom Send
Dicom Store Commitment
Connection to Dicom Laser Camera
Remote Diagnosis

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Proxima Family

Proxima **c**

Proxima **U**

Proxima **v**

ASSING SpA

Via E. Amaldi 14
00015 Monterotondo (RM), Italy
Ph. +39 06 906701
F. +39 06 90670200
sede@assing.it

www.assingmedical.it