

Operating Instruction for Type YD01A(LED)
Examination Lamp

YD01A(LED) Examination Lamp

I .Brief description

YD01A(LED) Examination Lamp gathers and enlarges through optics, making common of giving out light effect to become clearly and light spot even high-quality light flat surface effect. It is extensively applied in the industry, medical treatment, jewelry, photograph. scientific research etc., It is currently more advanced lighting equipments at home and abroad.


II .Characters

1. Small and varies, it is apply in many circumstance.
2. Low voltage, low power, soft and saving energy.
3. Light intensity high and life span is long.

III.Basic parameters

1. Voltage: AC 100V-240V,50Hz-60Hz.
2. Lamp power: 3.3V/3W.
3. Color temperature: 4,800K \pm 200K.
4. Property: AC.
5. Life: 50,000hours.
6. Illuminance: 35,000Lx 0.5m/10,000Lx 1m.
7. High and low many adjust the altitude.
8. Adjust the light.

IV.Basic component

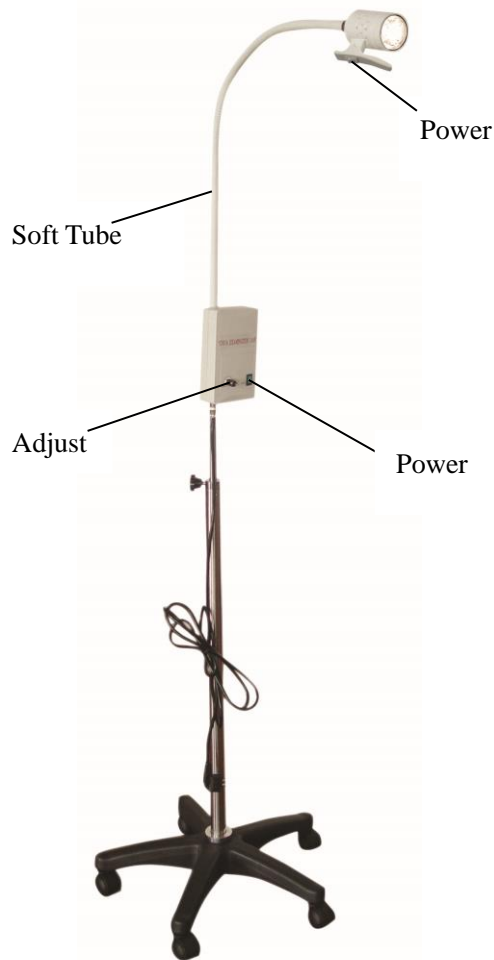

V.Floor style Examination Lamp YD01A(LED)

1. Open package, take out of clip tongs, basic component and power supply wire.
2. Please assemble as per diagram.
3. Make sure whether power supply voltage is the same with net voltage or not.
4. Insert the hook of the basic component to hang slot inside, screw the fix bolt tightly, make sure it doesn't loosen phenomenon.
5. Connect power wire to power box and net power supply.
6. Open the switch of power supply, Examination Lamp begin to work.

VI.Notice

1. Make sure whether power supply is the same with net power supply or not.
2. It doesn't bend soft tube on work.
3. It shouldn't touch the top lamp covering on work so that prevent burning,
4. The lens should be keep clearly, without dust, usually clear surface and lens.

VII.Replace lamp

1. Take off the underneath lamp convering and then take off the spring coil so that replace of the old lamp.
2. Make sure voltage and power of the new lamp is the same old lamp.
3. Insert new lamp into porcelain and fix the spring coil.
4. Screw underneath lamp covering, turn on the power supply switch.
5. YD01A(LED) without replace lamp.

VIII.Problem exclusion

1. Turn off the power supply switch, check the lamp whether loose or not, if loose, please assemble again.
2. Turn off the power supply, take off lamp and diagnose lamp whether problem or not, if lamp has problem, please replace of new lamp.
3. There are no problems of those and voltage is normal but lamp isn't work, please notice enterprise to check and maintain.

IX.Maintain and service

We promise to provide warranty within 12 months.