# USER MANUAL

## OPERATING TABLES ACCESSORIES

IV Stand, 2 Hooks

Arm Support, Single

Irrigation Bowl

Anesthesia Screen, Fixed Size

Body Fixing Strap

Proctology Set

Clamp

Knee Arthroscopy Support

Shoulder Support

INSPIRAL

## Copyright

This document has been prepared by INSPITAL Medikal Teknoloji. A.Ş. INSPITAL reserves the right to make changes in the technical specifications of the product. It is forbidden to reproduce or copy any part of this document without company's written permission.

## CE Declaration of Conformity

This device conforms to the requirements specified in TS EN 60601-2-46 "General Rules for the Safety of Operating Tables" and MDD/93/42/EEC Medical Devices Directive.

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E-Mail: posta@kiwa.com

## Patent

Trade name and brand qualification are held by INSPITAL A.Ş.

## Product Lifespan

The product lifespan of the accessories as determined by the Ministry of Industry and Trade is 10 years.

## Guarantee Period

INSPITAL brand operating table accessories, including all parts, are under warranty for 2 (two) years against material defects, faulty workmanship and manufacturing defects provided that they are used as shown in user manuals. However, the following are not under warranty:

1. Damages and failures that result from misuse,
2. Damages and failures that occur during loading & unloading and transportation after the delivery of the product to the customer,
3. Failures that result from using the product in contravention of the matters contained in user manuals.

Thank you for purchasing INSPITAL brand operating table accessories. Our product has been meticulously designed and produced for offering quality and performance to you in the best possible way. Please carefully read all information about the operation and safety of the operating table accessories.

This manual explains the steps of Safety and Cleaning for INSPITAL brand operating table accessories.

## **Points to Consider During Handling and Transportation**

Please follow the instructions on equipment boxes during handling and transportation.

## **Matters Related to Maintenance and Repair**

Please call the authorized service of the equipment in case of situations that require Maintenance and Repair.

## **Information about Connection and Assembly**

Please refer to the accompanying documents for connection and assembly information.

## **In order to use and service the operating table accessories by ensuring patient and user safety;**

- Make sure that both this user manual and its warnings and explanations have been read and understood before servicing the operating table accessories.
- The buyer is responsible for ensuring that the staff in charge of using or servicing the accessories is informed about the content of this user manual.
- Keep the user manual readily available to the staff in charge of using or servicing the accessories.
- Use the table only for the purposes explained in this manual.
- INSPITAL R&D Department reserves the right to change and/or abolish the technical specifications of the device and the information contained in this manual.

## TRANSPORTATION AND STORAGE ENVIRONMENT

### In Transportation and Storage:

- Ambient temperature should be between -15°C and + 40°C.
- Ambient humidity should be between 30% and 70%.
- Atmospheric pressure should be between 0.49 atm and 1 atm.

## STORAGE CONDITIONS

**The operating table accessories dispatched as packaged in a special cardboard box. Upon receiving the accessories ;**

- Open the package.
- Carefully remove the nylon bag.
- Take the accessories box and carefully open the it.
- Carefully unpack the accessories.
- Carefully examine the accessories for the purpose of determining whether or not they were damaged during transportation. For the table accessories that were damaged during transportation, consult the relevant point of sale.

## WORKING CONDITIONS

- Ambient temperature should be between 0°C and + 50°C.
- Ambient humidity should be between 30% and 70%.
- Atmospheric pressure should be between 0.49 atm and 1 atm.

## IMPORTANT SYMBOLS AND EXPLANATIONS

Follow instructions for use.

Date of manufacture

Requires Special Cleaning

Catalogue number

Manufacturer

Keep away from sunlight.

Batch Number

Caution: Consult accompanying documents

Keep away from rain.

## OVERVIEW

Thanks to the wide range of INSPITAL brand Operating Table Accessories, operating tables can be used safely in various surgical operations.

## INTENDED USE AND AREAS OF USE

### Intended Use

- Operating table accessories are used to provide easier access to the surgical site. They are used in operating rooms where surgical operations are performed.
- They can be safely used in various surgical operations.

### Intended User

- Operating tables accessories are used by doctors and healthcare personnel.

### Side Effects

- If the user manual is followed, the accessories do not have a defined side effect.

### Target Population

- It is suitable for patients from all ages.

### Indications

- It has no defined indications. The operating table accessories has no therapeutic feature. It is intended to support treatment.
- Operating tables accessories are used in surgical operations especially in General Surgery, Orthopedics, Gynecology, Urology, Pediatric Surgery, Cardiovascular Surgery, Brain Surgery, Thoracic Surgery, ENT, Eye, Plastic Surgery and all surgical branches and interventional applications.

### Contraindications

- It has no defined contraindications.

## CLEANING

The operating tables accessories are specific to each surgery. Each accessory is subject to the maintenance and cleaning instructions of the operating table. The accessories are important parts in the operating rooms. For this reason, when you are cleaning these parts, please be extra careful and sensitive.

**Responsible Person:** Operating room staff

### **Likely Cleaning List:**

- Disinfectants containing alcohol are suitable for cleaning.
- Never use steel wool or steel wire for cleaning.
- For cleaning, do not use clothes that are fibrous or contain solid particles
- Do not clean with hard water.
- Avoid the over-contact of sterilizing solutions with the equipment.
- Avoid using chemicals such as bleach, hydrochloric acid, etc.
- Highly used bleacher (NaOCl) can damage the device.
- Do not use corrosive products for cleaning.
- Do not use solvents for cleaning the operating table accessories.
- Avoid cleaning with cleaning agents that contain chloride.
- Dilute the solutions that are used for operating rooms cleaning and disinfection at required rates for using in the stainless steel parts. Wipe the stainless steel with the help of a soft and wet cloth and then dry it so that no stain is left.
- Remove the rough dirt of the liquids (blood, body fluid, etc.) dripping or splashing onto the surfaces with a wet swab.
- If it is possible, prepare separate cleaning solutions for each accessories.
- The accessories should be cleaned after each surgery.
- When you are cleaning the accessories, please pay attention to all surfaces of the mattress in detail. Especially, clean the all indentations of the accessories.

## CAUTION!

Operating tables accessories should be disinfected after each incident to prevent contamination.

### **Stainless Steel Cleaning Application:**

- Compared to other materials, stainless steel is easier to clean and requires disinfectants in lower concentrations for its cleaning. Stainless steel can be cleaned better than many polymers and other metals. Thanks to its smooth surface, stainless steel makes it difficult for bacteria and other microorganisms to adhere onto the surface and to survive.
- Stainless steel equipment should be regularly cleaned so that they maintain their aesthetic appearance and corrosion resistance. Routine cleaning to be performed by employing proper methods shall ensure the use for a longer time of such equipment without loss of their visual characteristics.
- The most suitable method for cleaning stainless steel equipment is to decontaminate with lukewarm water or soap-added lukewarm water or ammonium-added (1%) water with the help of a soft cloth and then to dry.
- In the use of steel cream that is manufactured specially for cleaning stainless steel, make sure to rinse and dry well. For cleaning your stainless steel equipment, you may also use INSPITAL Spray Disinfectant.

**NOTE:** If you request INSPITAL Spray disinfectant, contact your sales consultant.

Besides, stainless steel should be cleaned by choosing the method suitable for the substance that causes contamination.

- Remove the dirt on the accessory with mild detergent-added lukewar water or with just lukewarm water with the help of a soft cloth or a soft sponge. In cleaning deep dirt, vinegar in small quantities can also be added to water.
- Following cleaning, rinse with hot fresh water again with the help of a clean cloth and then dry with a clean cloth.
- Remove the finger prints and stains on the equipment with chloride-free glass cleaners or sodium carbonate added to water in small quantities with the help of a clean cloth. After cleaning, make sure to rinse with lukewarm water again with the help of a clean cloth and then dry.

**CAUTION!**

Avoid the contact of stainless steel equipment with acids.

- For the oil stains on the equipment, first remove the oil that has contaminated the equipment with a soft cloth or paper towel. Following pre-cleaning with ammonium solution or detergent (non-scratching) in lukewarm water, perform routine cleaning operations.
- It is possible to use also organic solutions for removing oil stains. However, make sure that such solutions do not contain chlorine, acetone, methyl alcohol and mineral spirits.
- In case the equipment contacts with acid, immediately deacidify and rub with ammonium- or sodium bicarbonate-containing solution and then dry.

**CAUTION!**

Stainless steel may be scratched in case of contact with hard materials. Prevent the stainless steel part of the operating table accessories from getting in contact with hard materials.

## RECYCLING

If you shall dispose of an operating table accessories or replace any part of it, check the recyclability of each part. Many of the metals used in the operating table accessories are stainless steel.

Please implement the recycling procedure related to stainless steel.

For more information about recycling, please get in contact with relevant institutions and foundations or visit the websites that give information about recycling.

## AUTHORIZED SERVICES

**Ankara – Headquarters / Production Plant**

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