


2.2 Operation Panel

2.2.1 Operation Panel for Tube Stand

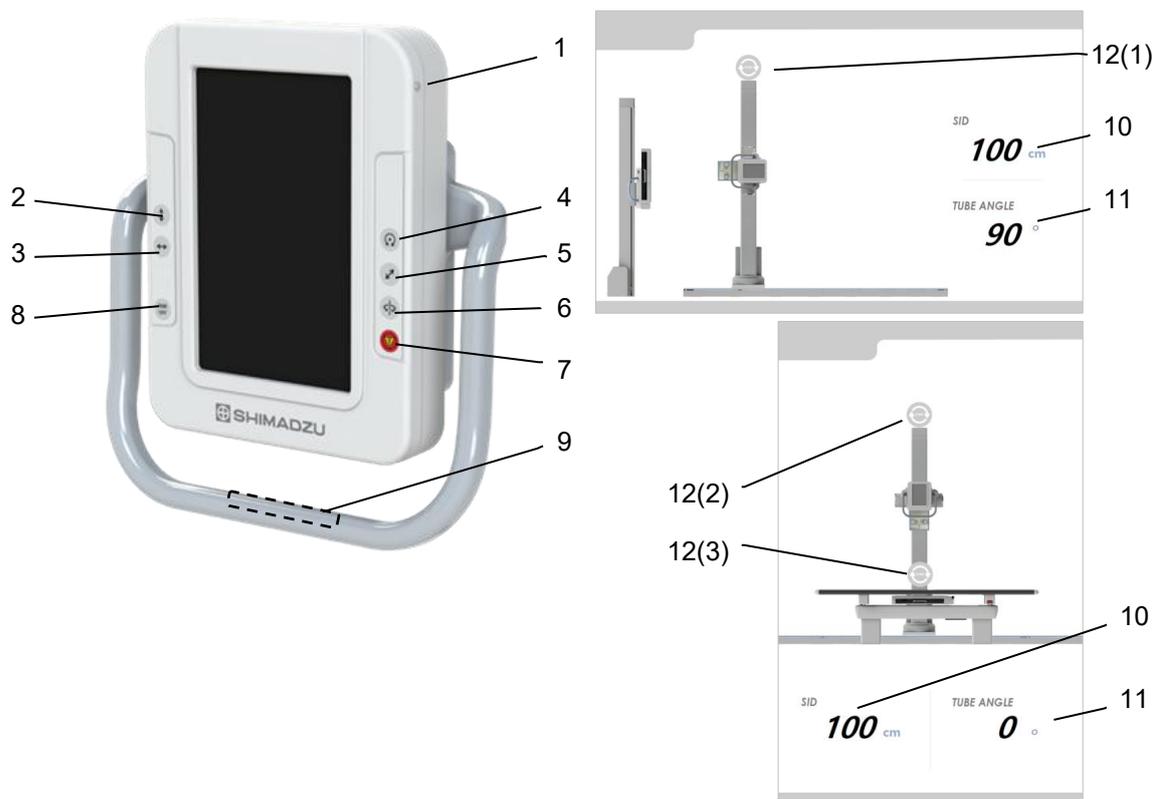

No.	Name	Function
1	Display ON/OFF button	This button turns ON/OFF the display.
2	Vertical release button	This button releases the lock when moving the Tube Stand vertically.
3	Longitudinal release button	This button releases the lock when moving the Tube Stand longitudinally.
4	H-axis rotation release button	This button releases the lock when rotating the Tube Stand around the horizontal axis.
5	Transverse release button	This button releases the lock when moving the Tube Stand transversely.
6	Column rotation release button	This button releases the lock when rotating the Tube Stand column.
7	Emergency button	This button is used for emergency stop.
8	Emergency release button	This button releases the emergency stop.
9	All-free button	This button placed at the rear side of the handle is to release the button longitudinal and vertical direction.
10	SID indicator	This shows the Source Image Distance (SID) between X-ray tube focus and X-ray detector.
11	Rotation angle indicator	This shows the angle at which the Tube Stand is rotated around the horizontal axis.