AUML

AUML-0427	R 6 x 50 mL + Std 1 x 5 mL
AUML-0497	R 1 x 100 mL + Std 1 x 5 mL
AUML-0507	R 6 x 100 mL + Std 1 x 5 mL
AUML-0707	R 4 x 250 mL + Std 1 x 5 mL
AUML-0420	R 6 x 50 mL
AUML-0500	R 6 x 100 mL
AUML-0250	R 12 x 20 mL
☞ AUML-5405	R 50 mL
☞ AUML-5505	R 100 mL
☞ AUML-5710	R 250 mL
☞ AUML-5220	R 20 mL

FTRO-AUML-v25 (06/2023)_ (PIT-AUML-4-v25)

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☞ BIBLIOGRAFIE

1. Panteghini, M., Bais, R., *Enzymes, Tietz Fundamentals of Clinical Chemistry*, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E., (W.B. Saunders eds.), (2008), 317.
2. Wu, A.H.B., *Tietz Clinical guide to laboratory test*, 4th Ed., (W.B. Saunders eds.), (2006), 64.
3. Dufour, R., *The liver: Function and Chemical Pathology, Clinical Chemistry: Theory Analysis, Correlation*, 5th Ed., Kaplan, L.A., Pesce, A.J., (Mosby Inc. eds.), (2010), 586 and appendix.
4. Schumann, G., et al., *Clin Chem Lab Med.*, (2002), **40**, 718.
5. Guder, W.G., et al., *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2.
6. Berth, M. & Delanghe, J., *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, *Acta Clin Belg.*, (2004), **59**, 263.
7. Young, D.S., *Effects of preanalytical variables on clinical laboratory tests*, 2nd Ed., AACC Press, (1997).
8. Young, D.S., *Effects of drugs on clinical laboratory tests*, 4th Ed., AACC Press, (1995).

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☞ BIBLIOGRAFIE

1. Wu, A.H.B., Clinical guide to laboratory tests, 4th Ed., (W.B. Saunders eds.), (2006), 66.
2. Dufour D. R., The Liver: Function and Chemical Pathology. Clinical Chemistry: Theory, Analysis, Correlation, 5th Ed., Kaplan, L.A, Pesce, A.J., (Mosby Inc. eds), (2010), 586 and appendix.
3. Johnson, A. M., Amino Acids and Proteins. Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E. (W.B. Saunders eds.), (2008), 286.

4. Doumas, B. T., et al., Albumin standards and the measurement of serum albumin with bromocresol green, Clin Chim Acta (1971), **31**, 87.

5. Berth, M. & Delanghe, J. Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature, Acta Clin Belg., (2004), **59**, 263.

6. Young, D.S., Effects of preanalytical variables on clinical laboratory tests, 2nd edition, AACC Press (1997).

7. Young D.S., Effects of drugs on clinical laboratory tests, 4th edition, AACC Press (1995).

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BIBLIOGRAFIE

1. Higgins, T., et al., *Hemoglobin, Iron, and Bilirubin*, Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A., Ashwood, E.R. Bruns, D.E. (W.B. Saunders eds.), (2008), 509.
2. Dufour, D.R. *The liver: Function and Chemical Pathology*, Clinical Chemistry: Theory, Analysis, Correlation, 5th Ed., Kaplan, L.A, Pesce, A.J., (Mosby Inc. eds.), (2010), 586 and appendix.
3. Guder, W.G., et al., *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2
4. Wu, A.H.B., Clinical guide to laboratory tests, 4th Ed., (W.B. Saunders eds.), (2006), 172.
5. Howard, C.R., *Breastmilk & Breastfeeding Jaundice*, Pediatric Clinical Advisor: Instant Diagnosis and Treatment, 2nd Ed., Garfunkel, L.C., Kaczorowski, J.M. & Christy, C., (Mosby Inc. eds.), (2007), 82.
6. Berth, M. & Delanghe, J. *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, Acta Clin Belg., (2004), **59**, 263.
7. Young, D.S., Effects of preanalytical variables on clinical laboratory tests, 2nd edition, AACC Press (1997).
8. Young D.S., Effects of drugs on clinical laboratory tests, 4th edition, AACC Press (1995).

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BIBLIOGRAFIE

1. Higgins, T., *et al.*, *Hemoglobin, Iron, and Bilirubin*, Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A., Ashwood, E.R. Bruns, D.E. (W.B. Saunders eds.), (2008), 509.
2. Dufour, D.R. *The liver: Function and Chemical Pathology*, Clinical Chemistry: Theory, Analysis, Correlation, 5th Ed., Kaplan, L.A, Pesce, A.J., (Mosby Inc. eds.), (2010), 586 and appendix.
3. Guder, W.G., *et al.*, *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2
4. Wu, A.H.B., Clinical guide to laboratory tests, 4th Ed., (W.B. Saunders eds.), (2006), 172.
5. Howard, C.R., *Breastmilk & Breastfeeding Jaundice*, Pediatric Clinical Advisor: Instant Diagnosis and Treatment, 2nd Ed., Garfunkel, L.C., Kaczorowski, J.M. & Christy, C., (Mosby Inc. eds.), (2007), 82.
6. Berth, M. & Delanghe, J. *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, Acta Clin Belg., (2004), **59**, 263.
7. Young, D.S., Effects of preanalytical variables on clinical laboratory tests, 2nd edition, AACC Press (1997).
8. Young D.S., Effects of drugs on clinical laboratory tests, 4th edition, AACC Press (1995).

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BIBLIOGRAFIE

1. Rifai, N., Warnick, G.R., Remaley, A.T., *Lipids, lipoproteins, apolipoproteins and other cardiovascular risk factors*. Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E. (W.B. Saunders eds.), (2008), 402.
2. Burnett, J.R., *Coronary Artery Disease: Lipid metabolism*. Clinical Chemistry: Theory, Analysis, Correlation, 5th Ed., Kaplan, L.A., Pesce, A.J., (Mosby Inc. eds.), (2010), 691 and appendix.
3. Langlois, M.R., *et al.* for the European Atherosclerosis Society (EAS) and the European Federation of Clinical Chemistry and Laboratory Medicine (EFLM) Joint Consensus Initiative, Clin. Chem. Lab. Med., (2020), **58**, 496.
4. Allain, C.C., *et al.*, Clin. Chem., (1974), **20**, 470.
5. Guder, W.G., *et al.*, *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2.
6. Berth, M. & Delanghe, J., *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, Acta Clin Belg., (2004), **59**, 263.
7. Young, D.S., Effects of preanalytical variables on clinical laboratory tests, 2nd Ed., AACC Press, (1997).
8. Young D.S., Effects of drugs on clinical laboratory tests, 4th Ed., AACC Press, (1995).

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Attention User: Before using this product, please read the following important information:

The Creatinine Jaffe kit (CRCO-0600) you have received has being modified.

- For the ELITechGroup ECS recommended procedure, the order of addition of reagents has changed. The Sodium Hydroxide (clear /colourless reagent) is now added first (R1). The Alkaline Picrate (yellow coloured reagent) is now the second reagent (R2) added.
- Labelling has been adapted accordingly, and the red hazard pictogram is now located on the R1 bottle (Sodium hydroxide).
- Place “R1” in first position and “R2” in second position on the Selectra Pro analyser.
- The change in the order of reagent addition is not compatible with older lots of aqueous standard CREN-4050, and vice versa. It is necessary to use the standard provided in this kit.
- This modification requires the uploading of a modified instrument application, which is available in the Documentation section of the ELITechGroup website.
- Adjusted values for Elical and Elitol I & II are also provided hereafter and in the Documentation section of the ELITechGroup website.
- All elements of the CREATININE JAFFE system (reagent, standard, application & control recoveries) need to be updated at the same time.

The modification does NOT impact those users that are combining Reagent 1 and Reagent 2 prior to use (mono reagent procedure).

ELICAL 2 - values for "CRCO improved"				
Elical	vial lot	kit lot	expiration	value
ELICAL 2	R200385	03-3130	2022-03	4.03
ELICAL 2	R200946	03-3140	2022-05	
ELICAL 2	R210511	03-3150	2023-01	4.03
ELICAL 2		03-3160	2023-01	
ELICAL 2	R211194	03-3170	2023-06	4.01

ELITROL 1 - values for "CRCO improved"				
Elitrol	vial lot	kit lot	expiration	target(range)
Elitrol 1	R200718 R201112	01-1110	2022-05	0.84(0.69-0.99)
Elitrol 1	R210334 R210590	01-1120 01-1130	2022-10	0.87(0.71-1.03)
Elitrol 1	R210858	01-1140	2022-12	0.87(0.71-1.03)
Elitrol 1	R211203	01-1150	2023-08	0.92(0.75-1.09)

ELITROL 2 - values for "CRCO improved"				
Elitrol	vial lot	kit lot	expiration	target(range)
Elitrol 2	R200719 R201113	02-2110	2022-05	3.27(2.68-3.86)
Elitrol 2	R210335 R210591	02-2120 02-2130	2022-10	3.38(2.77-3.99)
Elitrol 2	R210859	02-2140	2022-12	3.38(2.77-3.99)
Elitrol 2	R211205	02-2150	2023-08	3.37(2.76-3.98)

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BIBLIOGRAFIE

- Kaplan, J.M., First, M.R., *Renal Function. Clinical Chemistry: Theory, Analysis, Correlation*, 5th Ed., Kaplan, L.A, Pesce, A.J., (Mosby Inc. eds), (2010), 567 and appendix.
- Delaney, M.P., Price, C.P., Lamb, E.J., *Kidney Function and Disease. Tietz Fundamentals of Clinical Chemistry*, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E. (W.B. Saunders eds.), (2008), 631.
- Wu, A. H. B., *Clinical guide to laboratory tests*, 4th Ed., (W.B. Saunders eds.), (2006), 316.
- Vasiliades, J., *Clin. Chem.*, (1976), **22**, 1664.
- Guder, W.G., *et al.*, *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2.
- Cerioti, F. *et al.*, *Reference Intervals for Serum Creatinine Concentrations: Assessment of Available Data for Global Application, Clin. Chem.*, (2008), **54**, 559.
- Young D.S., *Effects of drugs on clinical laboratory tests*, 4th Ed., AACC Press (1995).
- Berth, M. & Delanghe, J. *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature, Acta Clin Belg.*, (2004), **59**, 263.
- Young, D.S., *Effects of preanalytical variables on clinical laboratory tests*, 2nd Ed., AACC Press (1997).

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English - EN

INTENDED USE

This *in vitro* diagnostic device is intended for the calibration of ELITech Clinical Systems quantitative tests listed in the value sheet.
This *in vitro* diagnostic device is for professional use only.

COMPOSITION

- Lyophilized product prepared from human serum spiked with chemical and biological additives.
- Sodium azide < 0.1 % (w/w)
- Concentrations for each analyte to test are lot-specific.

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TRACEABILITY

The traceability is indicated in the value sheet enclosed in the kit.

PRECAUTIONS FOR USE AND WARNINGS

- Each unit of human blood used in the manufacture of these products was tested and found to be negative/non-reactive for the presence of HbsAg, HCV and HIV1/2. The methods used were FDA approved or cleared in compliance with European Directive 98/79/EC, Annex II, List A. Nevertheless, since the risk of infection cannot be fully excluded these products must be handled as potentially infectious. In case of exposure, follow the guidelines of the competent health authorities.
- Take precautions when handling broken glass vials as sharp edges can injure the user.
- These products contain sodium azide which may react with lead or copper plumbing to form potentially explosive metal azides. When disposing of these reagents always flush with copious amounts of water to prevent azide buildup.
- Take normal precautions and adhere to good laboratory practice.
- Use clean or single use laboratory equipment only to avoid contaminations.
- Consult Safety Data Sheet (SDS) for a proper handling.

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PREPARATION

- Carefully open the vial, avoiding the loss of lyophilizate.
- Add in exactly 3 mL of distilled or deionized water.
- Carefully close the vial and dissolve the contents completely within 30 minutes by occasional gentle stirring avoiding the formation of foam.

PRODUCT DETERIORATION

- The product may present a slightly hazy appearance after reconstitution. This has no effect on the performances of the product. All presence of particulates would indicate deterioration.
- Do not use the product if there is visible evidence of contamination or damage (e.g. particle matter after reconstitution).
- Damage to the container may impact on product performance. Do not use the product if there is physical evidence of deterioration (e.g. leakages).

STABILITY

Prior to reconstitution :

- Stored at 2-8 °C and protected from light.
- Do not use after expiration date indicated on the vial label.

After reconstitution :

- These products should be immediately and tightly capped to prevent contamination and evaporation.

- Stability of the components:
Between 15 and 25 °C : 8 hours
Between 2 and 8 °C : 2 days
Between -25 and -15 °C : 4 weeks (when frozen once)

Exceptions:

- Stability of total bilirubin (when stored protected from light):
Between 15 and 25 °C : 6 hours
Between 2 and 8 °C : 1 day
Between -25 and -15 °C : 2 weeks (When frozen once)
- Stability of direct bilirubin (when stored protected from light):
Between 15 and 25 °C : 3 hours
Between 2 and 8 °C : 8 hours
Between -25 and -15 °C : 2 weeks (when frozen once)

Note: Store vials tightly capped and protected from light.

PROCEDURE

To use ELICAL 2, follow the procedure described in the instructions for use of the ELITech Clinical Systems reagent used.

LIMITATIONS

Using ELICAL 2 has been validated with the ELITech systems (Analyzers and reagents listed in the value sheet).

Using any other system should be validated by the laboratory.

VALUES

The concentrations are indicated in the value sheet enclosed in the kit.
For users of Selectra instruments allowing data import for tests, calibrators and controls, use corresponding barcode, available on the value sheet.

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BIBLIOGRAFIE

1. Sacks, D.B., *Carbohydrates*. Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A, Ashwood, E.R., Bruns, D.E. (W.B. Saunders eds.), (2008), 373.
2. Dods, R.F., *Diabetes Mellitus*. Clinical Chemistry : Theory, Analysis, Correlation, 5th Ed., Kaplan, L.A, Pesce, A.J., (Mosby Inc. eds.), (2010), 729 and appendix
3. Wu, A. H. B., Clinical guide to laboratory tests, 4th Ed., (W.B. Saunders eds.), (2006), 444.
4. World Health Organization (WHO), *Definition and Diagnosis of Diabetes Mellitus and Intermediate Hyperglycemia*, (2006).
5. Trinder, P., Ann. Clin. Biochem., (1969), 6, 24.
6. Guder, W.G., *et al.*, *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2.
7. Berth, M. & Delanghe, J., *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, Acta Clin Belg., (2004), 59, 263.
8. Young, D.S., Effects of preanalytical variables on clinical laboratory tests, 2nd Ed., AACC Press, (1997).
9. Young, D.S., Effects of drugs on clinical laboratory tests, 4th Ed., AACC Press, (1995).

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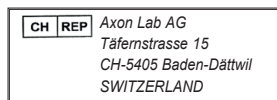
**English - EN****INTENDED USE**

This *in vitro* diagnostic device is intended for the calibration of VitalScientific reagents, used for HDL or LDL cholesterol quantitative determination. This *in vitro* diagnostic device is for professional use only.

COMPOSITION

- Lyophilized product prepared from human serum.
- The HDL and LDL concentrations of this calibrator are lot-specific.

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- VitalScientific reagent, used for HDL or LDL cholesterol quantitative determination (Ref. CHDL-XXXX or CLDL-XXXX)
- General Laboratory equipment.

TRACEABILITY

HDL LDL CALIBRATOR is traceable to the reference method approved by the CDC (Centers for Disease Control and Prevention).

PRECAUTIONS FOR USE AND WARNINGS

- Consult Safety Data Sheet (SDS) for a proper handling.
- Each unit of human blood used in the manufacture of these products was tested and found to be negative/non-reactive for the presence of HbsAg, HCV and HIV1/2. The methods used were FDA approved or CE compliant. Nevertheless, since the risk of infection cannot be fully excluded these products must be handled as potentially infectious. In case of exposure, follow the guidelines of the competent health authorities.
- Take precautions when handling broken glass vials as sharp edges can injure the user.
- Take normal precautions and adhere to good laboratory practice.
- Use clean or single use laboratory equipment only to avoid contaminations.

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PREPARATION

- Carefully open the vial, avoiding the loss of lyophilized.
- Add exactly 1 mL of distilled or deionised water.
- Carefully close the vial and dissolve the contents completely by occasional gentle stirring avoiding the formation of foam.
- Keep at room temperature for 30 minutes before use.

PRODUCT DETERIORATION

- The product may present a slightly hazy appearance after reconstitution. This has no effect on the performances of the product. All presence of particles would indicate deterioration.
- Do not use the product if there is visible evidence of contamination or damage (e.g. particle matter after reconstitution).
- Damage to the container may impact on product performance. Do not use the product if there is physical evidence of deterioration (e.g. leakages).

STABILITY**Prior to reconstitution :**

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After reconstitution :

- These products are stable 2 weeks at 2-8 °C or 10 weeks at -20°C.
- These products should be immediately and tightly capped to prevent contamination and evaporation.

PROCEDURE

To use HDL LDL CALIBRATOR, follow the procedure described in the instructions for use of the VitalScientific reagent used.

LIMITATIONS

Using HDL LDL CALIBRATOR has been validated with the VitalScientific systems (Analyzers and reagents used for HDL or LDL cholesterol quantitative determination (Ref. CHDL-XXXX or CLDL-XXXX)). Using any other system should be validated by the laboratory.

VALUES

The concentrations are indicated in the value sheet enclosed in the kit. For users of Selectra instruments allowing data import for tests, calibrators and controls, use the corresponding barcode available on the value sheet.

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- Symbols used on our documentation are defined on ISO-15223-1 standard, except for some presented in the symbols glossary available on the VitalScientific Website. (Symbols glossary).

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IMPORTANT NOTIFICATION

Adjustment of LDL CHOLESTEROL value in HDL LDL CALIBRATOR (ref HLCA-0041)

Document reference: ADD-CLDL-MP-1.1/2021-09

Dear Selectra users,

In order to maintain performances, the LDL CHOLESTEROL value in HDL LDL CALIBRATOR (ref HLCA-0041) has been adjusted for the LDL CHOLESTEROL reagent (ref CLDL-xxxx) **from lot 21-0894 and greater.**

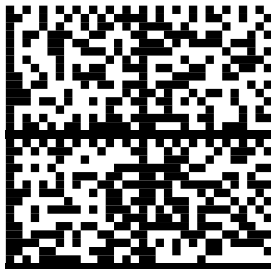
	Reagent lots before lot 21-0894	Reagent lots from lot 21-0894
LDL CHOLESTEROL value in HLCA lot 20-4005	60 mg/dL 1.54 mmol/L	66 mg/dL 1.71 mmol/L
LDL CHOLESTEROL value in HLCA lot 21-4019	62 mg/dL 1.61 mmol/L	67 mg/dL 1.73 mmol/L

TouchPro users on Selectra PRO can scan the barcode on the back.

Results obtained previously with former lots of LDL CHOLESTROL and former value of LDL CHOLESTEROL in HLCA are not questioned.

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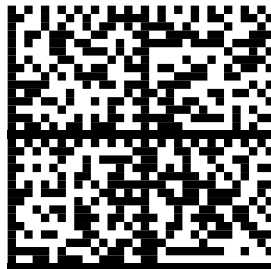
BIBLIOGRAFIE

1. Rifai, N., Warnick, G.R., Remaley, A.T., *Lipids, lipoproteins, apolipoproteins and other cardiovascular risk factors*. Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E. (W.B. Saunders eds.), (2008), 402.
2. Burnett, J.R., *Coronary Artery Disease: Lipid metabolism*. Clinical Chemistry: Theory, Analysis, Correlation, 5th Ed., Kaplan, L.A., Pesce, A.J., (Mosby Inc. eds), (2010), 691 and appendix.
3. Expert Panel on Detection, Evaluation, and Treatment of High Blood Cholesterol in Adults (Adult Treatment Panel III), JAMA, (2001), **285**, 2486.
4. Wu, A.H.B., Clinical guide to laboratory tests, 4th Ed., (W.B. Saunders eds.), (2006), 684.
5. Ahmadraji, T. & Killard, A., Analytical Methods, (2013), **5**, 3612.
6. Langlois, M.R., *et al.* for the European Atherosclerosis Society (EAS) and the European Federation of Clinical Chemistry and Laboratory Medicine (EFLM) Joint Consensus Initiative, Clin. Chem. Lab. Med., (2020), **58**, 496.
7. Berth, M. & Delanghe, J., *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, Acta Clin Belg., (2004), **59**, 263.
8. Young, D.S., Effects of preanalytical variables on clinical laboratory tests, 2nd Ed., AACC Press, (1997).
9. Young, D.S., Effects of drugs on clinical laboratory tests, 4th Ed., AACC Press, (1995).

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BIBLIOGRAFIE

1. Dufour D. R., *The Liver: Function and Chemical Pathology. Clinical Chemistry: Theory, Analysis, Correlation*, 5th Ed., Kaplan, L.A, Pesce, A.J., (Mosby Inc. eds), (2010), 586 and appendix.
2. Wu, A.H.B., *Clinical guide to laboratory tests*, 4th Ed., (W.B. Saunders eds.), (2006), 916.
3. Dumas, B.T., *et al.*, *Clin. Chem.*, (1981), **27**, 1642.
4. Guder, W.G., *et al.*, *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*.(2002). WHO/DIL/LAB/99.1 Rev.2.
5. Berth, M. & Delanghe, J. *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, *Acta Clin Belg.*, (2004), **59**, 263.
6. Young, D.S., *Effects of preanalytical variables on clinical laboratory tests*, 2nd Ed, AACC Press (1997).
7. Young D.S., *Effects of drugs on clinical laboratory tests*, 4th Ed, AACC Press (1995).

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English - EN**INTENDED USE**

These *in vitro* diagnostic devices are intended for the quality control of the performances of ELITech Clinical Systems quantitative tests listed in the value sheet. These *in vitro* diagnostic devices are for professional use only.

COMPOSITION

- Lyophilized product prepared from human serum spiked with chemical and biological additives.
- Concentrations for each analyte to test are lot-specific.

MATERIALS REQUIRED BUT NOT PROVIDED

- ELITech Clinical Systems reagents listed in the value sheet.
- General Laboratory equipment.

TRACEABILITY

The traceability is indicated on the instructions for use of the ELITech Clinical Systems reagent used in combination with the recommended calibrator.

PRECAUTIONS FOR USE AND WARNINGS

- Each unit of human blood used in the manufacture of these products was tested and found to be negative/non-reactive for the presence of HBsAg, HCV and HIV1/2. The methods used were FDA approved. Nevertheless, since the risk of infection cannot be fully excluded these products must be handled as potentially infectious. In case of exposure, follow the guidelines of the competent health authorities.
- Take precautions when handling broken glass vials as sharp edges can injure the user.
- Take normal precautions and adhere to good laboratory practice.
- Use clean or single use laboratory equipment only to avoid contaminations.
- Consult Safety Data Sheet (SDS) for a proper handling.

WASTE MANAGEMENT

Disposal of all waste material should be in accordance with local, state and federal regulatory requirements (please refer to the Safety Data Sheet (SDS)).

PREPARATION

- Gently tap the vial so that the control material can be at the bottom of the vial.
- Carefully open the vial, avoiding the loss of lyophilized.
- Add exactly 5 mL of distilled or deionized water.
- Carefully close the vial and dissolve the contents completely within 30 minutes by occasional gentle stirring avoiding the formation of foam.

Important: To reactivate the alkaline phosphatase, allow the reconstituted control serum to stand for one hour at 15-25 °C.

PRODUCT DETERIORATION

- The product may present a slightly hazy appearance after reconstitution. This has no effect on the performances of the product. All presence of particules would indicate deterioration.
- Do not use the product if there is visible evidence of contamination or damage (e.g. particle matter after reconstitution).
- Damage to the container may impact on product

performance. Do not use the product if there is physical evidence of deterioration (e.g. leakages).

STABILITY

Prior to reconstitution :

- Store at 2-8 °C and protect from light. Do not freeze.
- Do not use after the expiry date stated on the vial label.

After reconstitution :

- These products should be immediately and tightly capped to prevent contamination and evaporation.
- Stability of the components:
Between 2 and 8 °C : 5 days
Between -25 and -15 °C : 4 weeks (when frozen once)

Exceptions:

- Stability of total bilirubin (when stored protected from light) :
Between 2 and 8 °C : 24 hours
Between -25 and -15 °C : 5 days (when frozen once)

- Stability of direct bilirubin (when stored protected from light) :
Between 2 and 8 °C : 8 hours
Between -25 and -15 °C : 5 days (when frozen once)

- Stability of alkaline phosphatase:

- Between 2 and 8 °C : 24 hours
Between -25 and -15 °C : 4 weeks (when frozen once)

Note:

- Store the vials tightly capped and protected from light after reconstitution.
- Mix content thoroughly prior to use.
- Thawing frozen specimen at 37°C, and vigorous mixing (vortex), should be avoided as this may denature ferritin.

PROCEDURE

To use ELITROL I & II, follow the procedure described in the instructions for use of the ELITech Clinical Systems reagent used.

LIMITATIONS

Using ELITROL I & II has been validated with the ELITech systems (Analyzers and reagents listed in the value sheet).
Using any other system should be validated by the laboratory.

VALUES

The concentrations are indicated in the value sheet enclosed in the kit.
Individual laboratories may obtain values different from those announced. Technique, equipment and experimental error may produce slightly different values. Each laboratory should determine their own mean values.
For users of Selectra instruments allowing data import for tests, calibrators and controls, use corresponding barcode, available on the value sheet.

DECLARATION OF SERIOUS INCIDENT

Please notify the manufacturer (through your distributor) and competent authority of the Member State of the European union in which the user and/or the patient is established, of any serious incident that has occurred in relation to the device. For other jurisdictions, the declaration of serious incident should be in accordance with local, state and federal regulatory requirements. By reporting a serious incident, you provide information that can contribute to the safety of *in vitro* medical devices.

ASSISTANCE TECHNIQUE

Contact your local distributor or ELITech Clinical Systems SAS (CCsupport@elitechgroup.com).

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BIBLIOGRAFIE

1. Rifai, N., Warnick, G.R., Remaley, A.T., *Lipids, lipoproteins, apolipoproteins and other cardiovascular risk factors*, Tietz Fundamentals of Clinical Chemistry, 6th Ed., Burtis, C.A., Ashwood, E.R., Bruns, D.E. (W.B. Saunders eds.), (2008), 402.
2. Burnett, J.R., *Coronary Artery Disease: Lipid Metabolism*. Clinical Chemistry: Theory, Analysis, Correlation, 5th Ed., Kaplan, L.A., Pesce, A.J., (Mosby Inc. eds.), (2010), 691 and appendix.
3. Expert Panel on Detection, Evaluation, and Treatment of High Blood Cholesterol in Adults (Adult Treatment Panel III), *Executive Summary of the Third Report of the National Cholesterol Education Program (NCEP)*, JAMA, (2001), **285**, 2486.
4. Fossati, P. & Prencipe, L., Clin. Chem., (1982), **28**, 2077.
5. Guder, W.G., *et al.*, *Use of anticoagulants in diagnostic laboratory investigations and stability of blood, plasma and serum samples*. (2002). WHO/DIL/LAB/99.1 Rev.2.
6. Langlois, M.R., *et al.* for the European Atherosclerosis Society (EAS) and the European Federation of Clinical Chemistry and Laboratory Medicine (EFLM) Joint Consensus Initiative, Clin. Chem. Lab. Med., (2020), **58**, 496.
7. Berth, M. & Delanghe, J. *Protein precipitation as a possible important pitfall in the clinical chemistry analysis of blood samples containing monoclonal immunoglobulins: 2 case reports and a review of literature*, Acta Clin Belg., (2004), **59**, 263.
8. Young, D.S., Effects of preanalytical variables on clinical laboratory tests, 2nd Ed., AACC Press (1997).
9. Young D.S., Effects of drugs on clinical laboratory tests, 4th Ed., AACC Press (1995).

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Contactați distribuitorul dumneavoastră local sau ELITech Clinical Systems SAS (ccsupport@elitechgroup.com).