SERIE 1

Frumusetea sta in
lucruri mici

Soluscope

CE
1434

SERIE 1

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Sistemul "All-in-one"

MONITORIZARE LA FIECARE NIVEL

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	Cycle 1 GTA/OPA version	
Type of program	Cleaning /Disinfection	
Type of device concerned	All flexible endoscopes	
Total duration	26 minutes	
Pre-check	Setting and bowl draining self-check	
Leak test	4 control phases: start, middle of cleaning, middle of disinfection and end of cycle	
Prerinse	Number of phases	1
	Duration per phase	15 seconds of cycling
	Amount of water per phase	6 l
	Temperature	water mains temperature /max 50°C
Cleaning	Contact time	100 s
	Amount of product	50 ml (Soluscope EZ/NW)
	Amount of water	13 l
	Temperature	35°C (setpoint)
Rinse	Number of phases	1
	Duration per phase	20 seconds of cycling
	Amount of water per phase	6 l
	Temperature	water mains temperature /max 50°C
Disinfection	Contact time	10 min
	Amount of product	100 ml (Soluscope GTA or OPA)
	Amount of water	13 l
	Temperature	40°C (setpoint) / max 45°C
Final rinse	Number of phases	3
	Duration per phase	20 seconds of cycling
	Amount of water per phase	6 l
	Temperature	water mains temperature /max 50°C
For AL version		
Blowing	Description	30s of blowing into channels
		if option selected: 4.5 s of alcohol injection (60 ml of 70% alcohol solution) and 30 s of additional blowing (channel pressure alarm 15 after 30 s)

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¹Opțional

EC MANUFACTURER'S DECLARATION OF CONFORMITY

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I undersigned, Christian AFFRE, C.E.O. of SOLUSCOPE SAS, declare that the above-mentioned device complies with the relevant provisions according to the European Directive 93/42/EEC concerning Medical Devices as amended, Annex II (with exemption of section 4), as transposed in the French decree N° 2009-482 of 28th April 2009; and particularly, fulfils the Essential Requirements according to the Annex I; consequently, the CE mark is affixed on the above cited device.

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
SOLUSCOPE GTA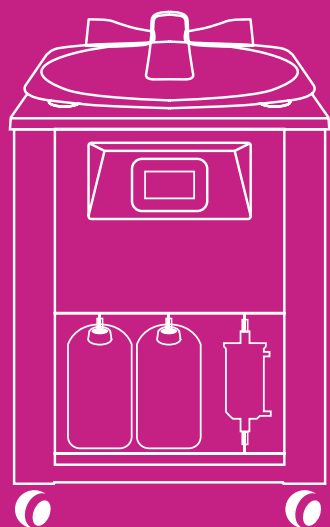

GLUTARALDEHYDE-
BASED FORMULA

BROAD SPECTRUM
ANTIMICROBIAL
ACTIVITY
DEMONSTRATED FROM
10 MINUTES

COMPLIANT WITH
EN ISO 15883-4

COMPATIBLE WITH
THE MAJORITY OF
FLEXIBLE ENDOSCOPES
AVAILABLE ON THE
MARKET

CE
1434

ENDOSCOPE DISINFECTANT

INDICATIONS

- Endoscope disinfection in the *Soluscope Serie 1 GTA* version.
- *Soluscope GTA* is used in combination with *Soluscope EZ* detergent.

COMPOSITION

- Glutaraldehyde 24.8%
- Sequestering agent
- Colouring and excipients

CHARACTERISTICS

- Clear green solution
- Density at 20°C: between 1.02 and 1.10
- pH (pure product): between 3.5 and 4.5
- Stability: 36 months (closed),
2 months (opened)

USER MANUAL

- Insert the container.
- Screw the white cap connected to your *Soluscope Serie 1* onto the container.
- Make sure that the vent hole is not blocked.
- Concentrated solution for professional use.

MICROBIOLOGICAL PROPERTIES

ACTIVE AGAINST	STANDARDS	IN-USE CONDITIONS
Bacteria	EN 13727, EN 14561	1% for 10 minutes at 40°C
Mycobacteria	EN 14348, EN 14563	1% for 10 minutes at 40°C
Yeasts/Molds	EN 13624, EN 14562	1% for 10 minutes at 40°C
Viruses	EN 14476	1% for 10 minutes at 40°C

PRECAUTIONS FOR USE

- Warning: respect the precautions for use (established under the *European Regulations 1272/2008*).
- Storage between 5 °C and 35 °C.
- *Class IIb* medical device (amended *directive 93/42/CEE*).

PACKAGING

- 4 containers of 5L

Soluscope

100, rue du Fauge - Z.I. Les Paluds - 13400 Aubagne - France
☎ +33 (0)4 91 83 21 22 📠 +33 (0)4 91 83 21 10
✉ info@soluscope.com