


# LANCING DEVICE and TWIST LANCET

## TD-5010

TaiDoc's Lancing Device is designed with more comfortable grip, different depth selections, and twist-release cap. It provides more convenient, more comfortable, and easier operation for the users.

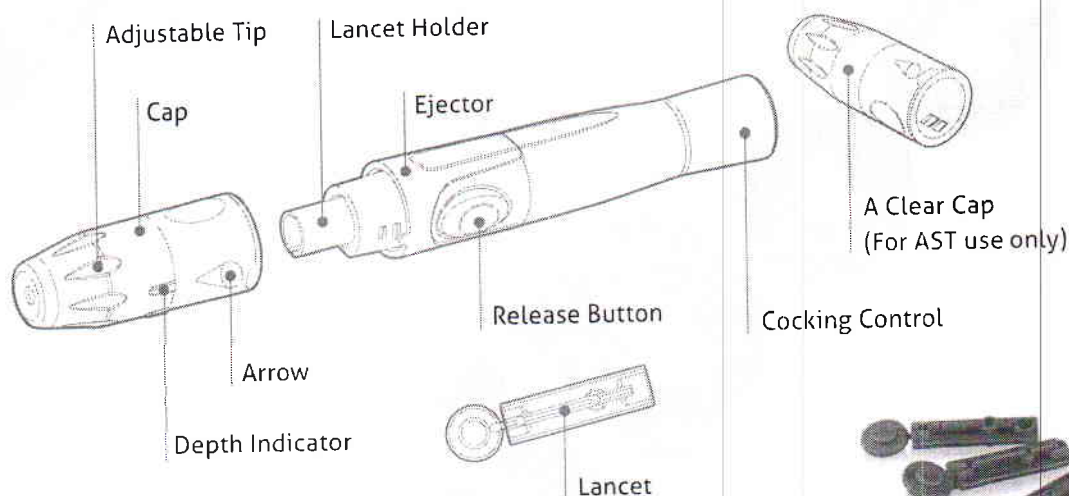

## FEATURES

- Twist-Release provides safer and easier operation for users.
- 6-level depth selection. More noticeable clicks between each level.
- The lancet can be ejected anytime.
- Large contact area – provides steady and comfortable puncture.
- Better adjustable tip grip
- Large holding area on the cap
- Status display on the release button is more visible
- Less vibration less pain

## SPECIFICATIONS

### LANCING DEVICE

Model Number: TD-5010  
 Penetration Depth Setting: 6-level comfort dial  
 Penetration Range: 0.8 mm to 1.8 mm  
 Dimension: 118.7 (L) x 17.5 (D) mm  
 Weight: 16.5 g  
 Priming of Device:  
 Push button to prime with status display window  
 Lancet Ejection: Yes  
 Alternative Site Testing: Yes

### SHIPPING INFO

Units per Carton: 100  
 Gross Weight per Carton: 13 kg  
 Carton size: 43.8 (L) x 35.8 (W) x 41.5 (H) cm



### Quality Compliance

ISO 11137, ISO 11737,  
 ISO 10993-10:2002/Amd 1:2006,  
 ISO 10993-5:2009

### ISTA 2A Compliance

ASTM E171, ASTM D4332,  
 ASTM D642, ASTM D999,  
 ASTM D5276


### TWIST LANCET

Model Number: TD-5084  
 Lancet Gauge: 30G  
 Weight: 0.4g  
 Dimension: 32.6mm x 6.4mm  
 Units per Carton: 100 pcs/box and 140 boxes/carton  
 Net Weight per Carton: 7.62 kg  
 Gross Weight per Carton: 8.62 kg

### SHIPPING INFO

Units per Carton: 140  
 Gross Weight per Carton: 9.8 kg  
 Carton size: 43.8 (L) x 36.8 (W) x 25 (H) cm

# INSTRUCTION FOR USE

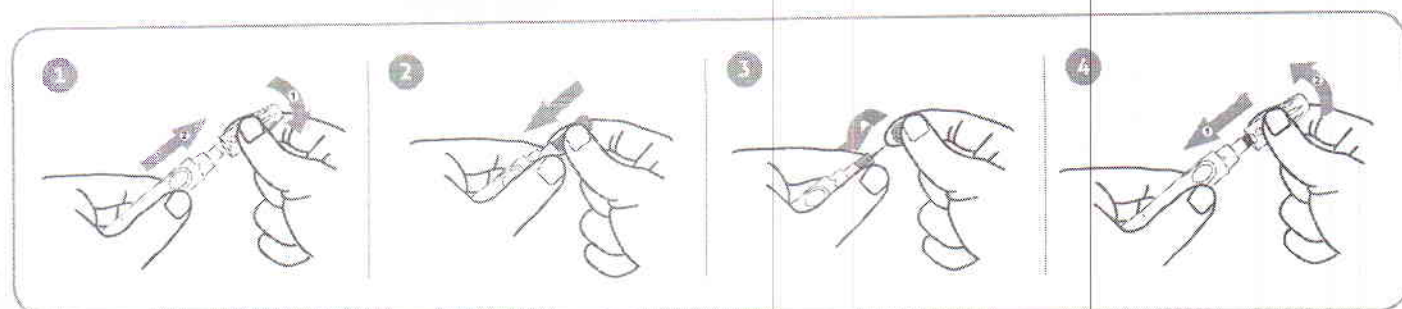


1 Twist and pull off the cap of the lancing device.

2 Insert a lancet into the lancet holder and push down firmly until it is fully secured.

3 Twist the protective disk off the lancet.

4 Replace and twist the cap as shown in the picture.

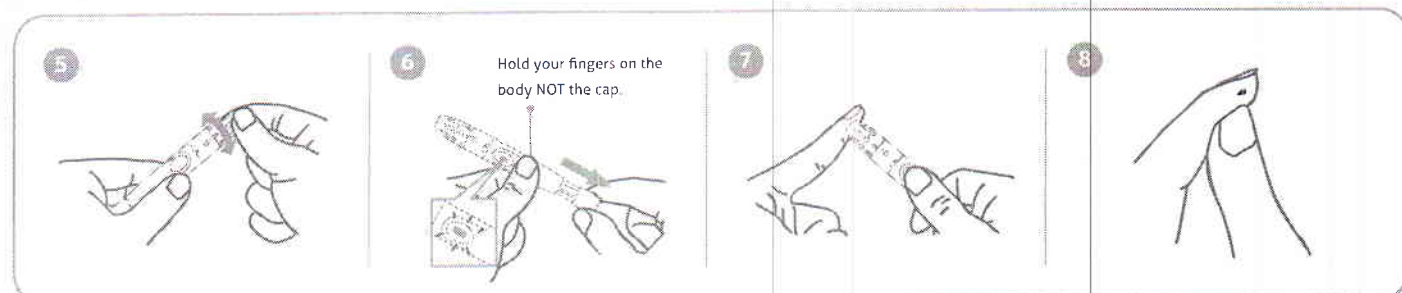


5 Select the depth of penetration by turning the adjustable tip in either direction so that the arrow on the cap points to the desired depth.

6 Pull the cocking control back until it clicks. You will see a color change inside the release button when it is ready.


7 Lancing the finger  
Press the lancing device's tip firmly against the lower side of your fingertip. Press the release button to prick your finger, then a click indicates that the puncture is complete.

8 Remove the device from the puncture site. After penetration, remove the first drop of blood with a clean cotton swab. Gently squeeze the punctured area to obtain another drop of blood. Be careful NOT to smear the blood sample. Apply the blood to the test strip.

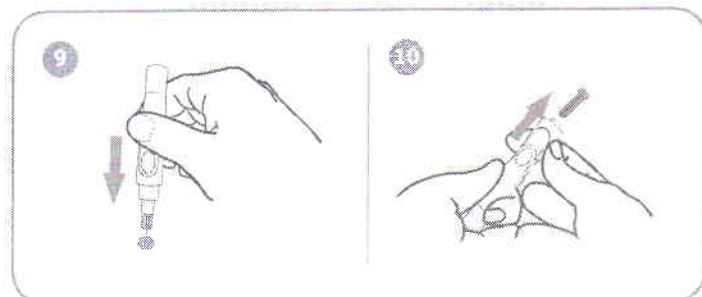

9 Remove the lancet by pulling the cap off first. Safely dispose of the used lancet by placing the protective disk on a hard surface, and pushing the exposed tip into the disk.

10 Slide the ejector forward to remove the used lancet. Dispose of the lancet according to your local regulations.

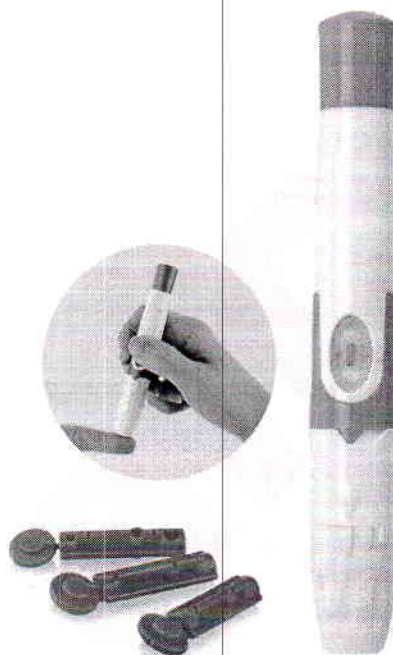

[www.taidoc.com](http://www.taidoc.com)

**TaiDoc Technology Corp.**

81-7F., No.127, Wugong 2nd Rd., Wugu Dist.,  
New Taipei City 24888, Taiwan  
Tel: +886-2-6625-8188 Fax: +886-2-6625-0288