


7.3.5 Illumination


This procedure sets the illumination color and lighting mode during equipment operation.

1 Press the **Illumination** key.

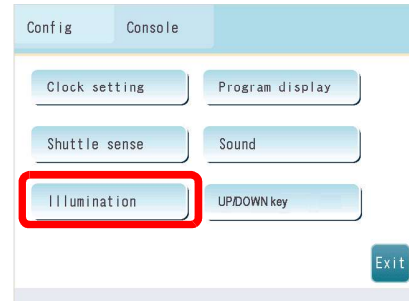

2 Press the color frame.  
The setting items are displayed.


3 Press the key to select the required color.  
The illumination lights in the selected color.

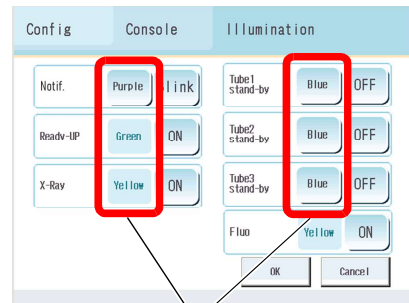

Color frames

4 Press the lighting mode frame.  
The illumination status setting changes each time a key is pressed. Illumination occurs according to the selected illumination status.

5 Press the **OK** key.  
Press the **Cancel** key to proceed without saving the settings.

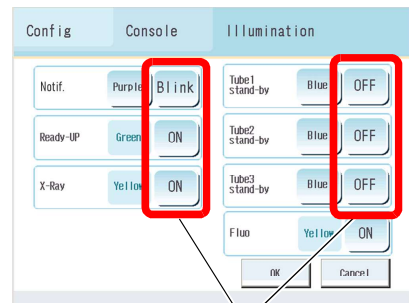

Lighting mode frames



💡 Depending on the installation settings, "Tube 1 stand-by", "Tube 2 stand-by", and "Tube 3 stand-by" can be changed to "Stand Technique", "Table Technique", and "Other Technique".

💡 Depending on the installation settings, the background color of the radiography step screen can be changed to the color set using the Illumination key.