

10.1 Overview

Presets are functions that make the system easy to use with the default settings. In the default settings, the system includes basic settings ([System Preset]), parameter settings dependent on probes ([Application]), and parameter settings not dependent on probes ([Region]). Applications can be added, removed, exported, or imported.

NOTE: Regarding preset data which is not saved by [Save], changes will not be exported. Please save them by selecting [Save] on the changed preset screen.

Presets are composed as follows:

SystemPreset	Sets common system parameters
Application	Parameter settings dependent on the probe
Region	Parameter settings independent of the probe
Measurement	Measurement settings ^{*1}
Filing	Parameter settings related to image output and playback
Input Device	Function assignment for switches and menus
Dictionary	Settings for user and system dictionaries
ColorMap	Color map menu choice settings

^{*1}.

For details, see the separate manual "Measurements 1".

10.2 System presets

System presets set common system parameters.

Procedure

1. Press the [Probe/Preset] key.
2. Select [Preset Setup] on the touch panel.
3. Select [SystemPreset].
4. Select a category.

SystemPreset categories

Classification	Description
General	Basic system parameters
Monitor/Panel	Adjusts the brightness levels of the monitor, operation panel, and the touch panel.
Trackball	Trackball sensitivity setting
Filing	Parameters related to image output
Network Setting	System network settings.
DICOM	DICOM-related settings
Stress Echo	Stress echo protocol settings ^{*1}
Shear Wave Meas.	Shear Wave Measurement-related settings ^{*2}