



Care & Cure Surgico

"Our Creation Is For Your Health"

The good practice of cleaning disinfection of medical equipment and medical-surgical!

Observe the precautions Placement of the product, use with care, Before use, read the label and product information.

Important rule: You wear protective gloves

Cleaning should be done by the continuous action of the product and mixing of the water using a brush.

The flushing is done with clean water.

Drying is carried out using a dry, lint-free.

Verification of instruments is very important to reduce the number of bacteria and inactivate some viruses on contaminated equipment, in order to limit the risk of infection during transport and cleaning of the equipment.

Use :

Allows the sterilization and storage of instruments in an aluminum box.

Box can be heat sterilized or autoclaved.

Check before use if the lid is positioned on the packaging. This box is sealed.

Do not use if the box is damaged, pierced or dirty.

Attention after the remaining instruments are not considered sterile.

Usage tips :

The preparation and sterilization of medical devices should only be performed by trained personnel in these techniques.

Clean, disinfect and sterilize the new product before first use.

Clean, disinfect and sterilize the products as soon as possible after use.

Dispose of used products, corroded, deformed, porous or showing any damage.

Cleaning, disinfection:

Carefully follow the dosage, application time and temperature parameters for disinfection and cleaning.

Avoid overloads in machinery to the worm and ultrasonic devices making sure to place the products without risk of shadows

Never use brushes or metal sponges for manual cleaning.

Rinse thoroughly after washing using deionized water as far as possible.

Dry products thoroughly after rinsing.

Do not forget

sterilization:

Remember that sterilization is not able to replace the preliminary treatments such as cleaning and disinfection. It nevertheless constitutes the indispensable complement.

The recommended sterilization process is an autoclave or wet heat:

- Temperature: 134 ° C

- Pressure: 2 bars

- Duration: 18 minutes

Cool and sterilized products, before use.

The boxes can be recycled to the container for metals.

Care & Cure Surgico, P.O BOX 919, NOAL MORE RORAS ROAD, SIALKOT (51310) PAKISTAN

PH: +92-523-563200 FAX: +92-523-53777

WWW.CARECURESURGICO.COM



ISO 9001: 2008
ISO 13485: 2003

