



Hebei Baiyang Bed Industry Manufactory

Electric Hospital Bed Maintenance Manual

Maintain Items	Inspection	Solution	Period	Maintainer
Appearance cleanliness and integrity inspection	Check, if Stains or corrosive liquids on the bed surface, avoid damage caused by long-time adhering	Use the clcohol to clean the stubborn stain right away	Daily	Medical staff or cleaner
	Check, if epoxy coated knocked or damage, leaking internal steel or rusty	Repair with epoxy coated to prevent the steel from being oxidized	Six weeks	Equipment division workers
screw fasten and integrity inspection	Check, if the screws loosed, losed or corroded, avoid damage to the mechanical structure	Fasten or replace new screws	Six weeks	Equipment division workers
Motor inspection	Check the performance of electrical control functions and electrical wiring	Fasten the wiring or replace the electrical control parts	Six weeks	Equipment division workers
	Check, if the integrity of motors appearance, avoid the abrasion to the motors and electrical wiring	Eliminate the reason of abrasion	Six weeks	Equipment division workers



Hebei Baiyang Bed Industry Manufactory

Side rail function and integrity inspection	Check, if the side rail support component damage or crack cause by mechanical collision and overload usage	Replace the side rail to avoid suddenness	Six weeks	Equipment division workers
Castors property inspection	Check, if the performance of brake, when brake release, the castors can rotate in 360 degree flexible and smooth; when brake the bed cann't move any more	Replace new castors when necessity	Six weeks	Equipment division workers
Head boards appearance and property inspection	Check, when unlock the head boards can take off from the bed body.	Replace the head boards or lock parts when necessity	Six weeks	Equipment division workers