

MAGLUMI[®] Reaction Cup

INTENDED USE

The Reaction Cup is an IVD accessory intended to be used with the MAGLUMI series Fully-auto chemiluminescence immunoassay analyzer and Biolumi series Integrated System.

SUMMARY

The Reaction Cup is as the reaction container when the MAGLUMI assay testing samples. The Reaction Cup will be loaded on cuvette storage drawer of the system.

MATERIALS PROVIDED

REF	Description	Specification
130105000101	Polypropylene	546 Cups/box

WARNINGS AND PRECAUTIONS

- For *in vitro* diagnostic use.
- For professional use only.
- Do not use the product beyond the expiration date indicated on the label.
- Do not re-use the product.
- Disposal of all waste material should be in compliance with local guidelines.
- Note: If any serious incident has occurred in relation to the device, please report to Shenzhen New Industries Biomedical Engineering Co., Ltd. (Snibe) or our authorized representative and the competent authority of the Member State in which you are established.

PROCEDURE

Materials Provided

Reaction Cup.

Materials Required (But Not Provided)

Fully-auto chemiluminescence immunoassay analyzer MAGLUMI X8, MAGLUMI X3 and MAGLUMI X6 or Integrated System Biolumi CX8.







Loading Procedure







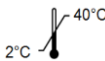

- MAGLUMI X8 and Biolumi CX8 can place a maximum of 16×182 reaction cups in the cuvette storage.
- MAGLUMI X3 can place a maximum of 2×182 reaction cups in the cuvette storage.
- MAGLUMI X6 can place a maximum of 11×182 reaction cups in the cuvette storage.
- For further information, please refer to the appropriate Operators Manual.

STORAGE AND STABILITY

- Unopened at 2-40°C until the expiry date as stated on the labels.
- Protect from direct sunlight.
- After use, reseal the box to protect the Reaction Cup from contamination.

SYMBOLS EXPLANATIONS

	Consult instructions for use		Manufacturer
	Temperature limit (Store at 2-40°C)		Keep away from sunlight
	Catalogue number		Use-by date

	<i>In vitro</i> diagnostic medical device		Batch code
	Authorized representative in the European Community		Kit components
	CE marking		Do not re-use
	Importer		Unique device identifier

MAGLUMI® and Biolumi® are trademarks of Snibe. All other product names and trademarks are the property of their respective owners.




Shenzhen New Industries Biomedical Engineering Co., Ltd.

No.23, Jinxiu East Road, Pingshan District, 518122 Shenzhen, P.R. China

Tel: +86-755-21536601

Fax: +86-755-28292740

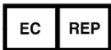

Shanghai International Holding Corp. GmbH (Europe)

Eiffestrasse 80, 20537 Hamburg, Germany

Tel: +49-40-2513175

Fax: +49-40-255726