

- The following message is displayed: "Keep the acoustic output level as low as possible. Refer to ALARA recommendations in the Instruction Manual."
- 2. Select [OK].
 - The ultrasound output power value is highlighted on the screen.
The limit is suspended until the [New Patient] key is pressed. To limit the ultrasound output power again, select [Power Limit Override] again on the touch panel.

4.5 Adjusting audio volume

This function adjusts the volume of the Doppler sound, R-wave beep, and external input audio.

Prior confirmation

Assign [Audio Volume] to the function menu.

Procedure

- Use [Audio Volume] on the touch panel to adjust.
The sound is muted when the volume is set to 0.

4.6 Mode display

4.6.1 Displaying B mode images

Prior confirmation

For a quad-screen view, use the presets to assign [Quad] to a direct switch.
For details on how to assign menu items, see the separate manual "Basic Operations".

Procedure

- Displaying a B mode image (single-screen view).
 - Press the [B] key.
It displays a real-time B mode image (single-screen view).
Pressing the [B] key while in the Freeze state displays the B mode image (single-screen view) in real time.
- Displaying a B mode image (dual-screen view).
 - a. Press the [Dual] key.
 - This displays the active screen in real time and the non-active screen as frozen images.
- Displaying a B mode image (quad screen).
 - a. Select [Quad], which you assigned to a direct switch or a custom switch.
 - This displays the active screen in real time and the others as frozen images.