



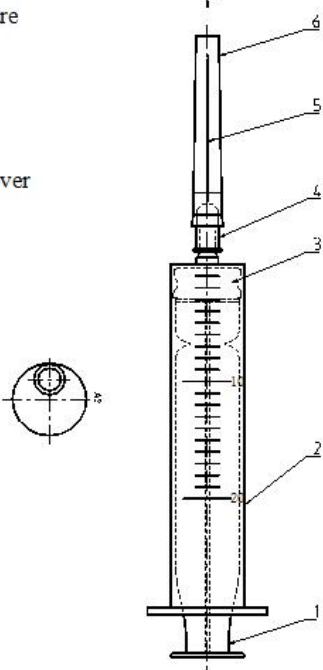
Technical Data Sheet

Revision : 5

Sterile Hypodermic Syringes for Single Use


Date : 2019-07-01

Page 1 of 2

<p>Manufacturer: Changzhou Jiafeng Medical Equipment Co., Ltd</p>	<p>20ML Syringe *With needle ** without needle</p>	<p>Product Structure</p> <ol style="list-style-type: none"> 1. Core Pin 2. Barrel 3. Plunger 4. Needle Seat 5. Needle tube 6. Protective cover 
---	--	---


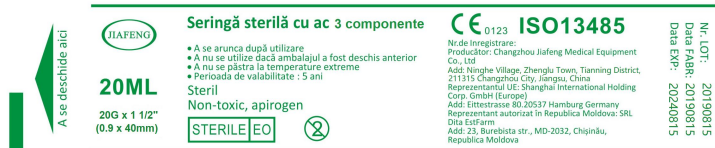
Product variants	Ref	Description	Luer type
	SS311202140	20ML with needle 21G*1 1/2"	Luer slip
	SS311202040	20ML with needle 20G*1 1/2"	Luer slip
Intended use		For hypodermic drug injection giving , extraction.	
Materials		PP, NR, stainless steel, silicone oil (Syringes are PVC- and DEHP-free)	
Needle		Designed with optimal point geometry for maximum comfort and reliability, thin-walled stainless steel, trilateral sharpened and ultrasound polished. Needle surface coated with special grease.	
Sterilization		Ethylene oxide (E.O)	
Shelf life		60months	
Product standards		ISO 7886-1: 2017 , ISO 7864: 2016	
Classification		MDD 93/42/EEC, Class IIa	
Certificates		CE,ISO13485, CFDA, FSC	
Packaging		Single PE bag, 36pcs per middle bag, 20middle bags per carton	
Storage		Store in relative humidity less than 80%, no corrosive gas, cool, dry, well ventilated, clean environment.	



	Technical Data Sheet	Revision : 5
	Sterile Hypodermic Syringes for Single Use	Date : 2019-07-01
		Page 1 of 2

Operation Instruction	<ol style="list-style-type: none"> 1.Open the individual packing and take out the syringe 2.Take off the needle cap, avoid touching with the needle tube 3.Extract the solution and exhaust the air 4.Push the plunger to the top of the barrel to finish fluid injection 5.Throw the used syringe in to to the medical waste box
Cautions	<ul style="list-style-type: none"> *Check the batch number and validity before use. *Tighten the needle before use. *It is strictly prohibited to use if the single packing damaged or needle cap drop off. *For single use only, use immediately after unsealed, destroy after use. It is strictly prohibited to reuse.

Marks	  
-------	---

Packing	<p>* Primary PE bag -Sample PE bag design</p> <div style="border: 1px solid black; padding: 10px; margin: 10px 0;">  </div> <div style="border: 1px solid black; padding: 10px; margin: 10px 0;">  </div>
---------	---

