



OXY'PHARM®

917, rue Marcel Paul • F-94500 Champigny-sur-Marne • +33 1 48 82 58 29 • +33 1 48 82 46 13 • commercial@oxypharm.net

TECHNICAL INSTRUCTIONS

NOCOLYSE

DEFINITION AND CHARACTERISTICS

- **NOCOLYSE®** is a BIO-disinfectant for surfaces.
- **NOCOLYSE®** is a product based on hydrogen peroxide (6%), ready for-use, conceived to be used exclusively with the appliances **NOCOSPRAY®** and **NOCOMAX®**.
- The association **NOCOLYSE®** / **NOCOSPRAY®** (or **NOCOMAX®**) is effective on all the types of microorganisms. The couple enables to make surface disinfection treatments with a bactericidal, virucidal, fungicidal and sporicidal efficiency.
- **NOCOLYSE®** exists in 3 versions: neutral fragrance, mint fragrance and with essential oils destroying smells.

REFERENCES AND PACKAGING

Packaging: 1L, 6x1L, 5L and 20L

Nocolyse Neutral	
Reference	Packaging
4000.001	1 liter
4000.001-6	6x 1 liter box
4000.005	5 liters
4000.020	20 liters

Nocolyse Mint	
Reference	Packaging
4001.001	1 liter
4001.001-6	6x 1 liter box
4001.005	5 liters
4001.020	20 liters

Nocolyse Nocodor	
Reference	Packaging
4030.001	1 liter
4030.001-6	6x 1 liter box
4030.005	5 liters
4030.020	20 liters

www.oxypharm.net

S.A.R.L. au capital de 10 000 Euros • SIREN 448 974 253 • R.C.S. Créteil • APE 4646 Z • N°TVA Intracommunautaire : FR 74 448 974 253



OXY'PHARM®

917, rue Marcel Paul • F-94500 Champigny-sur-Marne • +33 1 48 82 58 29 • +33 1 48 82 46 13 • commercial@oxypharm.net

STORAGE

- Store the product in the original packaging, vertically and in a fresh and well ventilated place.
- Shelf-life:
In the closed original packaging: 2 years from manufacturing date.
Once uncapped: 2 months from opening date

INSTRUCTIONS FOR USE

Preventive use:

- ⇒ Write the opening date on the bottle before first use
- ⇒ Screw the bottle on the appliance NOCOSPRAY® or NOCOMAX®
- ⇒ Set the machine on the volume of the room to be treated
- ⇒ After the end of diffusion, respect a contact time of minimum 30 minutes.

NOTE:

During the diffusion, let the room closed and do not enter in.
Treatment should be conducted with no human presence.

- ⇒ *The treatment has to be renewed every day*

IMPORTANT:

*Achieving a good quality of disinfection is directly linked to the respect of a strict cleaning protocol, carried out before the treatment.
Its maintenance depends on a regular application of preventive treatments.*

PRECAUTIONS FOR USE

Safety Data Sheet of the product, available on our website:

www.oxypharm.net

Version: B

Updated on: 27/10/2011