

Instructions of use sterilization reels and pouches

SURGIPACK reels and pouches are designed for sterilization of medical devices, recommended to be sterilized by steam or ethylene oxide before use.

Recommended precautions:

Keep in a cool, dry place

Do not use if the reel or pouch is damaged

Instructions for use:

Reels (60g/m² paper / film thickness 52um)

1. Cut the appropriate length of reel, please note the device should be less than 3/4 length of the pouch;
2. The device should be clean and dry before put into reel;
3. Get hot one side of the cut piece of reel;
4. Place the medical device within the reel;
5. Sharpe instrument should be put opposite to the open side, so as to ensure the safety when opening the pouches.
6. Get hot the other side of the reel;
7. Put the sealed pouches with the direction of paper to paper. Then place them into sterilizer.
8. Apply the sterilizing agent: STEAM (steam) or EO (ethylene oxide)
9. The recommended hot-sealing temperature: 180°C-190°C
10. Verify the color change of indicator ink:
STEAM Blue - Greenish black
EO Pink - Yellow
11. Store the sterilized pouches in dry, clean places, and pay attention to the direct light.
12. Check the pouches before use, if break, please do not use.