

- **Transfer Syntax:** Sends the study by changing compression (transfer syntax).

Transfer Syntax	Description
Keep Transfer Syntax	Uses existing transfer syntax.
Uncompress	Decompresses the compressed image.
DICOM JPEG 2000 Lossless	Uses JPEG 2000 lossless compression.
DICOM JPEG 2000 Lossy (Maximum)	Lossy compression. (Maximum)
DICOM JPEG 2000 Lossy (High)	Lossy compression. (High)
DICOM JPEG 2000 Lossy (Medium)	Lossy compression. (Medium)
DICOM JPEG Lossless	Uses JPEG lossless compression.
DICOM JPEG Lossy (Maximum)	Lossy compression. (Maximum)
DICOM JPEG Lossy (High)	Lossy compression. (High)
DICOM JPEG Lossy (Medium)	Lossy compression. (Medium)

- **Security Mode:** Sends the study by applying the selected security mode.

Security Mode	Description
TLS 1.0 Secure Mode	Applied TLS 1.0




- The **Default** item set in the server as a default value cannot be modified.
- You cannot send studies searched from database to the **Default** server.


### 3.3.16 Exporting Study

#### Export


- 1 Select search conditions, and search studies you want.

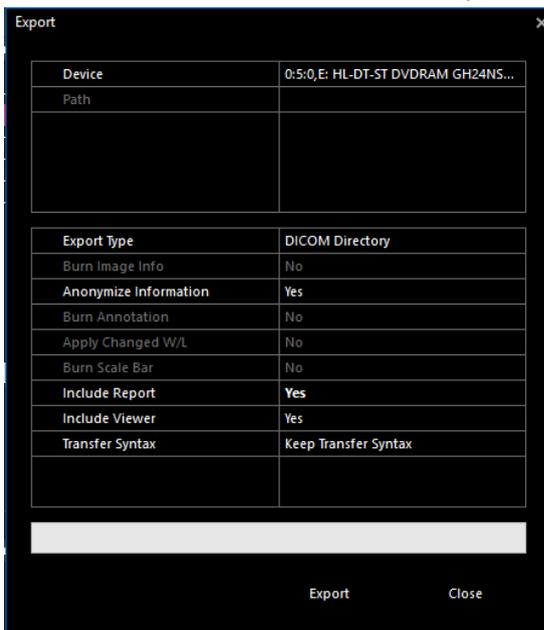

- 2 Click on  button on the toolbar.
- 3 Insert CD/DVD or a USB drive.
- 4 Click **Device** menu to select a desired option. The **Path** menu becomes activated when you select the **User Directory** option, and you can specify a folder to export images.

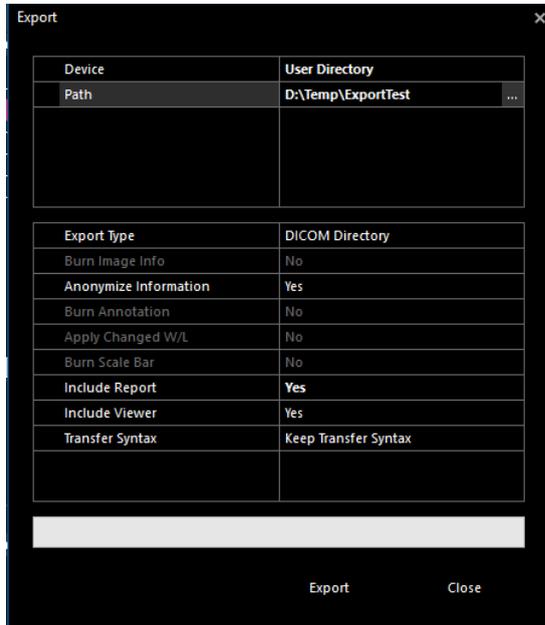

- 5 Click **Export Type** to select options and click on the **Export** button.

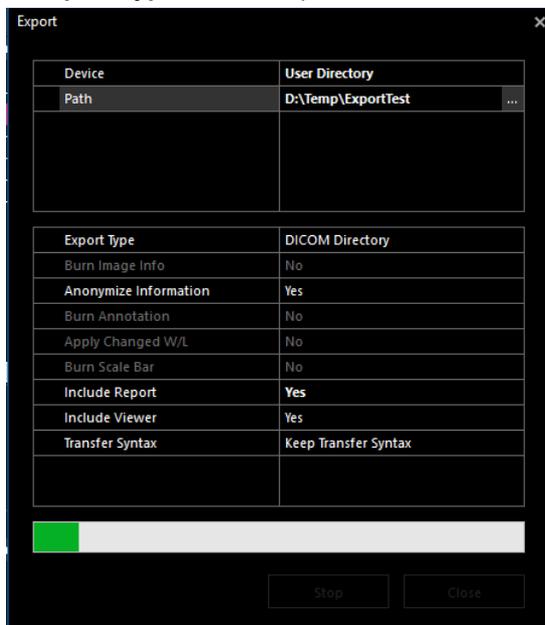

- 6 Check the completion message or the error message.



- The message asking whether to remove the USB device appears when the study is exported to USB.