


[Products](#) / [Atherectomy Systems](#) / ROTAPRO™

## ROTAPRO™

### Rotational Atherectomy System

ROTAPRO System Training

Coronary Calcium Therapies

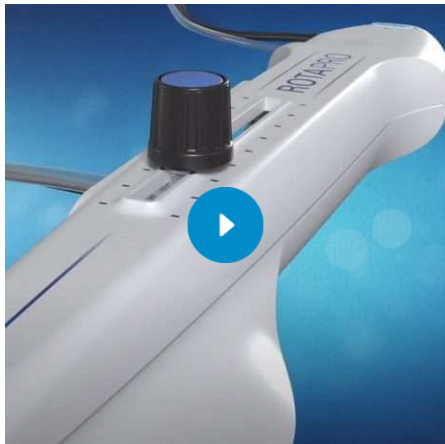

Easy to Use. Hard on Calcium. **ROTAPRO** Rotational Atherectomy System is the gold standard mechanism of action on an easy to use platform.

RECEIVE UPDATES



– [Indications, Safety, and Warnings](#)



– [Request a Sales Rep](#)



– [Modern PCI Prep Portfolio](#)



– [See the ROTA Data](#)



– [Interactive Education on EDUCARE](#)



ROTAPRO Product Animation video

### Explore

[System Components](#) ▼

[Ordering Information](#) ▼

[Reimbursement](#) ▼

### System Components

#### Console

The **ROTAPRO** Rotational Atherectomy System console regulates the flow of air to the advancer, controlling burr rotation speed. It also monitors and displays burr rotation speed and rotational atherectomy procedural time. The console may be powered by both nitrogen and compressed air.



- Vibrant Digital Display – Enhanced feedback and deceleration indicator
- Streamlined Connections – Quick and easy setup
- IV Pole Clamp – Installation flexibility

#### Advancer

The **ROTAPRO** Rotational Atherectomy system features an Advancer which is used to control movement of the burr. The advancer can be purchased individually.



- Easy-to-use Controls – Incorporated on the advancer
- Ergonomic Dynaglide™ Design – Simple system removal
- Hybrid Harness – Improved cable management


Burrs

The ROTAPRO Atherectomy System offers multiple burr size options to achieve predictable lumen size.



Guidewires

ROTAWIRE™ Drive Guidewires are designed specifically for use with the ROTAPRO Rotational Atherectomy System.

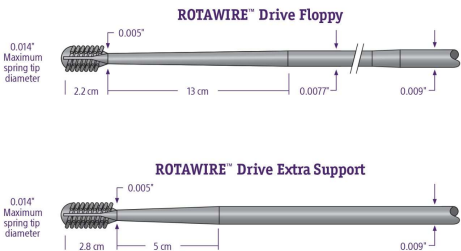

- Steerability – able to navigate calcified lesions with 1:1 torque through tortuous anatomy, providing access for ROTAPRO Rotational Atherectomy System
- Tip Design - .014" platinum coil to provide visibility during Rotation atherectomy
- Core Wire – one-piece core wire with Asahi technology, transmits torque for predictable steering

Choose between the ROTAWIRE™ Drive Floppy or ROTAWIRE™ Drive Extra Support models depending on procedure requirements.

ROTAGLIDE™



ROTAGLIDE is a lipid based emulsion designed to lubricate the ROTAPRO System.\*

- Reduces friction and improves tactile feel
- Reduces heat build-up around the ROTAPRO burr
- Reduces sudden drops in RPMs caused by lesion feedback

\*Contraindicated if patient is allergic to eggs or olive oil

Ordering Information

ROTAPRO System Console

Order Number	Model/ Description
H749 3930901 0	ROTAPRO Console Kit

ROTAPRO Pre-Connected Burr and Advancing Device

Order Number	Model/Description	Burr Size	Length	Maximum Diameter
H749 39467125 0	ROTAPRO Pre-Connected Burr and Advancing Device	1.25 mm	135 cm	0.58 inch
H749 39467150 0	ROTAPRO Pre-Connected Burr and Advancing Device	1.5 mm	135 cm	0.58 inch
H749 39467175 0	ROTAPRO Pre-Connected Burr and Advancing Device	1.75 mm	135 cm	0.58 inch
H749 39467200 0	ROTAPRO Pre-Connected Burr and Advancing Device	2.0 mm	135 cm	0.58 inch
H749 39467215 0	ROTAPRO Pre-Connected Burr and Advancing Device	2.15 mm	135 cm	0.58 inch
H749 39467225 0	ROTAPRO Pre-Connected Burr and Advancing Device	2.25 mm	135 cm	0.58 inch
H749 39467238 0	ROTAPRO Pre-Connected Burr and Advancing Device	2.38 mm	135 cm	0.58 inch
H749 39467250 0	ROTAPRO Pre-Connected Burr and Advancing Device	2.5 mm	135 cm	0.58 inch

ROTAWIRE™ Drive Guidewire

Order Number	Model/ Description	Length	Tip Length	Flexibility	Spring Tip Diameter	Maximum Diameter	Quantity
--------------	--------------------	--------	------------	-------------	---------------------	------------------	----------

H749 3946300 5	ROTAWIRE™ Drive Extra Support Guide Wire with WireClip™ Torquer	330 cm	2.8 cm	Stiff	0.009 inch	0.014 inch	Box of 5
H749 3946200 5	ROTAWIRE™ Drive Floppy Guide Wire with WireClip™ Torquer	330 cm	2.2 cm	Flexible	0.009 inch	0.014 inch	Box of 5

Accessories

Order Number	Model/Description	Quantity
H802 2160003 1	Replacement Braided Air Hose	Single
H749 3936801 0	Replacement Power Cord – North America	Single
H802 1590101 1	Regulator Kit – North America	Single
H802 22196003 2	WireClip Torquer	Box of 5

ROTAGLIDE™ Lubricant

Order Number	Model/Description	Quantity
H749 23548-001 62	ROTAGLIDE Lubricant Mixture 20 cc vials	Box of 6

Reimbursement

The C-code used for this product is C1724, catheter, transluminal atherectomy, rotational. The ROTAWIRE™ Drive Guidewire C-code is C1769. C-codes are used for hospital outpatient device reporting for Medicare and some private payers.

Note: Boston Scientific is not responsible for the correct use of codes on submitted claims; this information does not constitute reimbursement or legal advice.





[Visit our GuidePoint reimbursement site for more information](#)

The most complete  
Modern PCI portfolio  
in the industry



Explore products →

Boston Scientific is dedicated to transforming lives through innovative medical solutions that improve the health of patients around the world.



Professionals

- [Medical Specialties](#)
- [Reimbursement](#)
- [EDUCARE – Medical Education and Training Courses](#)
- [Investigator-Sponsored Research Program](#)

About

- [About Us](#)
- [Corporate Responsibility](#)
- [Careers](#)

Patients

- [Patients and Caregivers - Support and Resources](#)
- [All Treatments and Conditions](#)
- [Device Support](#)

Administrators

- [Administrators](#)

Products

- [Products](#)
- [Customer Support](#)
- [ImageReady™ - MRI Information](#)
- [Product Security](#)

Compliance & Ethics

- [Compliance and ethics](#)
- [Policy and Advocacy](#)
- [UK Modern Slavery Disclosure Statement](#)
- [Do not sell or share my personal information](#)

IC-561109-AC